Dry & Clean Charger

MEENERGY™ BTE P/SP/R/iRIC Charger Manual

www.miracle-ear.com

Content

Before you start **4**

Components 6

Connecting the USB power cable 7

Charging **8**

Correct position of the hearing instruments 11

Cleaning and Drying 12

Cleaning status LED 13

Charging status LEDs 13

Charging tips 18

Further information **26**

Maintenance and care 26

Troubleshooting 27

Important safety information **30**

General warnings 30

Personal safety 32

Product safety 38

Important information **40**

Explanation of symbols 40

Operating, transport, and storage conditions 42

Technical data of the power supply adapter 44

Disposal information 44

Conformance information 45

Country-specific information 47

Before you start

This device is intended to charge our specific hearing instruments with built-in power cells (lithium-ion rechargeable batteries). Your Hearing Care Professional will advise you on the compatible models.

The charger is only intended for indoor use. Observe and follow the operating conditions in section "Operating, transport, and storage conditions".

The charger is not intended as a means of storage for transportation of hearing instruments.


CAUTION

Read this user guide thoroughly and completely and follow the safety information in this document to avoid damage or injury.

The device may not look exactly like the guiding illustrations in these instructions. We reserve the right to make any changes we consider necessary.


Components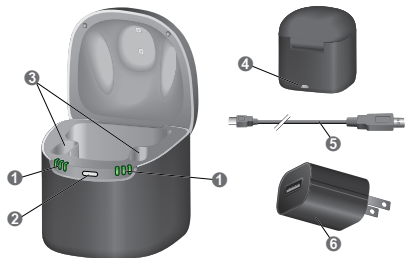

- 1 Status LEDs, charging
- 2 Status LED, UV cleaning
- 3 Charging slots
- 4 USB port for power supply
- 5 USB power cable
- 6 Power adapter

Connecting the USB power cable

- ▶ Connect your USB power cable to the provided power adapter.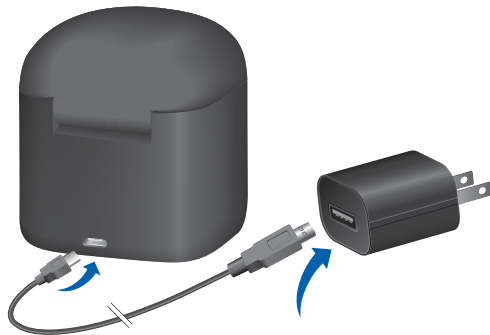

Charging


NOTICE

Please make sure that the power plug is easily accessible to remove it from power supply if necessary.

- ▶ Connect the charger to the power supply.
All LEDs for each charging slot turn on successively indicating that the charger has power and is ready for use.

- ▶ Put the hearing instruments in the charging slots as shown in the picture. You can charge both hearing instruments at the same time.
The hearing instruments are automatically turned off and start charging. The charging status is indicated, see table "Charging status" on the next pages.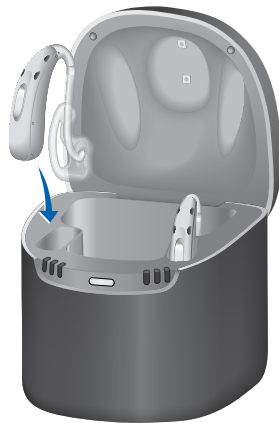

- ▶ When you want to use your hearing instruments, take them out of the charger. They will automatically turn on.
- ▶ You can leave the charger plugged in or remove it from the power supply.

The device may not look exactly like the guiding illustrations in these instructions. We reserve the right to make any changes we consider necessary.

Correct position of the hearing instruments

- ▶ The left and right hearing instruments can be inserted into either charging slot. The slots are not side specific.
- ▶ Make sure the bottom parts of the hearing instruments are placed well into the charging slots. The ear piece of the hearing instrument should always be placed in the compartment. Correct placement will be indicated by the LED.

Cleaning and Drying

Drying and cleaning focuses on tubes, ear tips and molds.

- Put the hearing instruments into the charging slots and close the lid.
UVC cleaning* starts automatically after closing the lid and lasts 15 min
- Ongoing UVC cleaning process is indicated by LEDs (see table "Charging and cleaning status")
- Drying starts automatically after closing the lid and lasts 4 hours.

*Ultraviolet C light, used for hygienic cleansing. Significantly reduces risk for ear infections caused by germs, bacteria and fungi.

Cleaning status LED

LED	Description
	UVC cleaning status LED is slowly blinking white: Cleaning for 15 minutes and only with closed lid.

Charging status LEDs

Left LED	Right LED	Description
		No LED light: The charger is not connected to a power supply. No hearing instrument is inserted or the hearing instrument is inserted incorrectly.
		LEDs rapidly blinking green in sequence (1, 2, 3): The charger is connected to a power supply and ready to charge.

Left LED	Right LED	Description
		1 LED rapidly blinking green 3 times: Charging of the correctly placed hearing instrument is initiated.
		1 LED slowly blinking green: Charging is in progress; battery SOC (State of Charge)* is < 33 %.

*Level of battery charge relative to its capacity.

Left LED	Right LED	Description
		1 LED solid green, 2 nd LED slowly blinking: Charging is in progress; battery SOC is 34-66 %.
		2 LEDs solid green, 3 rd LED slowly blinking: Charging is in progress; battery SOC is 67-99 %.
		3 LEDs solid green: The hearing instrument is fully charged.

Left LED	Right LED	Description
		LEDs blinking rapidly orange in sequence (1, 2, 3): The charger is cooling down.
		All LEDs blinking red 3 times in every 2 sec: Hearing instrument error.

Left LED	Right LED	Description
		LEDs blinking red in a sequence (1, 2): Charger error. Read more in section "Troubleshooting".

Charging tips

You may have some questions on how to best charge your hearing instruments. Read the questions and answers below. If you have more questions, your Hearing Care Professional will help you.

How often do I need to charge the hearing instruments?

We recommend charging the hearing instruments daily, even if the power cell is not empty. Charge them overnight so you can be sure to start your day with fully-charged hearing instruments.

How long does charging take?

A full charging cycle takes approximately 3 to 4 hours.

After 30 minutes, it is possible to use your hearing instruments for up to 7 hours.

Charging times may vary due to the use of the hearing instruments and the age of the power cell.

Can I leave the hearing instruments in the charger?

Even when fully charged, you can leave the hearing instruments in the charger. They automatically stop charging when they are fully charged.

What happens if I forget to charge?

Your hearing instruments will automatically switch-off when the power cell is empty. Charge the hearing instruments as soon as possible. Hearing instruments which have not been charged for over 6 months may be damaged due to an unrecoverable deep-discharge of the power cell. Deep-discharged power cells cannot be charged anymore and have to be replaced.

What should I pay attention to when charging?

The hearing instruments and charger must be clean and dry. Observe the operation conditions in section "Operating, transport, and storage conditions". Avoid charging at high ambient temperatures, as this can shorten the life span and capacity of the power cell. If you use the charger in a very warm environment, note that it will stop charging in order to cool down if the charger's temperature exceeds 45 °C. The LED light will blink 3 times in 2 seconds when this happens. Charging resumes automatically after the device has cooled down. Do not touch the hearing instruments, they may be hot.

What if I intend not to use the hearing instruments for several days or longer?

- Short-term storage (up to several days): Turn off your hearing instruments via the control element on the housing or place them in the powered charger.

The charger must be connected to the power supply. If the charger is not connected to power supply and you place your hearing instruments in the charger, the hearing instruments are not turned off.

Note that when you turn off the hearing instruments via a remote control or smartphone app, the hearing instruments are not turned off completely. They are in standby-mode and still draw some power.

- Long-term storage (weeks, months, ...): First, fully charge your hearing instruments. Turn them off via the control element on the housing before storing them. We recommend to use drying products while storing the hearing instruments.

At 6 month intervals you will have to charge the hearing instruments to avoid an unrecoverable deep-discharge of the power cell. Deep-discharged power cells cannot be charged anymore and they have to be replaced. We recommend recharging more frequently than 6 months intervals.

Observe the storage conditions in hearing instruments' user guide.

What should I pay attention to when using a power strip with energy-saving features?

- Turn on the power strip before you insert the hearing instruments in the charger. Otherwise, the hearing instruments may not be detected by the charger.
- Do not turn off the power strip while the hearing instruments are in charger. Otherwise the hearing instruments will automatically turn on and discharge in the charger.

Before you turn off the power strip, remove the hearing instruments from the charger and turn them off via the control element on the housing.

What happens if I open the lid during the cleaning process?

The cleaning process will be paused. After closing the lid again, the process will resume.

But if you remove the hearing instruments from the charger before end of the cleaning process, the timer will reset.

A new 15 min. cleaning cycle will start after reinserting the hearing instruments and closing the lid.

Further information

Maintenance and care

NOTICE

Risk of injuries caused by electric shock!

- ▶ If cleaning is required, disconnect the charger from the power supply.
- ▶ Clean the charger with a soft, dry tissue.
- ▶ Never use running water or immerse the charger in water.
- ▶ Do not sterilize or disinfect the charger.

- ▶ After use and before placing in the charger, wipe the charging pins on the hearing aids with a soft, dry tissue to remove grease or moisture.

Troubleshooting

Problem	Possible solutions
LED light is off	■ Check if the power supply is connected correctly.

Problem	Possible solutions
LED lights are blinking red 3 times every 2 sec. or LED lights are blinking red in a sequence	<p data-bbox="538 184 1295 288">An error in the hearing instrument or in the charger has occurred.</p> <ul data-bbox="538 308 1295 888" style="list-style-type: none"> <li data-bbox="538 308 1295 453">■ Take the hearing instruments out of the charger and disconnect the charger from the power supply. <li data-bbox="538 474 1295 619">■ Reconnect the charger to the power supply and re-insert the hearing instruments in the charging slots. <li data-bbox="538 640 1295 888">■ Charger has reached the temperature limit and needs to cool down. Charging is resumed when the temperature decreases below 45 °C.

Problem	Possible solutions
If the LEDs are still blinking red	<ul data-bbox="1980 184 2751 447" style="list-style-type: none"> <li data-bbox="1980 184 2751 339">■ Take the hearing instruments out of the charger and disconnect the charger from the power supply. <li data-bbox="1980 360 2751 447">■ Contact your Hearing Care Professional for support.

Consult your Hearing Care Professional if you encounter further problems.

Important safety information

General warnings

- Observe the operating conditions. Refer to the section "Operating, transport, and storage conditions".
- Pay special attention to protect your device from extreme heat (e.g. fire, microwave oven, induction oven or other high induction fields) and sunlight.
- Charge your hearing instruments only with the approved charger. Ask your Hearing Care Professional for support.
- Do not disassemble the device.
- Do not use the device in explosive atmospheres.

- Do not use devices that are deformed or obviously damaged.
- Do not use the device when its performance after charging decreases considerably. Return the device to your Hearing Care Professional.
- Do not dispose of the device in fire or water.

In any of the above cases, return your device to the Hearing Care Professional.

WARNING

Risk of electric shock!

- ▶ Connect the charger only with a USB power supply which complies with IEC60950-1 and/or IEC62368-1.

Personal safety

WARNING

Risk of burns!

- ▶ Do not place anything other than your hearing instruments in the charger.

WARNING

Risk of electrical shock!

- ▶ Do not use obviously damaged devices and return them to point of sale.

WARNING

Choking hazard!

Your device contains small parts which can be swallowed.

- ▶ Keep hearing instruments, batteries and accessories out of reach of children and mentally disabled persons.
- ▶ If parts have been swallowed consult a physician or hospital immediately.

WARNING

Note that any unauthorized changes to the product may cause damage to the product or cause injury.

- ▶ Use only approved parts and accessories. Ask your Hearing Care Professional for support.

WARNING

Risk of explosion!

- ▶ Do not use your device in explosive atmospheres (e. g. in mining areas).

WARNING

Risk of eye injury

- ▶ In the unlikely event that the ultraviolet (UV) light stays on when the charger lid is opened, do not look directly at the UV light and return the charger to your point of sale.

WARNING

Charging systems may interfere with measuring devices and electronic equipment.

- ▶ In hospitals: The charger complies with CISPR 35 and CISPR 32. Therefore the use of the charger in hospitals should be permissible. Nevertheless the charger may interfere with highly sensitive devices. In those cases appropriate steps shall be taken (increase distance from sensitive devices or disconnect the charger from power supply).

WARNING

Interference with implantable devices

- ▶ The device can be used with all electronic implants that comply with the ANSI/AAMI/ISO 14117:2012 Electromagnetic compatibility standard for human implants.
- ▶ In case your implant does not comply with the ANSI/AAMI/ISO 14117:2012 standard, consult the manufacturer of your implantable device for information about the risk of disturbance.

Product safety

NOTICE

- ▶ Protect your devices from extreme heat. Do not expose them to direct sunlight.

NOTICE

- ▶ Protect your devices from high humidity.

NOTICE

- ▶ Do not dry your devices in the microwave oven.

NOTICE

- ▶ Do not drop. Internal components may get damaged.

NOTICE

- ▶ Do not disassemble the device.

NOTICE

Use of incompatible parts poses a risk of malfunction.

- ▶ Use only the power supply and country specific adapter that is provided with the charger.

Important information

Explanation of symbols

Symbols used in this document

Points out a situation that could lead to serious, moderate, or minor injuries.

Indicates possible property damage.

Advice and tips on how to handle your device better.

Symbols on the device or packaging

Indicates the legal manufacturer of the device.

Do not dispose of the device with general domestic waste. Read more in section "Disposal information".

Read and follow the instructions in the user guide.

Medical device

Operating, transport, and storage conditions

Operating

Temperature	10-35°C 50-95°F
Relative humidity	20-75%
Atmospheric pressure	700-1060 hPa

During extended periods of transport and storage, please observe the following conditions:

	Storage	Transport
Temperature	10-40°C 50-104°F	-20-60°C -4-140°F
Relative humidity	10-80%	5-90%
Atmospheric pressure	700-1060 hPa	700-1060 hPa

For other parts, such as hearing instruments, other conditions may apply.

Technical data of the power supply adapter

Input voltage:	100-240 V AC
Input current:	0.15-0.3 A
Input frequency:	50-60 Hz
Output voltage:	5 V DC
Output current:	1 A

Disposal information

Recycle devices, accessories and packaging according to local regulations.

To avoid environmental pollution, do not throw the device into household trash. Dispose of devices according to local regulations.

Conformance information

This device complies with Part 15B of the FCC Rules and with CAN ICES-003 / NMB-003.

Operation is subject to the following conditions:

- This device may not cause harmful interference,
- this device must accept any interference received, including interference that may cause undesired operation, and
- the user shall keep a minimal distance of 20 cm (8 in) during the charging process.

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Copyright ©2022 Miracle-Ear, Inc.
Distributed By Miracle-Ear, Inc.
Derechos de autor ©2022 Miracle-Ear, Inc.