GO Remote

User Manual

www.miracle-ear.com

Content

Your remote control	5
Components	6
Key lock	7
Pairing hearing aids	8
Function overview	13
Further information	15
Cleaning	15
Replacing the battery	16
Troubleshooting	20

Service	22
Adjusting the signal strength	23
Important safety information	25
<hr/>	
Personal safety	25
Product safety	27
Important information	30
<hr/>	
Intended use	30
Symbols	31
Operating, transport, and storage conditions	33
Disposal information	34

Your remote control

GO Remote is a remote control for hearing aids. It works with specific Miracle-Ear hearing aids. Your Hearing Care Specialist will consult with you on the compatible models.

CAUTION

Read this user manual thoroughly and completely and follow the safety information in this document to avoid damage or injury.

Components

- ① Status LED
- ② Volume buttons
- ③ Keyring
- ④ Key lock switch
- ⑤ Program change button

Key lock

- ▶ To prevent any accidental operation while the remote control is in a pocket or bag, move the key lock switch to the lock position (red color visible).

When the key lock is active, all buttons on the remote control are disabled.

Pairing hearing aids

Your hearing aids need to be paired with your remote control. After that, the pairing is saved in the hearing aids and remembered each time they are turned on.

- ▶ Lock the remote control (red color visible).
- ▶ First turn the hearing aids off:
Open the battery compartments. If your hearing aids do not have a battery compartment, turn them off as described in the hearing aids' user manual.

► Then turn the hearing aids on:

Close the battery compartments. If your hearing aids do not have a battery compartment, turn them on as described in the hearing aids' user manual.

The hearing aids are now in pairing mode.

You have 3 minutes to complete the pairing.

Follow the steps on the next page.

- ▶ Insert the hearing aids into your ears.
- ▶ Hold down the **P** button on the remote control while switching the key lock to the unlock position (green color visible).
- ▶ Release the **P** button.

The remote control is now in setup mode, the LED will be on.

While the remote control is in setup mode, do not press ⊕ or ⊖. This would change other settings.

- ▶ Hold down the **P** button until the pairing confirmation sound is played in the hearing aids. It takes approximately 2 seconds for the sound to start playing.
- ▶ Ensure that the confirmation sound is heard in both hearing aids. When heard, the pairing will be complete.

The remote control will remain in setup mode for an additional 30 seconds. The confirmation sound can be played again by pressing **P during these 30 seconds.**

- ▶ To exit the setup mode of the remote control, wait 30 seconds or move the key lock switch to the lock position (red color visible).

When pairing is finished, the remote control is ready for use.

▶ Ensure that it is unlocked (green color visible).

If the hearing aids could not be paired, refer to the "Troubleshooting" section.

Function overview

Function	Description
Volume	⊕ or ⊖ Press once to adjust the volume one level at a time. Or, press and hold until desired volume is reached.
Hearing program	P short press ➤ Switches to the next hearing program.

Function	Description
Turn on/off (standby)	P and ⊖ long press (2 seconds) ▶ Turns your hearing aids on or off.
Reset	P long press (2 seconds) ▶ Switches to hearing program 1 and sets the default volume level.

Further information

Cleaning

NOTICE


- ▶ Do not put your remote control in water!
- ▶ Do not clean your remote control with alcohol or benzine.
- ▶ Clean the remote control as necessary with a dry or slightly damp soft cloth.
Use only distilled water to dampen the cloth.
- ▶ Avoid any cleaning agents.

Replacing the battery

The remote control uses a CR 2450 button cell battery.

- ▶ Slide the battery compartment cover in the direction of the arrow **1**.
- ▶ Remove the empty battery by pulling the removal flap **2**.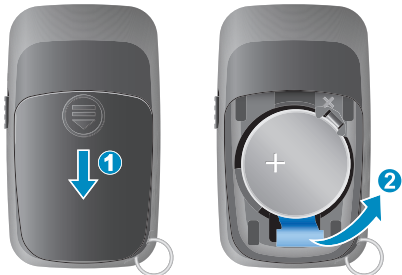

Insert the battery with the "+" symbol facing up:

- ▶ Slide it under the holder, then push it down.

- Attach the cover by sliding it onto the remote control.

The pairing status is remembered by the remote control when replacing the battery. There is no need to pair the hearing aids again after battery replacement.

Troubleshooting

If the remote control does not work:

- Ensure that the battery is inserted correctly.
Refer to section "Replacing the battery."
- Ensure that the keys are not locked.
Refer to section "Key lock."
- Ensure that there is nothing between the hearing aids and the remote control.
- Increase the signal strength.
Refer to section "Adjusting the signal strength."
- Pair the hearing aids again.
Refer to section "Pairing hearing aids."

- Ask your Hearing Care Specialist if this remote control is compatible with your hearing aids.

If the hearing aids cannot be paired:

- Increase the signal strength.
Refer to section "Adjusting the signal strength."
- Pair the hearing aids again.
Refer to section "Pairing hearing aids."

If the LED is no longer activating:

- Ensure that the keys are not locked. Refer to section "Key lock."
- The battery may be low. Replace the empty battery.

Consult your Hearing Care Specialist if you encounter further problems.

Service

If you have technical problems with the device, consult your Hearing Care Specialist for service purposes. Do not service or maintain the device while in use.

Adjusting the signal strength

The remote control works by emitting a high-frequency audio signal. This signal may be heard by some people. Two signal strength levels of the control signal are available. By default, the remote control uses the strongest setting to ensure a reliable connection with your hearing aids. A lower signal strength can be chosen if the control signal disturbs others. Be aware that this can affect the reliability of the connection.

To adjust the signal strength:

- ▶ Lock the remote control (red color visible).
- ▶ Hold down the **P** button on the remote control while switching the key lock to the unlock position (green color visible).

- ▶ Release the **P** button.
The remote control is now in setup mode, the LED will be on and the signal strength can now be changed.
- ▶ To switch to the **lowest** signal strength, press ⊖ until the LED flashes slowly.
To switch to the **strongest** signal strength, press ⊕ until the LED flashes quickly.
- ▶ To exit the setup mode of the remote control, wait 30 seconds or move the key lock switch to lock position (red color visible).
- ▶ To continue using your remote control, ensure that it is unlocked (green color visible).

Important safety information

Personal safety

WARNING

Choking hazard!

Your device contains small parts which can be swallowed.

- ▶ Keep hearing aids, batteries and accessories out of reach of children and cognitively impaired persons.
- ▶ If parts have been swallowed consult a physician or hospital immediately.

WARNING

Risk of injury!

- ▶ Do not use obviously damaged devices and return them to point of sale.

WARNING

Note that any unauthorized changes to the product may cause damage to the product or cause injury.

- ▶ Use only approved parts and accessories.
Ask your Hearing Care Specialist for support.

WARNING

Risk of explosion!

- ▶ Do not use your remote control in explosive atmospheres (e. g. in mining areas).

Product safety

NOTICE

- ▶ Protect your hearing aids and accessories from extreme heat. Do not expose them to direct sunlight.

NOTICE

- ▶ Protect your hearing aids and accessories from high humidity.

NOTICE

- ▶ Do not dry your hearing aids and accessories in the microwave oven.

NOTICE

Different types of strong radiation, e. g. during X-ray or MRI head examinations, may damage hearing aids and accessories.

- ▶ Do not use the devices during these or similar procedures.

Weaker radiation, e. g. from radio equipment or airport security, does not damage the hearing aids and accessories.

NOTICE

- ▶ Your remote control is programmed to exclusively communicate with your hearing aids. If you experience problems with another person's remote control, consult your Hearing Care Specialist.

Important information

Intended use

The remote control is intended to operate hearing aids.

Use the remote control only as described in this user manual.

This device is not intended for use and operation by children under the age of 12 years or persons with a developmental age of under 12 years.

Symbols

Symbols used in this document

Points out a situation that could lead to serious, moderate, or minor injuries.

Indicates possible property damage.

Advice and tips on how to handle your device better.

Symbols on device or packaging

Read and follow the instructions in the user guide.

Do not dispose of the device with household waste.

Refer to section "Disposal information".

Operating, transport, and storage conditions

Operating conditions

Temperature	32 to 113 °F
-------------	--------------

Relative humidity	5 to 93 %
-------------------	-----------

During extended periods of transport and storage, please observe the following conditions:

	Storage	Transport
Temperature	50 to 104 °F	-4 to 140 °F
Relative humidity	10 to 80 %	5 to 90 %

For other parts, such as batteries, other conditions may apply.

Disposal information

NOTICE

- ▶ Recycle hearing aids, accessories and packaging according to national regulations.

NOTICE

- ▶ To avoid environmental pollution, do not throw batteries into household trash.
- ▶ Recycle or dispose of batteries according to national regulations or return them to your Hearing Care Specialist.

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Derechos de autor ©2020 Miracle-Ear, Inc.
Distributed By Miracle-Ear, Inc.