Miracle-Ear**LINK**[™] Mic+
User Manual

Table of Contents

Overview	2
Basic Use	6
Daily Use	8
Wearing the Miracle-EarLINK™ Mic+	10
Pairing with Hearing Aids	11
Pairing with Bluetooth® Devices	13
Using the Input Mode Button	14
Remote Microphone	14
Bluetooth Phone and Audio	15
Loop System	17
FM System	17
Line-In	17
Start/Stop Audio Streaming	18
Assembling the Power Adapter	19
Troubleshooting	20
Safety Information	23
Regulatory Notices	26

Overview

The Miracle-Ear**LINK™** Mic+ is designed to stream audio from different audio sources directly to your Miracle-Ear® Avail™ wireless hearing aids.

When worn by a distant speaker in noisy environments, the Miracle-Ear**LINK™** Mic+ functions as an assistive listening device. The Miracle-Ear**LINK™** Mic+ can also interface with the following audio sources:

- Bluetooth® A2DP profile (for example, music)
- Bluetooth Handsfree profile (phone calls)
- Loop systems
- FM systems
- Line-in audio

Streaming - Audio or sound transmitted wirelessly from the Miracle-Ear**LINK™** Mic+ directly to your hearing aids.


Package Contents

- Miracle-Ear**LINK™** Mic+
- Power Adapter
- USB-to-microUSB cable
- Operations Manual

Optional Accessories

- Europin FM receiver
- Line-In Cable (part #91259-000)
- Lanyard (part #91324-000)

Overview

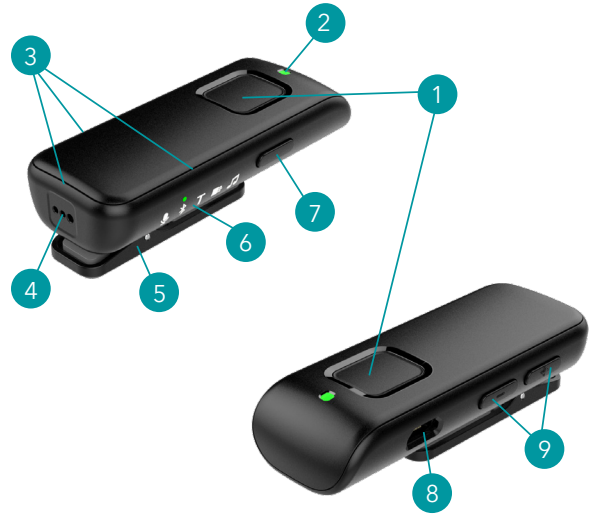- | | |
|--------------------------|----------------------|
| 1. Multi-Function Button | 6. Input Mode LEDs |
| 2. Status Indicator LED | 7. Input Mode Button |
| 3. Microphones | 8. MicroUSB Jack |
| 4. FM Receiver Jack | 9. Volume Buttons |
| 5. Clip | |

Overview

Overview

1. Multi-Function Button

- Controls operational state of the Miracle-Ear**LINK**™ Mic+

2. Status Indicator LED

- Identifies the operational state of the Miracle-Ear**LINK**™ Mic+

LED	OPERATIONAL STATE
Off	Power is off
Slow Pulsing Green	Power is on, but no active Bluetooth connection
Slow Pulsing Blue	Power is on plus an active Bluetooth connection
Slow Pulsing Red	Low battery condition
Solid Red	Possible fault condition

3. Microphones

- Picks up speaker's voice during remote mic use
- Picks up your voice during Bluetooth phone calls

4. FM Receiver Jack

- Accepts Europin FM Receivers

Overview

5. Clip

- Affixes Miracle-Ear**LINK**™ Mic+ to listener

6. Input Mode LEDs

- Identifies audio source

ICON	AUDIO SOURCE
	Remote Microphone (refer to page 14)
	Bluetooth Phone and Audio (refer to page 15)
	Loop (refer to page 17)
	FM System (refer to page 17)
	Line-In (refer to page 17)

7. Input Mode Button

- Changes audio source

8. MicroUSB Jack

- Recharges via USB-to-microUSB cable
- Plays line-in audio via optional cable

9. Volume Buttons

- Adjusts stream volume

Tip: Stream Volume can also be adjusted via the Miracle-Ear® app, Miracle-Ear**LINK**™ Remote AV, and/or hearing aid user controls. Review your hearing aid operation manual or contact your Hearing Care Professional for details on your particular system setup.

Basic Use

Basic Use

1. To turn on the Miracle-Ear**LINK**™ Mic+, press the multi-function button for 3 seconds and release when the green LED appears.

1

2. Ensure your hearing aids are powered on.
 - **Note:** Prior to streaming audio for the first time, your hearing aids will need to be paired to the Miracle-Ear**LINK**™ Mic+. Refer to **Pairing with Hearing Aids** on page 11.

3. To start audio streaming,
 - Open the Miracle-Ear® app and select Start Streaming from Accessories screen.
 - Refer to **Start/Stop Audio Streaming** on page 18 for additional options.

4. To change the audio source, press and release the input mode button. Repeat until you reach the desired input mode.

4

Basic Use

5. To access audio from a Bluetooth device , it must be connected to the Miracle-Ear**LINK**™ Mic+ (denoted by a slow pulsing blue status indicator LED).


- **Note:** Prior to streaming Bluetooth audio for the first time, pair your Bluetooth device with the Miracle-Ear**LINK**™ Mic+. Refer to **Pairing with Bluetooth Devices** on page 13.

5

6. To adjust the stream volume, , press the + or - button.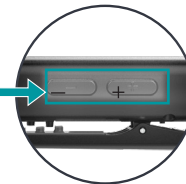

6

7. To stop streaming, do one of the following:
 - Turn off the Miracle-Ear**LINK**™ Mic+.
 - Open the Miracle-Ear® app and select Stop Streaming from Accessories screen.
 - Refer to **Start/Stop Audio Streaming** on page 18 for additional options.

Daily Use

Daily Use

Power On

Press the multi-function button for 3 seconds and release when the LED illuminates.

Power Off

Press the multi-function button for 2 seconds and release when the LED turns off.

Battery Low

LED slowly pulses red. Provides an audible indicator tone to hearing aids every 5 minutes.


Battery Recharge

Attach the microUSB end of the charging cable to the microUSB jack on the Miracle-EarLINK™ Mic+.

Attach the USB end of the charging cable to a female USB jack on the enclosed power adapter.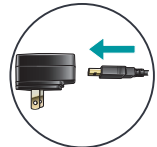

Daily Use

Recharging Status

Power	Charge State	LED
Off	Recharging	Solid Green
Off	Complete	Off
On	Recharging	Solid green briefly interrupted by other LED status behaviors
On	Complete	Off


Tip: Recharging the Miracle-EarLINK™ Mic+ can take longer while streaming audio. Recharging will be faster when the Miracle-EarLINK™ Mic+ is off.

Wearing the Miracle-EarLINK™ Mic+

Pairing

Wearing the Miracle-EarLINK™ Mic+

The Miracle-EarLINK™ Mic+ can be affixed to clothing or worn on a lanyard. When your audio source is the Remote Mic or a Bluetooth® device in phone mode, the Miracle-EarLINK™ Mic+ should be within 8 inches (20cm) of the speaker's mouth.

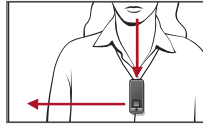Clip – Press the clip as shown. Slide clip opening onto clothing and release clip. Ensure the Miracle-EarLINK™ Mic+ is securely affixed to clothing.

Lanyard (optional) –

1. Press the clip as shown.
2. Insert lanyard loop into clip opening and lift loop toward top of clip. Release clip. Ensure the lanyard is securely attached to the Miracle-EarLINK™ Mic+.
3. Place lanyard around neck.

Pairing with Hearing Aids

For the Miracle-EarLINK™ Mic+ to interact with your hearing aid(s), you must first pair the Miracle-EarLINK™ Mic+ with each hearing aid. You only need to perform this action one time, as they should automatically recognize each other when both are powered on and within wireless operating range.*


To pair the Miracle-EarLINK™ Mic+ with your hearing aid(s):

1. Ensure your hearing aids are not connected to nearby Bluetooth® devices. To do this,
 - Disable Bluetooth on your mobile phone.
 - Power off any nearby Miracle-EarLINK™ TV Streamer AV or Miracle-EarLINK™ Mic+ accessories, which you previously paired to your hearing aids.
2. Power off, then power on your hearing aids.
 - **Note:** For a CROS hearing aid system, power off both your hearing aid and CROS transmitter. Power on only your hearing aid, ensuring the CROS transmitter remains powered off.
3. Place your hearing aids within 6 inches (15 cm) of the Miracle-EarLINK™ Mic+.

*Typical operating range is up to 10 meters (~30 feet). Operational range may be reduced by environmental factors, such as obstructions, interference, and human body positioning. Larger operational distances are achievable, though audio quality and connectivity may decrease.

Pairing

- Press the Input Mode and Volume Up (+) buttons simultaneously for 7 seconds until the LED status indicator alternately blinks blue and red.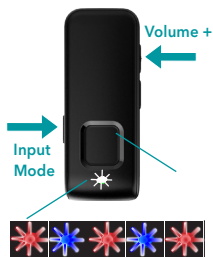

- For **a set of** hearing aids:

First LED Blink	Second LED Blink	Pairing Result	Next Step
1	2	Both hearing aids paired	Complete
1	2	Only one hearing aid paired	Repeat steps 1 - 5
1	n/a	No hearing aids paired	Repeat steps 1 - 5

- For a **single** hearing aid:


LED Blink	Pairing Result	Next Step
1	Successful	Complete
1	Unsuccessful	Repeat steps 1 - 4 and 6

Pairing

Pairing with Bluetooth Devices

For the Miracle-Ear**LINK**™ Mic+ to wirelessly receive audio from your Bluetooth® device, you must first pair them:

- Ensure the Miracle-Ear**LINK**™ Mic+ is powered on and not Bluetooth connected to other devices (the green LED will be slowly pulsing).
- Press the Input Mode and Volume Down (-) buttons simultaneously for 7 seconds until the LED status indicator rapidly blinks blue.
- Put your Bluetooth device into pairing mode. When Miracle-Ear**LINK**™ Mic+ appears, select it.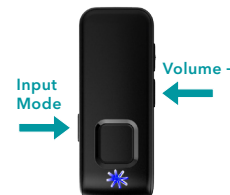

• **Note:** Review your Bluetooth device operations guide for more details.

- The Miracle-Ear**LINK**™ Mic+ LED provides the pairing status:

- Successful - green LED for 2 seconds
- Unsuccessful - red LED for 2 seconds


Input Mode/Remote Microphone

Using the Input Mode Button

View Current Audio Source

If Input Mode LED is off, press button to temporarily illuminate the LED.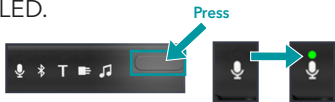

Change to Different Audio Source

While Input Mode LED is illuminated, press button to move to next available audio source.

Remote Microphone


The Miracle-EarLINK™ Mic+ automatically optimizes the microphone settings based on its physical orientation.

Focus on one person when background noise is present. When worn by a speaker, the Miracle-EarLINK™ Mic+ automatically prioritizes sound from above while reducing sound levels from other directions.


Listen to multiple nearby speakers in quiet environments. When placed horizontally on its clip, it receives sound from all directions.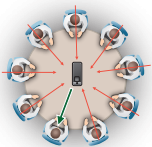

Bluetooth® Phone and Audio

Bluetooth® Phone and Audio

Conduct Phone Calls (Handsfree Profile) (A) You hear the phone call in your hearing aids. (B) The Miracle-EarLINK™ Mic+ picks up your voice and transmits it back to the mobile phone.

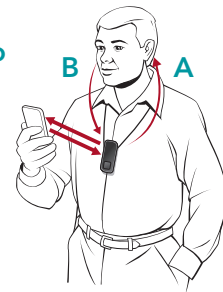Listen to Music (A2DP Profile) You hear the streamed audio in your hearing aids.

General Bluetooth Functionality

Pair with Bluetooth Devices	Miracle-EarLINK™ Mic+ can pair with up to 8 Bluetooth devices.
Connect with Bluetooth Device	When both are powered on and within wireless operating range, the Miracle-EarLINK™ Mic+ and paired Bluetooth device should automatically connect and the LED status indicator slowly pulses blue. The Miracle-EarLINK™ Mic+ can connect with up to 2 Bluetooth devices at a time.
Listening to Music (or Other Streamed Audio)	Ensure your input mode is Bluetooth on the Miracle-EarLINK™ Mic+. Play audio on your Bluetooth device and it will be automatically routed to your hearing aids.

Bluetooth® Phone and Audio

Phone Call Features


Change to Bluetooth® Audio Source	When the Remote Miracle-EarLINK™ Mic+ detects an incoming or outgoing phone call, it automatically changes to the Bluetooth audio source.
Accept Incoming Call	When you hear the ringtone in your hearing aids, briefly press the multi-function button.
Reject Incoming Call	When you hear the ringtone in your hearing aids, press the multi-function button for more than 2 seconds. Release the button after the ringtone stops.
End Call	During active phone call, briefly press the multi-function button.
Transfer Call between mobile phone and Miracle-EarLINK™ Mic+	During active phone call, press the multi-function button for more than 2 seconds. This lets you directly conduct the call on the phone itself. Re-press the button for more than 2 seconds to move the call back to the Miracle-EarLINK™ Mic+.
Make an Outgoing Call	Dial the phone number on your Bluetooth device. The phone call will be automatically routed to your hearing aids.

Loop System/FM System/Line-In

T Loop System


The Miracle-EarLINK™ Mic+ can receive audio from a Loop system (available in some public buildings). Wearing the Miracle-EarLINK™ Mic+ in a vertical orientation provides best performance.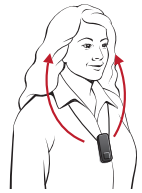

In this mode, the Miracle-EarLINK™ Mic+ disables Bluetooth® connections. Upon exiting this mode, the Miracle-EarLINK™ Mic+ will attempt to re-establish a Bluetooth connection. Manual reconnection may be necessary. Review your Bluetooth device operations guide for more details.


FM System

When an optional FM transmitter is attached via the Europin connector, the Miracle-EarLINK™ Mic+ can receive audio from an FM system. In this mode, the user can wear the Miracle-EarLINK™ Mic+ or place the Miracle-EarLINK™ Mic+ on a nearby stable surface.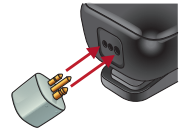

Line-In

When an optional Line-In cable is attached via the microUSB jack, the Miracle-EarLINK™ Mic+ can obtain audio from a non-Bluetooth-enabled mobile phone, MP3 player, tablet, or computer. In this mode, the user typically places the Miracle-EarLINK™ Mic+ on a nearby stable surface.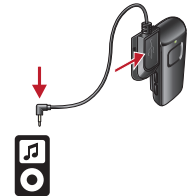

Start/Stop Audio Streaming

Start/Stop Audio Streaming

Audio streaming from the Miracle-Ear**LINK**™ Mic+ may be started (or stopped) in a variety of ways:

- **Turn On the Miracle-Ear**LINK**™ Mic+:** Audio streaming automatically starts in hearing aid.
- **Miracle-Ear® app:** Open the app on your mobile phone or tablet. Select the option to start/stop streaming from Accessories screen.
- **Miracle-Ear**LINK**™ TV Streamer AV:** If preconfigured by your Hearing Care Professional, press the Favorite button. 
- **Hearing aid user control:** If preconfigured by your Hearing Care Professional, a specific gesture (e.g. a Long Press on a user control) can start/stop streaming.

Audio streaming will also stop when:

- An incoming audio signal is not detected while in the Bluetooth® input mode.
- The Miracle-Ear**LINK**™ Mic+ is turned off.


An audible indicator may play in your hearing aids to signal the start/stop of audio streaming.

Assembling the Power Adapter

Assembling the Power Adapter


The Miracle-Ear**LINK**™ Mic+ may include a variety of plug configurations for use in different countries. If the power adapter requires assembly:

1. Select the appropriate plug configuration used in your country.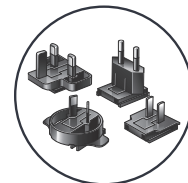


1

2. Slide the plug onto the power adapter until it snaps into place.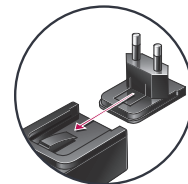

2

3. To remove the plug, press the tab and slide the plug away from the cable.

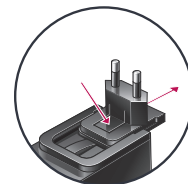3

Troubleshooting

SYMPTOM	POSSIBLE CAUSES	SOLUTIONS
One (or both) hearing aids unexpectedly drops out of audio streaming.	Hearing aids may be on the edge of wireless range.	Move closer to Miracle-Ear LINK ™ Mic+.
	Battery level in hearing aid can no longer support audio streaming.	<ul style="list-style-type: none"> • Replace hearing aid battery with new one. • Recharge your hearing aid, if applicable.
	Large object, such as furniture or a wall, may be degrading signal.	For best performance, your hearing aids should be within line-of-sight of Miracle-Ear LINK ™ Mic+.
	Wireless connection with Miracle-Ear LINK ™ Mic+ was lost.	To restart the audio stream, follow steps in the Start/Stop Audio Streaming section.
Streamed audio quality is distorted.	Low battery level in Miracle-Ear LINK ™ Mic+.	Recharge Miracle-Ear LINK ™ Mic+.
	Volume of audio source is not optimal.	Adjust volume level of audio source until better sound quality is achieved.
Streamed audio quality is inconsistent.	<u>Bluetooth® Mode:</u> Bluetooth device may be on the edge of wireless range.	Move Bluetooth® device closer to Miracle-Ear LINK ™ Mic+.
	<u>Loop Mode:</u> Miracle-Ear LINK ™ Mic+ may not be positioned for optimal streaming.	Re-orient Miracle-Ear LINK ™ Mic+ into a vertical position.

Troubleshooting

SYMPTOM	POSSIBLE CAUSES	SOLUTIONS
Audio stream is too loud or too soft.	Volume streamed from Miracle-Ear LINK ™ Mic+ to hearing aid not optimal.	Press volume + or - button on Miracle-Ear LINK ™ Mic+.
	Volume of audio source is not optimal.	Adjust volume level of audio source until better sound quality is achieved.
<u>Remote Mic Mode:</u> Difficult to hear speaker in noisy environments.	Miracle-Ear LINK ™ Mic+ may be too far from speaker.	Move Miracle-Ear LINK ™ Mic+ closer to speaker's mouth.
	Miracle-Ear LINK ™ Mic+ may be covered by clothing or poorly positioned.	Provide clear access from Miracle-Ear LINK ™ Mic+ to speaker's voice.
<u>Bluetooth® Mode:</u> Caller cannot hear your voice during phone calls.	Miracle-Ear LINK ™ Mic+ may be too far from your mouth.	Miracle-Ear LINK ™ Mic+ closer to your mouth or speak louder.
<u>Loop Mode:</u> Phone calls are not heard in hearing aids.	Bluetooth® connections with mobile phones are disabled while in Loop mode.	Move to another audio source. Reestablish your Bluetooth connection, if necessary.
LED is slowly pulsing green or blue, but no streamed sounds are heard in the hearing aids.	Hearing aids are not ready to receive audio streams.	To start the audio stream, follow steps in the Start/Stop Audio Streaming section.

SYMPTOM	POSSIBLE CAUSES	SOLUTIONS
LED is slowly pulsing green or blue, but no streamed sounds are heard in the hearing aids.	Hearing aids are not paired with Miracle-Ear LINK™ Mic+.	Follow steps in Pairing with Hearing Aids section.
	No incoming audio signal is available.	<ul style="list-style-type: none"> • Turn on the electronic audio source. • Unmute or increase volume on electronic audio source. • Ensure loop or FM system is present.
LED is solid red.	Possible error condition.	Turn off Miracle-Ear LINK™ Mic+, wait 30 seconds, and then turn it on.
Input Mode LED is rapidly blinking.	Something is absent from current audio source.	Attach line-in cable, attach FM receiver, or reestablish your Bluetooth connection.
Input Mode LED is not illuminated.	Input Mode LED is turned off to conserve battery power.	Press Input Mode button to illuminate the Input Mode LED.
Desired Input Mode not available.	Something is missing from desired input.	Attach line-in cable, attach FM receiver, or reestablish your Bluetooth connection.
Unable to pair with more than one Bluetooth device.	Miracle-Ear LINK™ Mic+ is already in an active Bluetooth connection.	Follow steps in Pairing with Bluetooth Devices section.
Unexpectedly lose TV (optional product) audio while LED is slowly pulsing blue on Miracle-Ear LINK™ Mic+	Some Bluetooth device sounds are higher priority than TV Streamer audio.	Move out of Bluetooth input mode on Miracle-Ear LINK™ Mic+.

Safety Information

Required Information

Waste from electronic equipment must be handled according to local regulations.

Instructions for Disposal of Old Electronics

Miracle-Ear® encourages, and your local community laws may require, that your Miracle-Ear**LINK™** Mic+ be disposed of via your local electronics recycling/disposal process.

The below instructions are provided for the benefit of disposal/recycling personnel. Please include this manual when disposing of your Miracle-Ear**LINK™** Mic+.

For Disposal/Recycling Personnel Only

This product contains a Lithium-ion Polymer battery. To remove this battery:

- Power off the Miracle-Ear**LINK™** Mic+.
- Pry apart the housing halves with a small screwdriver.
- Cut the battery wires from the circuit board to prevent accidental shorting.
- Pry the battery away from the housing using a wide flat blade.

Intended Use

The Miracle-Ear**LINK™** Mic+ is an accessory to a 2.4 GHz wireless air conduction hearing aid. It is intended as a wireless transmitter of sound from a variety of audio sources to air conduction hearing aids. When worn by a distant speaker in noisy environments, it functions as an assistive listening device. It also connects to loop signals, FM signals when an Europin FM receiver is attached, analog line-in audio, and standard Bluetooth audio.

The Miracle-Ear**LINK™** Mic+ is classified as a Type B applied part under the IEC 60601-1 medical instrument standard.

The Miracle-Ear**LINK™** Mic+ is designed to operate in public and residential environments and are designed to comply with international Electromagnetic Compatibility emissions and immunity standards for medical devices. However, it is still possible that you may experience interference caused by power line disturbances, airport metal detectors, electromagnetic fields from other medical devices, radio signals and electrostatic discharges.

The Miracle-Ear**LINK™** Mic+ is designed to be cleaned with a damp cloth moistened with soapy water.

Safety Information

If you use other medical devices or wear implantable medical devices such as defibrillators or pacemakers and are concerned that your Miracle-Ear**LINK™** Mic+ might cause interference with your medical device, please contact your physician or the manufacturer of your medical device for information about the risk of disturbance. The Miracle-Ear**LINK™** Mic+ should not be used during an MRI procedure or in a hyperbaric chamber, or other oxygen-rich environments. The Miracle-Ear**LINK™** Mic+ is not formally certified to operate in explosive atmospheres such as may be found in coal mines or certain chemical factories.


Use on Aircrafts

The Miracle-Ear**LINK™** Mic+ should not be used on aircrafts unless specifically permitted by the flight personnel.

International Use

The Miracle-Ear**LINK™** Mic+ is approved to operate at radio frequencies that are specific to your country or region and might not be approved for use outside your country or region.

Be aware that operation during international travel may cause interference with other electronic devices, or other electronic devices may cause interference with your Miracle-Ear**LINK™** Mic+.

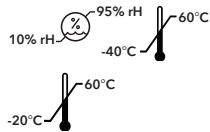The Miracle-Ear**LINK™** Mic+ is designed to the following temperature ranges:

Operating: -20°C to +60°C (-4°F to 140°F), relative humidity range 10%-95%

Storage: -40°C to +60°C (-40°F to 140°F)

We are required by regulations to provide the following warnings:

⚠ WARNING: Use of the Miracle-Ear**LINK™** Mic+ directly next to other electronic equipment should be avoided because it could result in improper performance. If such use is necessary, note as to whether your hearing aids and the other equipment are operating normally.

⚠ WARNING: Use of accessories, components or replacement parts other than those provided by the manufacturer of the Miracle-Ear**LINK™** Mic+ could result in increased electromagnetic emissions and decreased electromagnetic immunity and could result in degradation of performance.

⚠ WARNING: If Portable Radio Frequency communications equipment is used closer than 30 cm (12 inches) from your Miracle-Ear**LINK™** Mic+, degradation of the performance of your hearing aid could result. If this occurs, move away from the communications equipment.

Safety Information

Although the Miracle-Ear**LINK™** Mic+ is not formally certified (IP rated) for protection against dust or water ingress, per IEC 60529, it has been tested to the IP 5X level. This means that the Miracle-Ear**LINK™** Mic+ is protected from dust.

The battery in your Miracle-Ear**LINK™** Mic+ is not replaceable. Do not attempt to replace the battery. The Miracle-Ear**LINK™** Mic+ does not contain any user-serviceable parts. Do not open or modify it.

Do not connect the Miracle-Ear**LINK™** Mic+ to any devices not described in this manual. The expected service life of the Miracle-Ear**LINK™** Mic+ is 3 years.

Wireless Technical Description

Miracle-Ear**LINK™** Mic+ operates in the 2.4- 2.4835 GHz band with a maximum effective radiated power of 8 dBm with transmission modulation type of 1M12FXD and 1.1 MHz bandwidth. The receiver section of the radio has a bandwidth of 2 MHz.

The Miracle-Ear**LINK™** Mic+ has been tested to, and has passed, the following emissions and immunity tests:

- IEC 60601-1-2 radiated and conducted emissions requirements for a Group 1 Class B device as stated in CISPR 11.
- Harmonic distortion and voltage fluctuations affecting the power input source as stated in Table 2 of IEC 60601-1-2.
- RF radiated immunity at a field level of 10 V/m between 80 MHz and 2.7 GHz as well as higher field levels from communications devices as stated in Table 9 of IEC 60601-1-2.
- Immunity to power frequency magnetic fields at a field level of 30 A/m as well as proximity magnetic fields as defined in table 11 of 60601-1-2.
- Immunity to ESD levels of +/- 8 kV conducted discharge and +/- 15 kV air discharge.
- Immunity to electrical fast transients on the power input at a level of +/- 2 kV at a 100 Hz repetition rate.
- Immunity to surges on the power input of +/- 1 kV line to line.
- Immunity to conducted disturbances induced by RF fields on the power input as stated in Table 6 of IEC 60601-1-2.
- Immunity to voltage dips and interruptions on the power input as stated in Table 6 of IEC 60601-1-2.

Regulatory Notices

Regulatory Notices

FCC ID: EOA-24MFA

IC: 6903A-24MFA (Model 900)

FCC/IC Notice

This device complies with part 15 of the FCC rules and with ISED Canada's license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation of the device.

NOTE: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user's authority to operate the equipment.

Regulatory Notices

Manufactured For:

Miracle-Ear

150 South Fifth Street, Suite 2300

Minneapolis, MN 55402

This device produces radio frequency energy. Interference may occur in the vicinity of equipment marked with the following symbol.

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The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks is under license.
All trade names and trademarks are properties of their respective owners.
Distributed By Miracle-Ear, Inc.