## MECONNECT RIC 312

Receiver-In-The-Canal User Manual

# Content

<b>Welcome</b>	<b>4</b>
<hr/>	
<b>Your hearing aids</b>	<b>5</b>
<hr/>	
Hearing aid type	5
Getting to know your hearing aids	6
Components and names	7
Controls	9
Hearing programs	10
Features	10
<b>Batteries</b>	<b>11</b>
<hr/>	
Battery size and handling tips	11
Replacing batteries	12
<b>Daily use</b>	<b>13</b>
<hr/>	
Turning on and off	13
Inserting and removing your hearing aids	14
Adjusting the volume	18
Changing hearing programs	18
Further adjustments (optional)	19
<b>Special listening situations</b>	<b>20</b>
<hr/>	
On the phone	20
Audio streaming with iPhone®	21
Airplane mode	21
Audio induction loops (optional)	22

<b>Maintenance and care</b>	<b>23</b>
Hearing aids	23
Ear pieces	24
Professional maintenance	26
<b>Further information</b>	<b>27</b>
Safety information	27
Accessories	27
Symbols used in this document	27
Troubleshooting	28
Country-specific information	29
Service and warranty	30

# Welcome

Thank you for choosing Miracle-Ear® hearing aids to accompany you through everyday life. Like anything new, it may take you a little time to become familiar with them.

This guide, along with support from your Hearing Care Specialist, will help you understand the advantages and greater quality of life your hearing aids offer.

To receive the most benefit from your hearing aids, it is recommended that you wear them every day, all day. This will help you get adjusted to them.

## **CAUTION**

It is important to read this user guide and the safety manual thoroughly and completely. Follow the safety information to avoid damage or injury.

# Your hearing aids

This user guide describes optional features that your hearing aids may have. Ask your Hearing Care Specialist to indicate the features that apply to your hearing aids.

## Hearing aid type

Your hearing aids are a RIC (Receiver-in-Canal) model. The receiver is placed within the ear canal and connected to the instrument via a receiver cable. The instruments are not intended for children under the age of 3 years or persons with a developmental age of under 3 years.

The wireless functionality enables advanced audiological features and synchronization between your two hearing aids.

Your hearing aids feature Bluetooth® low energy\* technology allowing for easy data exchange with your smartphone, and for seamless audio streaming with your iPhone.\*\*

\*The Bluetooth word mark and logos are owned by the Bluetooth SIG, Inc., and any use of such marks by the legal manufacturer of this product is under licenses. Other trademarks and trade names are those of their respective owners.

\*\*iPad, iPhone, and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries.

## Getting to know your hearing aids


It is recommended you familiarize yourself with your new hearing aids. With the instruments in your hand, try using the controls and note their location on the instrument. This will make it easier for you to feel and press the controls while wearing the hearing aids.

If you have problems pressing the controls of your hearing aids while wearing them, you can use a remote control to control your instruments. With a dedicated smartphone app you have even more control options.

## Components and names

This user guide describes the hearing aid with several variations. Use the following pictures to identify the type of hearing instrument that you use.

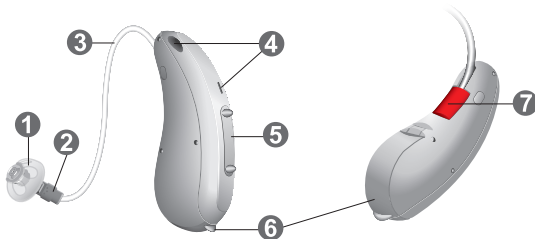1 Ear piece

2 Receiver

3 Receiver cable

4 Microphone openings

5 Rocker switch (control)

6 Battery compartment  
(on/off switch)

7 Side indicator  
(red = right ear,  
blue = left ear) and  
receiver connection

You can use the following standard ear pieces:

<b>Standard ear pieces</b>	<b>Size</b>
<input type="radio"/>  Click sleeve (vented or closed)	
<input type="radio"/>  Open or closed dome	
<input type="radio"/>  Semi-open dome	
<input type="radio"/>  Double dome	

You can easily exchange the standard ear pieces. Read more in section "Maintenance and care".

### Custom-made ear pieces

<input type="radio"/>  Custom shell	
<input type="radio"/>  Click Mold	

## Controls

The rocker switch allows you to adjust the volume or switch hearing programs. Your Hearing Care Specialist has programmed your desired functions to the rocker switch.

<b>Rocker switch function</b>	<b>L</b>	<b>R</b>
<hr/> Press briefly:		
Program up/down	<input type="radio"/>	<input type="radio"/>
Volume up/down	<input type="radio"/>	<input type="radio"/>
Tinnitus control signal level up/down	<input type="radio"/>	<input type="radio"/>
TV stream on/off	<input type="radio"/>	<input type="radio"/>
<hr/> Press for about 2 seconds:		
Program up/down	<input type="radio"/>	<input type="radio"/>
Volume up/down	<input type="radio"/>	<input type="radio"/>
Tinnitus control signal level up/down	<input type="radio"/>	<input type="radio"/>
TV stream on/off	<input type="radio"/>	<input type="radio"/>
<hr/> Press for more than 3 seconds:		
Standby/turn on	<input type="radio"/>	<input type="radio"/>
Activate/deactivate airplane mode	<input type="radio"/>	<input type="radio"/>

L = Left, R = Right

You can also use a remote control to change hearing programs and adjust the volume of your hearing aids. With our smartphone app you have even more control options.

## Hearing programs

---

1

---

2

---

3

---

4

---

5

---

6

---

Read more in section "Changing hearing programs."

## Features

- Power-on delay** enables whistle-free insertion of hearing aids.  
Read more in section "Turning on and off".
- The **tinnitus therapy feature** generates a sound to distract you from your tinnitus.
- A **telecoil** is integrated in the battery compartment, enabling the hearing instrument to connect with audio induction loops.  
Read more in section "Audio induction loops".

# Batteries

When the battery is low the sound becomes weaker or you may hear an alert signal. The battery type will determine how long you have to replace the battery.

## Battery size and handling tips

Ask your Hearing Care Specialist for recommended batteries.

---

Battery size:	312
---------------	-----


---

- Always use the correct battery size for your hearing aids.
- Remove the batteries if you intend not to use the hearing aids for several days.
- Always carry spare batteries.
- Remove empty batteries immediately and follow your local recycling rules for battery disposal.


## Replacing batteries

Removing the battery:

- ▶ Open the battery compartment.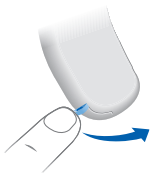

- ▶ Use the magnet stick to pull out the battery. The magnet stick is available as an optional accessory from your Hearing Care Specialist.

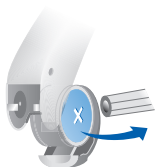Inserting the battery:

- ▶ If the battery has a protective film, remove it only when you are ready to use the battery.

- ▶ Insert the battery with the "+" symbol facing up (refer to the picture).

- ▶ Carefully close the battery compartment. If you feel resistance, the battery is not inserted correctly.

Do not attempt to close the battery compartment by force. It can be damaged.

# Daily use

## Turning on and off

You have the following options to turn your hearing aids on or off.

Via battery compartment:

- ▶ Turning on: Close the battery compartment.  
The **default** volume and hearing program are set.
- ▶ Turning off: Open the battery compartment to the first catch.

Via rocker switch:

- ▶ Turning on or off: Press and hold the rocker switch for several seconds. Refer to section "Controls" for the settings of the rocker switch.

After turning on, the **previously** used volume and hearing program remain set.

When wearing the hearing aids, an alert tone can indicate when a hearing aid is being turned on or off.


When the **power-on delay** is activated, the hearing aids turn on after a delay of several seconds. During this time you can insert the hearing aids into your ears without experiencing unpleasant feedback or whistling.

The "power-on delay" can be activated by your Hearing Care Specialist.

## Inserting and removing your hearing aids

Your hearing aids have been fine-tuned for your right and left ear. Colored markers indicate the side:


- red marker = right ear
- blue marker = left ear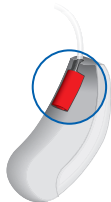


Inserting a hearing instrument:

- ▶ For Click Sleeves, make sure that the bend of the Click Sleeve is in line with the bend of the receiver cable.


Correct: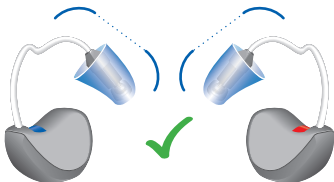

Incorrect:

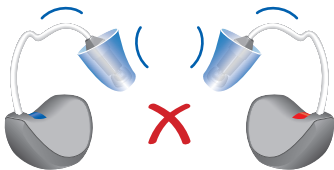- ▶ Hold the receiver cable at the bend closer to the ear piece.
- ▶ Carefully push the ear piece in the ear canal **1**.
- ▶ Twist it slightly until it sits well.  
Open and close your mouth to avoid accumulation of air in the ear canal.
- ▶ Lift the hearing instrument and slide it over the top of your ear **2**.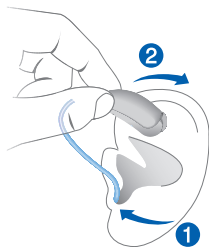

### CAUTION

Risk of injury!


- ▶ Insert the ear piece carefully and not too deeply into the ear.

- It may be helpful to insert the right hearing instrument with the right hand and the left hearing instrument with the left hand.
- If you have problems inserting the ear piece, use the other hand to gently pull your earlobe downwards. This opens the ear canal and eases insertion of the ear piece.

The optional retention strand helps to securely retain the ear piece in your ear. To position the retention strand:


- ▶ Bend the retention strand and position it carefully into the bottom of the bowl of your ear (refer to the picture).

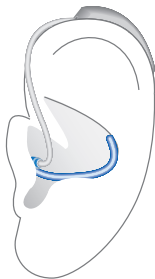Removing a hearing instrument:

- ▶ Lift the hearing instrument and slide it over the top of your ear **1**.
- ▶ If your hearing instrument is equipped with a custom shell or a click mold, remove it by pulling the small removal cord toward the back of your head.
- ▶ For all other ear pieces: Grip the receiver in the ear canal with two fingers and pull it carefully out **2**.

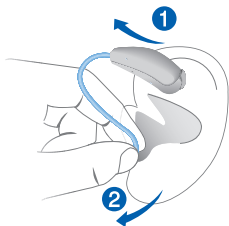Do not pull the receiver cable.

#### **CAUTION**

Risk of injury!

- ▶ In very rare cases the ear piece could remain in your ear when removing the hearing instrument. If this happens, have the ear piece removed by a medical professional.

Clean and dry your hearing aids after usage. Read more in section "Maintenance and care".

## Adjusting the volume

Your hearing aids automatically adjust the volume to the listening situation.

- ▶ If you prefer manual volume adjustment, press the rocker switch.

Refer to section "Controls" for the setting of the rocker switch.

An optional signal tone can indicate the volume change. Once the maximum or minimum volume has been reached, you may hear an optional signal tone.

## Changing hearing programs

Depending on the listening situation, your hearing aids automatically adjust their sound.

Your hearing aids may also have several hearing programs which allow you to change the sound, if needed. An optional signal tone can indicate the program change.

- ▶ To change the hearing program, press the rocker switch.

Refer to section "Controls" for the setting of the rocker switch. Refer to section "Hearing programs" for a list of your hearing programs.

The default volume is applied.

## **Further adjustments (optional)**


You can also use the hearing instrument controls to change other features, for example, the level of the tinnitus control signal.

Refer to section "Controls" for the setting of your controls.

## Special listening situations

### On the phone

When you are on the phone, hold the telephone receiver slightly above your ear. The hearing instrument and telephone receiver should be aligned. Turn the receiver slightly so that the ear is not completely covered.

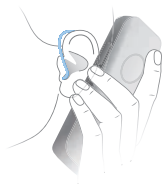### Telephone program

You may prefer a certain volume when using the phone. Ask your Hearing Care Specialist to configure a telephone program.

- ▶ Switch to the telephone program whenever you are on the phone.

If a telephone program is configured for your hearing aids, it is listed in section "Hearing programs".

## Audio streaming with iPhone

Your hearing aids are **Made for iPhone**<sup>®</sup> hearing aids. This means you can get phone calls and listen to music from your iPhone directly into your hearing aids.

For more information about compatible iOS devices, pairing, audio streaming and other useful functions, contact your Hearing Care Specialist.

## Airplane mode

In areas where the use of Bluetooth technology is not allowed (e.g. on some airplanes), you can activate airplane mode. This turns Bluetooth in your hearing aids off temporarily. The hearing aids still work without Bluetooth, however direct audio streaming will not be possible and other functions may not be available.

- ▶ To activate or deactivate the airplane mode, use the controls of your hearing aids.

Refer to section "Controls" for more information.

An alert tone indicates when airplane mode is activated or deactivated.

## Audio induction loops (optional)

Some telephones and public places like theatres offer the audio signal (music and speech) via an audio induction loop. With this system, your hearing aids can directly receive the desired signal - without distracting noise in the environment.

You can usually recognize audio induction loop systems by certain signs.

If you want to use this system, a special battery compartment is required. It is available as an accessory only for certain types of instruments. Consult your Hearing Care Specialist for more information.

Whenever you are in a place with an audio induction loop:

- ▶ Switch to the telecoil program.

If a telecoil program is configured for your hearing aids, it is listed in section "Hearing programs".

## Telecoil program and remote control

### NOTICE

For hearing aids with wireless connectivity:

When the telecoil program is active, the remote control may cause a pulsing noise.

- ▶ Use the remote control with a distance greater than 10 cm (4 inches).

# Maintenance and care

To prevent damage it is important that you take care of your hearing aids and follow a few basic rules, which will soon become a part of your daily routine.

## Hearing aids

### Drying and storage

- ▶ Dry your hearing aids overnight.
- ▶ For longer periods of non-use, store your hearing aids in a dry environment with an open battery compartment and the batteries removed.
- ▶ Ask your Hearing Care Specialist for more information.

### Cleaning

Your hearing aids have a protective coating. However, if not cleaned regularly you may damage the hearing aids or cause personal injury.

- ▶ Clean your hearing aids daily with a soft, dry tissue.
- ▶ Never use running water or immerse the devices in water.
- ▶ Never apply pressure while cleaning.
- ▶ Ask your Hearing Care Specialist for recommended cleaning products, special care sets, or more information on how to keep your hearing aids in good condition.

## Ear pieces


### Cleaning

Cerumen (ear wax) may accumulate on the ear pieces. This may affect the sound quality. Clean the ear pieces daily.

For all types of ear pieces:


- ▶ Clean the ear piece right after removal with a soft and dry tissue.

This prevents that cerumen becomes dry and hard.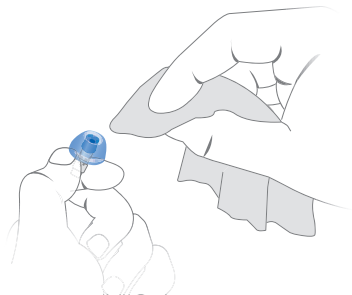

For click domes or click sleeves:

- ▶ Squeeze the tip of the ear piece.

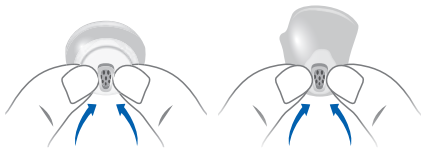
## Exchanging

Exchange standard ear pieces approximately every three months. Replace them sooner if you notice cracks, discoloration or other changes. The procedure for exchanging standard ear pieces depends on the type of ear piece. In section “Components and names,” your Hearing Care Specialist has marked your type of ear piece.

### Exchanging click domes

- ▶ To remove the old click dome, use the removal tool or follow the instructions on the packaging of the click domes. Be careful not to pull the receiver cable.

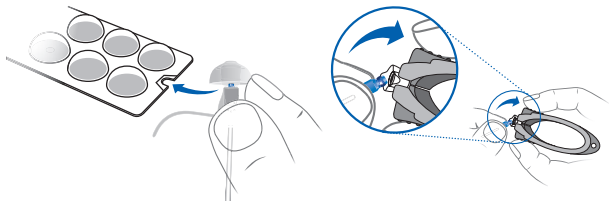The removal tool is available as an accessory.

- ▶ Take particular care that the new click dome clicks noticeably in place.
- ▶ Check the firm fit of the new click dome.

## Exchanging Click Sleeves

- ▶ Turn the click sleeve inside out.
- ▶ Grasp the click sleeve and the receiver (not the receiver cable) with your fingers and pull the click sleeve off.
- ▶ Take particular care that the new click sleeve clicks noticeably in place.

## Professional maintenance

Your Hearing Care Specialist can perform thorough professional cleaning and maintenance.

Custom-made and standard ear pieces and wax filters should be exchanged as needed by your Hearing Care Specialist.

Ask your Hearing Care Specialist for recommended maintenance intervals and support.

# Further information

## Safety information

For more safety information, refer to the safety manual supplied with the device.

## Accessories

You can use the Miracle-Ear App for convenient remote control of your hearing aids. The app also allows you to personalize your hearing experience with more functions.

You can also use a remote control.

Consult your Hearing Care Specialist for further information.

## Symbols used in this document

Points out a situation that could lead to serious, moderate, or minor injuries.

Indicates possible property damage.

Advice and tips on how to handle your device better.

---

Made for

iPhone | iPad | iPod

“Made for iPhone”, “Made for iPad”, and “Made for iPod” mean that an electronic accessory has been designed to connect specifically to iPhone, iPad, or iPod, respectively, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety

and regulatory standards. Please note that the use of this accessory with iPhone, iPad, or iPod may affect wireless performance.

## Troubleshooting

### Problem and possible solutions

---

Sound is weak.

---

- Increase the volume.
  - Replace the empty battery.
  - Clean or exchange the ear piece.
- 

Hearing instrument emits whistling sounds.

---

- Reinsert the ear piece until it fits securely.
  - Decrease the volume.
  - Clean or exchange the ear piece.
- 

Sound is distorted.

---

- Decrease the volume.
  - Replace the empty battery.
  - Clean or exchange the ear piece.
- 

Hearing instrument emits signal tones.

---

- Replace the discharged battery.
-

## **Problem and possible solutions**

---

Hearing instrument does not work.

---

- Turn the hearing instrument on.
  - Gently close the battery compartment completely.
  - Replace the empty battery.
  - Make sure the battery is inserted correctly.
  - Power-on delay is active. Wait several seconds and check again.
- 

Consult your Hearing Care Specialist if you encounter further problems.

## **Country-specific information**

### **USA**

Certification information for hearing aids with wireless functionality:

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[www.miracle-ear.com](http://www.miracle-ear.com)

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Copyright ©2019 Miracle-Ear, Inc.
Distributed By: Miracle-Ear, Inc.
Derechos de autor ©2019 Miracle-Ear, Inc.