Miracle-Ear**LINK**[™]
TV Streamer AV
User Manual

www.miracle-ear.com

Table of Contents

Introduction	4
Overview	6
Basic Use	8
Connecting to Audio Sources.....	12
Pairing	14
Start/Stop Audio Streaming	15
Assembling the Power Adapter	16
Troubleshooting	17
Safety Information.....	21
Regulatory Notices	24

Introduction

The 2.4 GHz Miracle-Ear**LINK**™ TV Streamer AV is designed to stream audio from electronic audio sources directly to your 2.4 GHz wireless hearing aids. The Miracle-Ear**LINK**™ TV Streamer AV can be successfully connected to many electronic audio sources, including televisions, computers, MP3 players, DVD players and cable boxes.

The Miracle-Ear**LINK**™ TV Streamer AV can concurrently stream audio to an unlimited number of users, allowing them to share the same listening experience.


Streaming - Audio or sound transmitted wirelessly from the Miracle-Ear**LINK**™ TV Streamer AV directly to your hearing aids

Package Contents


- Miracle-Ear**LINK**™ TV Streamer AV
- Power Adapter
- Optical Toslink Cable
- 3.5 mm to RCA cable
- User Manual

Introduction

Typical Setup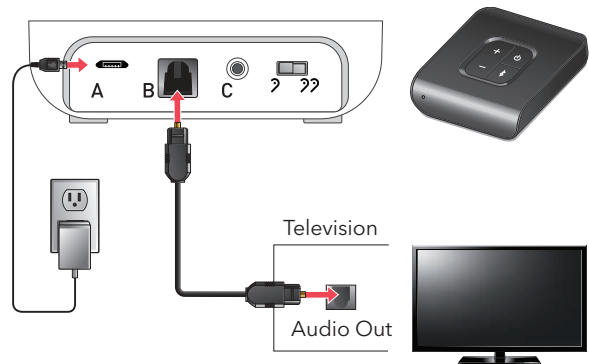

Overview

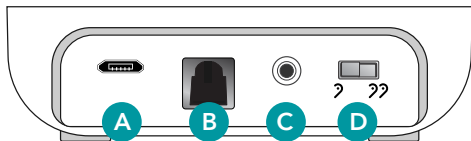- A. Power Supply Jack: A power adapter is provided.
- B. Optical TOSLINK Jack: A TOSLINK cable is provided.
- C. Analog Jack: 3.5 mm-to-RCA cable is provided.
- D. Mono/Stereo Switch
 - ? (MONO) - select if you wear one hearing aid
 - ?? (STEREO) - select if you wear two hearing aids
- ⏻ : Powers the Miracle-Ear**LINK**™ TV Streamer AV on and off
- 📶 : Pair with hearing aids
- + and - : To increase the audio stream volume, press the "+". To decrease the volume, press the "-".

Overview

Tip: Streaming volume can also be adjusted via the app, Miracle-Ear**LINK**™ Remote AV, and/or hearing aid user controls. Review your hearing aid User Manual or contact your Miracle-Ear® Hearing Care Professional for details on your particular system setup.

LED: Provides helpful information about the Miracle-Ear**LINK**™ TV Streamer AV.

LED	CURRENT STATE
Off	Power is off
Solid Green	Power is on, but not streaming audio
Solid Blue	Power is on and streaming audio
Blinking Blue	Hearing aid pairing mode is active. For more details, refer to Pairing section.
Solid Red	Possible fault condition

Basic Use

To quickly start using the Miracle-Ear**LINK**™ TV Streamer AV with a typical setup:

1. Remove the plastic caps from both ends of the TOSLINK cable.
2. Attach a TOSLINK cable into the Audio OUT jack on your electronic audio source. Power on your electronic audio source.


NOTE: If there is no TOSLINK audio OUT jack, try a different electronic audio source or try the 3.5 mm to RCA cable.

3. Attach the other end of TOSLINK cable to the rear of the Miracle-Ear**LINK**™ TV Streamer AV (jack B).
4. Plug the power adapter into the wall socket.


NOTE: If the power adapter does not have the proper plug configuration, refer to Assembling the Power Adapter section on page 16.

5. Insert the power adapter microUSB connector into the rear of the Miracle-Ear**LINK**™ TV Streamer AV (jack A).
6. Press the power button on the Miracle-Ear**LINK**™ TV Streamer AV to turn it on.


Basic Use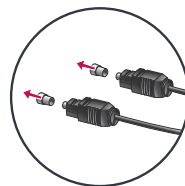


1

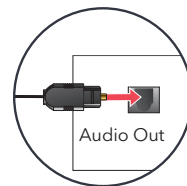
2

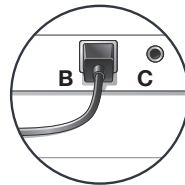
3

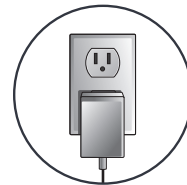4

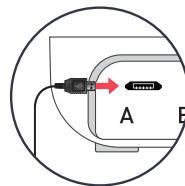5

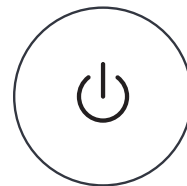6

Basic Use

7. If the front LED glows solid blue, then the Miracle-Ear**LINK**[™] TV Streamer AV has detected audio and is streaming.

Tip: If the LED remains solid green, recheck your setup or review **Troubleshooting** section.


8. Ensure your hearing aids are powered on.

NOTE: Prior to streaming audio for the first time, your hearing aids will need to be paired to the Miracle-Ear**LINK**[™] TV Streamer AV. Refer to the **Pairing** section on page 13.

9. To start streaming audio,
 - Refer to **Start/Stop Audio Streaming** section on page 15 for additional options.
10. Press the +/- button on the Miracle-Ear**LINK**[™] TV Streamer AV to adjust the stream volume.

Basic Use

11. To stop streaming, do one of the following:
 - Turn off the Miracle-Ear**LINK**[™] TV Streamer AV ().
 - Refer to **Start/Stop Audio Streaming** section on page 15 for additional options.
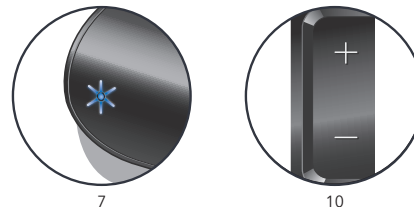
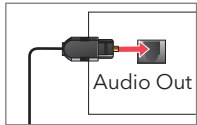
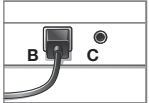
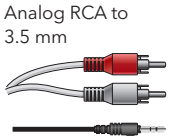
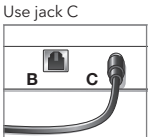
7

10

Connecting to Audio Sources

There are different ways to attach an audio source to the Miracle-Ear**LINK**™ TV Streamer AV. Carefully examine an electronic audio source to determine which AUDIO OUT options it offers.

	Insert cable into the AUDIO OUT jack on your electronic audio source	Attach the other end of cable to the rear of the Miracle-Ear LINK ™ TV Streamer AV
Optical TOSLINK 	 Audio Out	Use jack B 
Analog RCA to 3.5 mm 	 Audio Out Note: Connect the red-to-red and white-to-white.	Use jack C 

Tip: During initial setup, a solid blue LED confirms the audio cable is properly attached to the Miracle-Ear**LINK**™ TV Streamer AV and audio streaming is active.

Pairing

For the Miracle-Ear**LINK**™ TV Streamer AV to interact with your hearing aid(s), you must first pair the Miracle-Ear**LINK**™ TV Streamer AV with each hearing aid. You only need to perform this action one time, as they should automatically recognize each other when both are powered on and within wireless operating range.*

To pair the Miracle-Ear**LINK**™ TV Streamer AV with your hearing aid(s):

1. Ensure your hearing aids are not connected to nearby Bluetooth® devices. To do this:
 - Disable Bluetooth® on your mobile phone.
 - Power off any nearby Miracle-Ear**LINK**™ TV Streamer AV or accessories, which you previously paired to your hearing aids.
2. Power off, then power on your hearing aids.
3. Place your hearing aids within 15 cm (6 inches) of the Miracle-Ear**LINK**™ TV Streamer AV.
4. Press the  button on the Miracle-Ear**LINK**™ TV Streamer AV. The front LED will begin blinking blue. Audio streaming will be temporarily stopped during this step.

*Typical operating range is up to 15 meters (~50 feet). Operational range may be reduced by environmental factors, such as obstructions, interference, and human body positioning. Larger operational distances are achievable, though audio quality and connectivity may decrease.

Pairing

5. For a set of hearing aids:

First LED Blink	Second LED Blink	Pairing Result	Next Step
1	2	Both hearing aids paired	Complete
1	2	Only one hearing aid paired	Repeat steps 1 - 5
1	N/A	No hearing aids paired	Repeat steps 1 - 5

6. For a **single** hearing aids:

LED Blink	Pairing Result	Next Step
1	Successful	Complete
1	Unsuccessful	Repeat steps 1 – 4 and 6

Start/Stop Audio Streaming

Audio streaming from the Miracle-Ear**LINK**™ TV Streamer AV may be started (or stopped) in a variety of ways:

- Miracle-Ear**LINK**™ Remote AV - If preconfigured by your Miracle-Ear® Hearing Care Professional, press the Favorite button (★).
- **Hearing aid user control** - If preconfigured by your Miracle-Ear® Hearing Care Professional, a specific gesture (e.g., a Long Press on a Push Button) can start/stop streaming.

Audio streaming will also stop when:


- An incoming audio signal is not detected.
- Miracle-Ear**LINK**™ TV Streamer AV is turned off.

An audible indicator may play in your hearing aids to signal the start/stop of audio streaming.

Assembling the Power Adapter

The Miracle-Ear**LINK**™ TV Streamer AV may include a variety of plug configurations for use in different countries. If the power adapter requires assembly:

1. Select the appropriate plug configuration used in your country.
2. Slide the plug onto the power adapter until it snaps into place.
3. To remove the plug, press the tab and slide the plug away from the cable.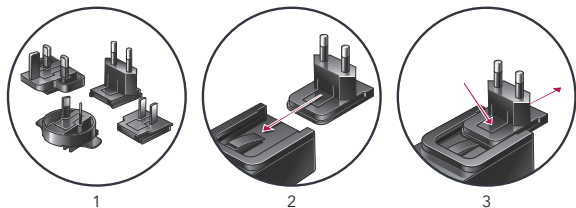

Troubleshooting

SYMPTOM	POSSIBLE CAUSES	SOLUTIONS
One (or both) hearing aids unexpectedly drops out of audio streaming.	Hearing aids may be on the edge of wireless range.	Move closer to the Miracle-Ear LINK ™ TV Streamer AV
	Battery level in hearing aid can no longer support audio streaming.	Replace hearing aid battery with new one.
	The Miracle-Ear LINK ™ TV Streamer AV may not be positioned for optimal streaming.	Re-orient the Miracle-Ear LINK ™ TV Streamer AV spatially by turning a few degrees to right or left.
	Large object, such as furniture or a wall, may be degrading signal.	For best performance, your hearing aids should be within line-of-sight of the Miracle-Ear LINK ™ TV Streamer AV.

Troubleshooting

SYMPTOM	POSSIBLE CAUSES	SOLUTIONS
	Hearing aids are not within wireless range.	Move closer to the Miracle-Ear LINK ™ TV Streamer AV.
LED is solid blue, but no streamed sounds are heard in the hearing aids.	Audio stream needs to be manually started to be heard in hearing aids.	Manually start the audio stream via your remote, or hearing aid user control.
	Hearing aids are not paired with the Miracle-Ear LINK ™ TV Streamer AV.	Refer to Pairing section to associate your hearing aids with the Miracle-Ear LINK ™ TV Streamer AV.

Troubleshooting

SYMPTOM	POSSIBLE CAUSES	SOLUTIONS
	Audio cable may be connected to "audio IN" on electronic audio source.	Ensure the cable is plugged into "audio OUT" jack on the electronic audio source.
I expect to hear audio streaming, but my LED is solid green.	No incoming audio signal is available.	Turn on the electronic audio source.
	Electronic audio source output is not correct format.	Set the digital output on your electronic audio source to PCM or Dolby Audio.

Troubleshooting

Safety Information

SYMPTOM	POSSIBLE CAUSES	SOLUTIONS
LED is solid red.	Possible error condition.	Remove power adapter from jack A, wait 5 seconds and reinsert power adapter.
Streamed audio is not synchronized with the television picture.	Many new televisions (such as HDTVs) take a long time to process the video signal, so they may delay the audio to maintain synchronization.	<ul style="list-style-type: none">• Try using another audio output jack (which may require a different type of cable) on your television.• Try using another electronic audio source. For example, if currently using cable box, try the audio OUT on a television or vice versa.• Some electronic audio sources (such as televisions or A/V receivers) have a built-in option to adjust the timing sequence of audio and video.

Intended Use

Miracle-Ear**LINK**™ TV Streamer AV is an accessory to a wireless air conduction hearing aid. It is intended as a wireless transmitter of sound from a television or other electronic audio source directly to an air conduction hearing aid.


The Miracle-Ear**LINK**™ TV Streamer AV is designed to operate in public and residential environments and are designed to comply with international Electromagnetic Compatibility emissions and immunity standards for medical devices. However, it is still possible that you may experience interference caused by power line disturbances, airport metal detectors, electromagnetic fields from other medical devices, radio signals and electrostatic discharges.

The Miracle-Ear**LINK**™ TV Streamer AV is classified as a Type B applied part under the IEC 60601-1 medical device standard.


If you use other medical devices or wear implantable medical devices such as defibrillators or pacemakers and are concerned that your Miracle-Ear**LINK**™ TV Streamer AV might cause interference with your medical device, please contact your physician or the manufacturer of your medical device for information about the risk of disturbance.

Aircraft Warning: Miracle-Ear**LINK**™ TV Streamer AV should not be used on aircraft unless specifically permitted by the flight personnel.

Safety Information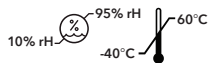

The Miracle-Ear**LINK**™ TV Streamer AV should be stored and transported within the temperature and humidity ranges of -40°C (-40°F) to +60°C (140°F) and 10%-95% relative humidity.

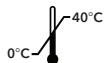The Miracle-Ear**LINK**™ TV Streamer AV is designed to operate indoors in the range of temperatures from 0°C (32°F) to 40°C (104°F).

We are required by regulations to provide the following warnings:

⚠ WARNING: Use of the Miracle-Ear**LINK**™ TV Streamer AV directly next to other electronic equipment should be avoided because it could result in improper performance. If such use is necessary, note as to whether your hearing aids and the other equipment are operating normally.

⚠ WARNING: Use of accessories, components or replacement parts other than those provided by the manufacturer of your Miracle-Ear**LINK**™ TV Streamer AV could result in increased electromagnetic emissions and decreased electromagnetic immunity and could result in degradation of performance.

⚠ WARNING: If Portable Radio Frequency communications equipment is used closer than 30 cm (12 inches) from your Miracle-Ear**LINK**™ TV Streamer AV, degradation of the performance of your hearing aid could result. If this occurs, move away from the communications equipment.

Safety Information

Although the Miracle-Ear**LINK**™ TV Streamer AV is not formally certified (IP rated) for protection against dust or water ingress, per IEC 60529, it has been tested to the IP 5X level. This means that the Miracle-Ear**LINK**™ TV Streamer AV is protected from dust.

The Miracle-Ear**LINK**™ TV Streamer AV does not contain any user-serviceable parts. Do not open or modify it.

Do not connect the Miracle-Ear**LINK**™ TV Streamer AV to any devices not described in this manual.

The expected service life of the Miracle-Ear**LINK**™ TV Streamer AV is 5 years.

Your Miracle-Ear**LINK**™ TV Streamer AV is approved to operate at a radio frequency that is specific to your country or region and might not be approved for use outside your country or region. Be aware that operation during international travel may cause interference to other electronic instruments, or other electronic instruments may cause interference to your Miracle-Ear**LINK**™ TV Streamer AV.

Regulatory Information

WIRELESS TECHNICAL DESCRIPTION

Miracle-Ear**LINK**™ TV Streamer AV operates in the 2.4- 2.4835 GHz band with a maximum effective radiated power of 21 dBm (North America), 10 dBm (rest of world) with transmission modulation type of GFSK and 2 MHz bandwidth. The receiver section of the radio has a bandwidth of 2 MHz.

The Miracle-Ear**LINK**™ TV Streamer AV device has been tested to, and has passed, the following emissions and immunity tests:

- IEC 60601-1-2 radiated and conducted emissions requirements for a Group 1 Class B device as stated in CISPR 11.
- Harmonic distortion and voltage fluctuations affecting the power input source as stated in Table 2 of IEC 60601-1-2.
- RF radiated immunity at a field level of 10 V/m between 80 MHz and 2.7 GHz as well as higher field levels from communications devices as stated in Table 9 of IEC 60601-1-2.
- Immunity to power frequency magnetic fields at a field level of 30 A/m as well as proximity magnetic fields as defined in table 11 of 60601-1-2.
- Immunity to ESD levels of +/- 8 kV conducted discharge and +/- 15 kV air discharge.
- Immunity to electrical fast transients on the power input at a level of +/- 2 kV at a 100 Hz repetition rate.
- Immunity to surges on the power input of +/- 1 kV line to line.
- Immunity to conducted disturbances induced by RF fields on the power input as stated in Table 6 of IEC 60601-1-2.
- Immunity to voltage dips and interruptions on the power input as stated in Table 6 of IEC 60601-1-2.

Regulatory Information

WIRELESS NOTICES

FCC ID: EOA-24TV

IC: 6903A-24TV (Models 800 and 801)

FCC/IC Notice

This device complies with part 15 of the FCC rules and with Industry Canada's license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation of the device. Users are cautioned that a 20 cm (8 inch) minimum separation distance must be maintained from the device to insure compliance with FCC and ISED Canada RF exposure requirements.

NOTE: The manufacturer is not responsible for any radio or TV Streamer interference caused by unauthorized modifications to this equipment. Such modifications could void the user's authority to operate the equipment.

Regulatory Information

Manufactured for:

Miracle-Ear

150 South Fifth Street, Suite 2300

Minneapolis, MN 55402

Instructions for Disposal of Old Electronics

The below instructions are provided for the benefit of disposal/recycling personnel. Please include this manual with your device when disposing of your Miracle-Ear**LINK**™ TV Streamer AV.

Miracle Ear encourages, and local community laws may require, that your Miracle-Ear**LINK**™ TV Streamer AV be disposed of via your local electronics recycling/disposal process.

Regulatory Information

This Miracle-Ear**LINK**™ TV Streamer AV produces radio frequency energy. Interference may occur in the vicinity of equipment marked with the following symbol.

Class II Device

Waste from electronic equipment must be handled according to local regulations

Consult User Manual



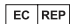
Keep dry

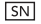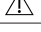Manufactured under license from Dolby Laboratories. Dolby, Dolby Audio, and the double-D symbol are trademarks of Dolby Laboratories.

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The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks is under license.
All trade names and trademarks are properties of their respective owners.
Distributed By Miracle-Ear, Inc.