Miracle-Ear Charge™

MECHARGE™ powerdry Charger Manual

www.miracle-ear.com

Content

Before you start **4**

Components 6

Connecting the USB power cable 7

Charging **8**

Correct position of the hearing aids 11

Cleaning and Drying 12

Cleaning status LED 13

Charging status LEDs 13

Charging tips 18

Further information **26**

Maintenance and care 26

Troubleshooting 27

Important safety information **30**

General warnings 30

Personal safety 32

Product safety 38

Important information **40**

Explanation of symbols 40

Operating, transport, and storage conditions 42

Technical data of the power supply adapter 44

Disposal information 44

Conformance information 45

Country-specific information 47

Before you start

This device is intended to charge our specific hearing aids with built-in power cells (lithium-ion rechargeable batteries). Your Hearing Care Professional will advise you on the compatible models.

The charger is only intended for indoor use. Observe and follow the operating conditions in section "Operating, transport, and storage conditions".

The charger is not intended as a means of storage for transportation of hearing aids.

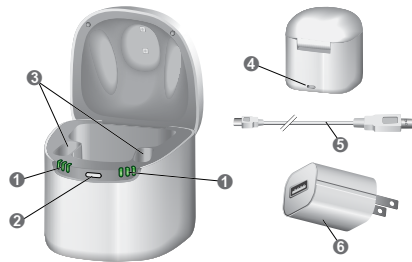
CAUTION

Read this user guide thoroughly and completely and follow the safety information in this document to avoid damage or injury.

The device may not look exactly like the guiding illustrations in these instructions. We reserve the right to make any changes we consider necessary.


Components

-
- | | |
|---------------------------|-----------------------------|
| ① Status LEDs, charging | ④ USB port for power supply |
| ② Status LED, UV cleaning | ⑤ USB power cable |
| ③ Charging slots | ⑥ Power adapter |
-

Connecting the USB power cable

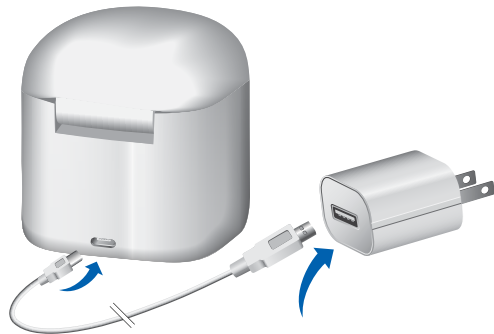- ▶ Connect your USB power cable to the provided power adapter.

Charging

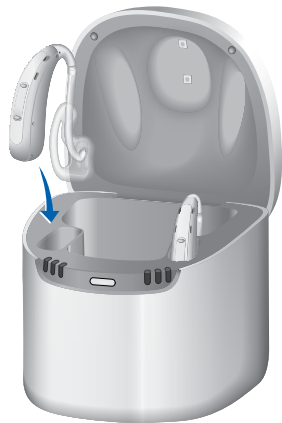
NOTICE

Please make sure that the power plug is easily accessible to remove it from power supply if necessary.

- ▶ Connect the charger to the power supply.
All LEDs for each charging slot turn on successively indicating that the charger has power and is ready for use.

- ▶ Put the hearing aids in the charging slots as shown in the picture. You can charge both hearing aids at the same time.
The hearing aids are automatically turned off and start charging. The charging status is indicated, see table "Charging status" on the next pages.


- ▶ When you want to use your hearing aids, take them out of the charger. They will automatically turn on.
- ▶ You can leave the charger plugged in or remove it from the power supply.

i The device may not look exactly like the guiding illustrations in these instructions. We reserve the right to make any changes we consider necessary.

Correct position of the hearing aids

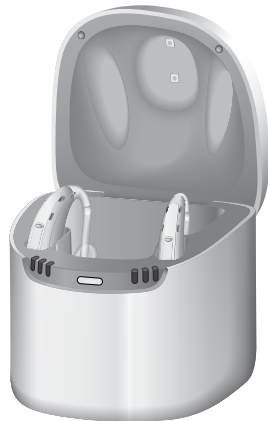- ▶ The left and right hearing aids can be inserted into either charging slot. The slots are not side specific.
- ▶ Make sure the bottom parts of the hearing aids are placed well into the charging slots. The ear piece of the hearing aid should always be placed in the compartment. Correct placement will be indicated by the LED.

Cleaning and Drying

i Drying and cleaning focuses on tubes, ear pieces and molds.

- Put the hearing aids into the charging slots and close the lid.
UVC cleaning* starts automatically after closing the lid and lasts 15 min
- Ongoing UVC cleaning process is indicated by LEDs (see table "Charging and cleaning status")
- Drying starts automatically after closing the lid and lasts 4 hours.

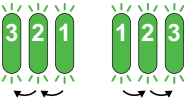
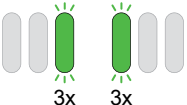
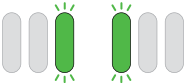
* Ultraviolet C light, used for hygienic cleansing. Significantly reduces risk for ear infections caused by germs, bacteria and fungi.

Cleaning status LED

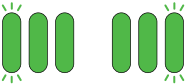
LED	Description
	UVC cleaning status LED is slowly blinking white: Cleaning for 15 minutes and only with closed lid.

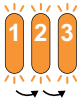
Charging status LEDs

Left LED	Right LED	Description
		No LED light: The charger is not connected to a power supply. No hearing aid is inserted or the hearing aid is inserted incorrectly.

Left LED	Right LED	Description
		LEDs rapidly blinking green in sequence (1, 2, 3): The charger is connected to a power supply and ready to charge.
		1 LED rapidly blinking green 3 times: Charging of the correctly placed hearing aid is initiated.
		1 LED slowly blinking green: Charging is in progress; battery SOC (State of Charge)* is < 33 %.

* Level of battery charge relative to its capacity.

Left LED	Right LED	Description
		1 LED solid green, 2 nd LED slowly blinking: Charging is in progress; battery SOC is 34-66 %.
		2 LEDs solid green, 3 rd LED slowly blinking: Charging is in progress; battery SOC is 67-99 %.
		3 LEDs solid green: The hearing aid is fully charged.

Left LED	Right LED	Description
		LEDs blinking rapidly orange in sequence (1, 2, 3): The charger is cooling down.
		All LEDs blinking red 3 times in every 2 sec: Hearing aid error.
		LEDs blinking red in a sequence (1, 2): Charger error. Read more in section "Troubleshooting".

Charging tips

You may have some questions on how to best charge your hearing aids. Read the questions and answers below. If you have more questions, your Hearing Care Professional will help you.

How often do I need to charge the hearing aids?

We recommend charging the hearing aids daily, even if the power cell is not empty. Charge them overnight so you can be sure to start your day with fully-charged hearing aids.

How long does charging take?

A full charging cycle takes approximately 3 to 4 hours.

After 30 minutes, it is possible to use your hearing aids for up to 7 hours.

Charging times may vary due to the use of the hearing aids and the age of the power cell.

Can I leave the hearing aids in the charger?

Even when fully charged, you can leave the hearing aids in the charger. They automatically stop charging when they are fully charged.

What happens if I forget to charge?

Your hearing aids will automatically switch-off when the power cell is empty. Charge the hearing aids as soon as possible.

Hearing aids which have not been charged for over 6 months may be damaged due to an unrecoverable deep-discharge of the power cell. Deep-discharged power cells cannot be charged anymore and have to be replaced.

What should I pay attention to when charging?

The hearing aids and charger must be clean and dry. Observe the operation conditions in section "Operating, transport, and storage conditions". Avoid charging at high ambient temperatures, as this can shorten the life span and capacity of the power cell.

If you use the charger in a very warm environment, note that it will stop charging in order to cool down if the charger's temperature exceeds 45 °C. The LED light will blink 3 times in 2 seconds when this happens. Charging resumes automatically after the device has cooled down. Do not touch the hearing aids, they may be hot.

What if I intend not to use the hearing aids for several days or longer?

- Short-term storage (up to several days): Turn off your hearing aids via the control element on the housing or place them in the powered charger.

The charger must be connected to the power supply. If the charger is not connected to power supply and you place your hearing aids in the charger, the hearing aids are not turned off.

Note that when you turn off the hearing aids via a remote control or smartphone app, the hearing aids are not turned off completely. They are in standby-mode and still draw some power.

- Long-term storage (weeks, months, ...): First, fully charge your hearing aids. Turn them off via the control element on the housing before storing them. We recommend to use drying products while storing the hearing aids.

At 6 month intervals you will have to charge the hearing aids to avoid an unrecoverable deep-discharge of the power cell.

Deep-discharged power cells cannot be charged anymore and they have to be replaced. We recommend recharging more frequently than 6 months intervals.

Observe the storage conditions in hearing aids' user guide.

What should I pay attention to when using a power strip with energy-saving features?

- Turn on the power strip before you insert the hearing aids in the charger. Otherwise, the hearing aids may not be detected by the charger.
- Do not turn off the power strip while the hearing aids are in charger. Otherwise the hearing aids will automatically turn on and discharge in the charger.

Before you turn off the power strip, remove the hearing aids from the charger and turn them off via the control element on the housing.

What happens if I open the lid during the cleaning process?

The cleaning process will be paused. After closing the lid again, the process will resume.

But if you remove the hearing aids from the charger before end of the cleaning process, the timer will reset.

A new 15 min. cleaning cycle will start after reinserting the hearing aids and closing the lid.

Further information

Maintenance and care

NOTICE

Risk of injuries caused by electric shock!

- ▶ If cleaning is required, disconnect the charger from the power supply.
- ▶ Clean the charger with a soft, dry tissue.
- ▶ Never use running water or immerse the charger in water.
- ▶ Do not sterilize or disinfect the charger.

- ▶ After use and before placing in the charger, wipe the charging pins on the hearing aids with a soft, dry tissue to remove grease or moisture.

Troubleshooting

Problem	Possible solutions
LED light is off	■ Check if the power supply is connected correctly.

Problem	Possible solutions
LED lights are blinking red 3 times every 2 sec. or LED lights are blinking red in a sequence	<p>An error in the hearing aid or in the charger has occurred.</p> <ul style="list-style-type: none">■ Take the hearing aids out of the charger and disconnect the charger from the power supply.■ Reconnect the charger to the power supply and re-insert the hearing aids in the charging slots.■ Charger has reached the temperature limit and needs to cool down. Charging is resumed when the temperature decreases below 45 °C.

Problem

Possible solutions

If the LEDs are still blinking red

- Take the hearing aids out of the charger and disconnect the charger from the power supply.
- Contact your Hearing Care Professional for support.

Consult your Hearing Care Professional if you encounter further problems.

Important safety information

General warnings

- Observe the operating conditions. Refer to the section "Operating, transport, and storage conditions".
- Pay special attention to protect your device from extreme heat (e.g. fire, microwave oven, induction oven or other high induction fields) and sunlight.
- Charge your hearing aids only with the approved charger. Ask your Hearing Care Professional for support.
- Do not disassemble the device.
- Do not use the device in explosive atmospheres.

- Do not use devices that are deformed or obviously damaged.
- Do not use the device when its performance after charging decreases considerably. Return the device to your Hearing Care Professional.
- Do not dispose of the device in fire or water.

In any of the above cases, return your device to the Hearing Care Professional.

WARNING

Risk of electric shock!

- ▶ Connect the charger only with a USB power supply which complies with IEC 60950-1 and/or IEC 62368-1.

Personal safety

WARNING

Risk of burns!

- ▶ Do not place anything other than your hearing aids in the charger.

**WARNING**

Risk of electrical shock!

- ▶ Do not use obviously damaged devices and return them to point of sale.

**WARNING**

Choking hazard!

Your device contains small parts which can be swallowed.

- ▶ Keep hearing aids, batteries and accessories out of reach of children and mentally disabled persons.
- ▶ If parts have been swallowed consult a physician or hospital immediately.

**WARNING**

Note that any unauthorized changes to the product may cause damage to the product or cause injury.

- ▶ Use only approved parts and accessories. Ask your Hearing Care Professional for support.

**WARNING**

Risk of explosion!

- ▶ Do not use your device in explosive atmospheres (e.g. in mining areas).

WARNING

Risk of eye injury

- ▶ In the unlikely event that the ultraviolet (UV) light stays on when the charger lid is opened, do not look directly at the UV light and return the charger to your point of sale.

WARNING

Charging systems may interfere with measuring devices and electronic equipment.

- ▶ In hospitals: The charger complies with CISPR 35 and CISPR 32. Therefore the use of the charger in hospitals should be permissible. Nevertheless the charger may interfere with highly sensitive devices. In those cases appropriate steps shall be taken (increase distance from sensitive devices or disconnect the charger from power supply).

WARNING

Interference with implantable devices

- ▶ The device can be used with all electronic implants that comply with the ANSI/AAMI/ISO 14117:2012 Electromagnetic compatibility standard for human implants.
- ▶ In case your implant does not comply with the ANSI/AAMI/ISO 14117:2012 standard, consult the manufacturer of your implantable device for information about the risk of disturbance.

Product safety

NOTICE

- ▶ Protect your devices from extreme heat. Do not expose them to direct sunlight.

NOTICE

- ▶ Protect your devices from high humidity.

NOTICE

- ▶ Do not dry your devices in the microwave oven.

NOTICE

- ▶ Do not drop. Internal components may get damaged.

NOTICE

- ▶ Do not disassemble the device.

NOTICE

Use of incompatible parts poses a risk of malfunction.

- ▶ Use only the power supply and country specific adapter that is provided with the charger.

Important information

Explanation of symbols

Symbols used in this document

Points out a situation that could lead to serious, moderate, or minor injuries.

Indicates possible property damage.

Advice and tips on how to handle your device better.

Symbols on the device or packaging

Indicates the legal manufacturer of the device.

Do not dispose of the device with general domestic waste. Read more in section "Disposal information".

Read and follow the instructions in the user guide.

Medical device

Operating, transport, and storage conditions

Operating

Temperature	10-35°C 50-95°F
Relative humidity	20-75%
Atmospheric pressure	700-1060 hPa

During extended periods of transport and storage, please observe the following conditions

	Storage	Transport
Temperature	10-40°C 50-104°F	-20-60°C -4-140°F
Relative humidity	10-80%	5-90%
Atmospheric pressure	700-1060 hPa	700-1060 hPa

For other parts, such as hearing aids, other conditions may apply.

Technical data of the power supply adapter

Input voltage:	100-240 V AC
Input current:	0.15-0.3 A
Input frequency:	50-60 Hz

Output voltage:	5 V DC
Output current:	1 A

Disposal information

Recycle devices, accessories and packaging according to local regulations.

To avoid environmental pollution, do not throw the device into household trash. Dispose of devices according to local regulations.

Conformance information

This device complies with Part 15B of the FCC Rules and with CAN ICES-003 / NMB-003.

Operation is subject to the following conditions:

- This device may not cause harmful interference,
- this device must accept any interference received, including interference that may cause undesired operation, and
- the user shall keep a minimal distance of 20 cm (8 in) during the charging process.

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Copyright ©2021 Miracle-Ear, Inc.
Distributed By Miracle-Ear, Inc.
Derechos de autor ©2021 Miracle-Ear, Inc.