MEEASY™ ITE, ITC & CIC

with Audiotone® Pro Technology User Manual

Content

Welcome	4
<hr/>	
Your hearing aids	5
<hr/>	
Hearing aid type	5
Getting to know your hearing aids	5
Components and names	6
Controls	8
Settings	10
Batteries	11
<hr/>	
Battery size and handling tips	11
Replacing batteries	12
Daily use	13
<hr/>	
Turning on and off	13
Inserting and removing the hearing aids	14
Adjusting the volume	17
Changing the hearing program	17
Special listening situations	18
<hr/>	
On the phone	18
Audio induction loops	19
Maintenance and care	20
<hr/>	
Hearing aids	20
Professional maintenance	21

Further information	22
Safety information	22
Accessories	22
Symbols used in this document	22
Troubleshooting	23
Service and warranty	25

Welcome

Thank you for choosing Miracle-Ear® hearing aids to accompany you through everyday life. Like anything new, it may take you a little time to become familiar with them.

This guide, along with support from your Hearing Care Specialist, will help you understand the advantages and greater quality of life your hearing aids offer.

To receive the most benefit from your hearing aids, it is recommended that you wear them every day, all day. This will help you get adjusted to them.

CAUTION

It is important to read this user manual and the safety manual thoroughly and completely. Follow the safety information to avoid damage or injury.

Your hearing aids

This user manual describes optional features that your hearing aids may or may not have.

Ask your Hearing Care Specialist to indicate the features that are valid for your hearing aids.

Hearing aid type

Your hearing aids are customized models that have been individually manufactured based on the anatomy of your ears. They are worn directly in the ear, either in the bowl of the ear or deep inside the ear canal. They are not intended for children under the age of 3 years or persons with a developmental age of under 3 years.

Getting to know your hearing aids

It is recommended to familiarize yourself with your new hearing aids. With the hearing aids in your hand try using the controls and note their location on the hearing aid. This will make it easier for you to feel and press the controls while wearing the hearing aids.


If you have problems pressing the controls of your hearing aids while wearing them, you can ask your Hearing Care Specialist about a remote control option.

Components and names

This user guide describes several types of hearing aids. Use the following pictures to identify the type of hearing aid that you use.


Note that the push button, volume control wheel and vent are individually positioned on your hearing aid to best suit your ear. The appearance of your hearing aids may therefore differ from that of those pictured.

ITE (In The Ear)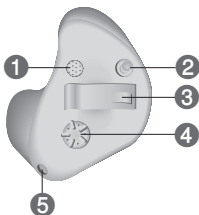


- ① Microphone
 - ② Push button (optional)
 - ③ Battery compartment
 - ④ Volume control wheel (optional)
 - ⑤ Vent (optional)
-

ITC (In The Canal) with controls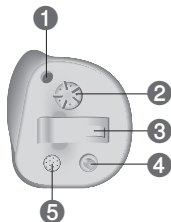

- ① Vent (optional)
 - ② Volume control wheel (optional)
 - ③ Battery compartment
 - ④ Push button (optional)
 - ⑤ Microphone
-

CIC (Completely In the Canal)

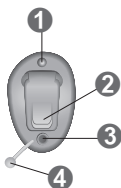- ① Vent (optional)
 - ② Battery compartment (with optional push button function)
 - ③ Microphone
 - ④ Removal cord
-

Controls

With the controls you can, for example, adjust the volume or switch hearing programs. Your hearing aids either have a push button, a volume control wheel, or no control at all. For some hearing aids, the battery compartment also functions as a push button.

Your Hearing Care Specialist has programmed your desired functions to the controls.

Control	Left	Right
Push button	<input type="radio"/>	<input type="radio"/>
Battery compartment with push button function	<input type="radio"/>	<input type="radio"/>
Volume control wheel	<input type="radio"/>	<input type="radio"/>
No control	<input type="radio"/>	<input type="radio"/>

You can also ask your Hearing Care Specialist whether a remote control to control your hearing aids is available.

Push button function

L R

Press briefly:

Program change

○ ○

Press long:

Standby/turn on

○ ○

L = Left, R = Right

Volume control wheel function

Increase volume:

Rotate wheel forward.

Decrease volume:

Rotate wheel backward.

Settings

Hearing programs

1

2

3

4

Read more in section "Changing the hearing program".

Features

- Power-on delay** enables whistle-free insertion of hearing aids.

Read more in section "Turning on and off".

- A **telecoil** is built into the hearing aid, so it can connect with audio induction loops.

Read more in section "Audio induction loops".

Batteries

When the battery is low the sound becomes weaker or you will hear an alert signal. The battery type will determine how long you have to replace the battery.

Battery size and handling tips

Ask your Hearing Care Specialist for recommended batteries.

Battery size: 10 312 13

- Always use the correct battery size for your hearing aids.
- Remove the batteries if you intend not to use the hearing aids for several days.
- Always carry spare batteries.
- Remove empty batteries immediately and follow your local recycling rules for battery disposal.

Replacing batteries


Removing the battery:

- ▶ Open the battery compartment.
- ▶ Remove the battery by hand.

Inserting the battery:

- ▶ If the battery has a protective film, remove it only when you are ready to use the battery.
- ▶ Insert the battery with the "+" symbol facing upwards.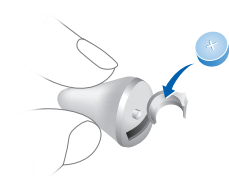

- ▶ Carefully close the battery compartment. If you feel resistance, the battery is not inserted correctly. Do not attempt to close the battery compartment by force. It can be damaged.

Daily use

Turning on and off

You have the following options to turn your hearing aids on or off.

Via battery door:

- ▶ Turning on: Close the battery compartment.
The **default** volume and hearing program are set.
- ▶ Turning off: Open the battery compartment fully.

Via push button:

- ▶ Turning on or off: Press and hold the push button for several seconds. Refer to section "Controls" for the settings of your controls.
After turning on, the **previously** used volume and hearing program remain set.

When wearing the hearing aids, an alert tone can indicate when an hearing aid is being turned on or off.

When the **power-on delay** is activated, the hearing aids turn on after a delay of several seconds. During this time you can insert the hearing aids into your ears without experiencing unpleasant feedback whistling.

The "power-on delay" can be activated by your Hearing Care Specialist.

Inserting and removing the hearing aids


Your hearing aids have been fine-tuned for your right and left ear. The color of the labeling on the hearing aids indicates the side:

- red labeling = right ear
- black or blue labeling = left ear

Inserting a hearing aid:

- ▶ Carefully push the hearing aid into the ear canal.
- ▶ Twist it slightly until it sits well.

Open and close your mouth to avoid accumulation of air in the ear canal.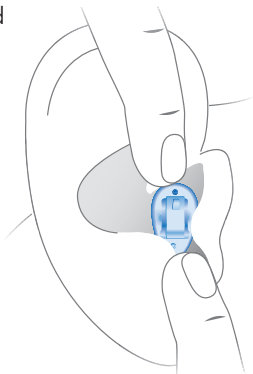

For CIC styles:

- ▶ Pay special attention to the removal cord indicating the bottom.

i

- It may be helpful to insert the right hearing aid with the right hand and the left hearing aid with the left hand.
- If you have problems inserting the ear piece, use the other hand to gently pull your earlobe downwards. This opens the ear canal and eases insertion of the ear piece.


Removing a hearing aid:

NOTICE

Never pull on the battery door or volume control wheel to remove the hearing aid. This could damage your hearing aid.

- ▶ Push lightly on the back of your ear to loosen the hearing aid.
- ▶ If your hearing aid is equipped with a removal cord, remove the hearing aid by pulling the removal cord towards the back of your head.
- ▶ For all other hearing aids: Grip the hearing aid in the ear canal with two fingers and pull it carefully out.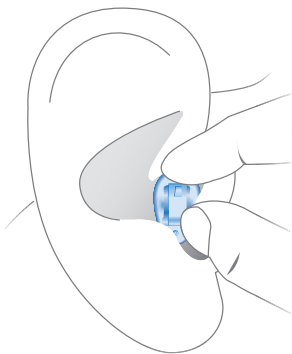

Clean and dry your hearing aids after usage. Read more in section "Maintenance and care".

Adjusting the volume

Your hearing aids automatically adjust the volume to the listening situation.

- ▶ If you prefer manual volume adjustment, rotate the volume control wheel.

Refer to section "Controls" for the settings of your controls.

An optional signal tone can indicate the volume change. Once the maximum or minimum volume has been reached, you may hear an optional signal tone.

Changing the hearing program

Depending on the listening situation, your hearing aids automatically adjust their sound.

Your hearing aids may also have several hearing programs which allow you to change the sound, if needed. An optional signal tone can indicate the program change.

- ▶ To change the hearing program, press the push button briefly.

Refer to section "Controls" for the setting of your controls. Refer to section "Settings" for a list of your hearing programs.

Special listening situations

On the phone

When you are on the phone, turn the receiver slightly so that it does not completely cover your ear.

Telephone program

You may prefer a certain volume when using the phone. Ask your Hearing Care Specialist to configure a telephone program.

- ▶ Switch to the telephone program whenever you are on the phone.

If a telephone program is configured for your hearing aids, it is listed in section "Settings".

Audio induction loops

Some telephones and public places like theatres offer the audio signal via an audio induction loop. With this system, your hearing aids can directly receive the desired signal - without distracting noise in the environment.

You can usually recognize audio induction loop systems by certain signs.

Whether a telecoil is built into your hearing aid is listed in section "Settings". If a telecoil is built in, you have the following options.

Ask your Hearing Care Specialist to configure a telecoil program.

- ▶ Switch to the telecoil program whenever you are in a place with an audio induction loop.

If a telecoil program is configured for your hearing aids, it is listed in section "Settings".

Maintenance and care

To prevent damage it is important that you take care of your hearing aids and follow a few basic rules, which will soon become a part of your daily routine.

Hearing aids


Drying and storage

- ▶ Dry your hearing aids overnight.
- ▶ Ask your Hearing Care Specialist for recommended drying products.
- ▶ For longer periods of non-use, store your hearing aids with open battery compartment and batteries removed in a drying system to avoid the adverse effects of moisture.

Cleaning

For hygiene reasons and to maintain functionality, clean your hearing aids daily.

- ▶ Clean your hearing aids daily with a soft, dry tissue.
- ▶ Your hearing aids may have a wax protection. This prevents the ingress of earwax. Be careful not to damage the wax protection when cleaning your hearing aids.

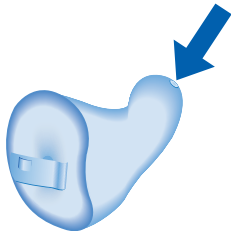- ▶ Never use running water or immerse the devices in water.
- ▶ Never apply pressure while cleaning.
- ▶ Ask your Hearing Care Specialist for recommended cleaning products, special care sets, or more information on how to keep your hearing aids in good condition.

Professional maintenance

Your Hearing Care Specialist can perform a thorough professional cleaning and maintenance.

In the event that the wax protection or microphone protection (windscreen) needs to be replaced, this should be carried out by a professional.

Ask you Hearing Care Specialist for your individual recommendation of maintenance intervals and for support.

Further information

Safety information

For more safety information, refer to the safety manual supplied with the device.

Accessories

The type of available accessories depends on the hearing aid type. An example is: a remote control.

Consult your Hearing Care Specialist for further information.

Symbols used in this document

Points out a situation that could lead to serious, moderate, or minor injuries.

Indicates possible property damage.

Advice and tips on how to handle your device better.

Troubleshooting

Problem and possible solutions

Sound is weak.

- Increase the volume.
 - Gently close the battery compartment completely.
 - Clean the hearing aid.
 - Replace the empty battery.
 - Ask your Hearing Care Specialist to exchange the wax protection and/or the microphone cover.
-

Hearing aid emits whistling sounds.

- Reinsert the hearing aid until it fits securely.
 - Decrease the volume.
 - Clean the hearing aid.
-

Sound is distorted.

- Decrease the volume.
 - Replace the empty battery.
 - Clean the hearing aid.
 - Ask your Hearing Care Specialist to exchange the wax protection and/or the microphone cover.
-

Hearing aid emits signal tones.

- Replace the empty battery.
-

Problem and possible solutions

Hearing aid does not work.

- Turn the hearing aid on.
 - Gently close the battery compartment completely.
 - Replace the empty battery.
 - Make sure the battery is inserted correctly.
 - Power-on delay is active.
Wait several seconds and check again.
-

Consult your Hearing Care Specialist if you encounter further problems.

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Derechos de autor ©2020 Miracle-Ear, Inc.
Distributed by Miracle-Ear, Inc.