ME CHARGE™

Inductive Charger with Lid User Manual

www.miracle-ear.com

Content

Before you start	4
Components	6
Preparing the charger	7
Charging	10
Correct position of the hearing aids	12
Charging status	13
Charging tips	16
Further information	23
Maintenance and care	23
Troubleshooting	24

Important safety information	27
Personal safety	27
Product safety	33
Important information	35
Explanation of symbols	35
Operating, transport, and storage conditions	37
Technical data of the power supply	38
Disposal information	39

Before you start

This device is intended to charge Miracle-Ear® specific BTE and RIC hearing aids with built-in power cells (lithium-ion rechargeable batteries). Your Hearing Care Specialist will advise you on the compatible models.

The charger is only intended for indoor use. Review and follow the operating conditions in section "Operating, transport, and storage conditions".


The charger is not intended as a means of storage for transportation of hearing aids.

CAUTION

Read this user manual thoroughly and completely and follow the safety information in this document to avoid damage or injury.

Components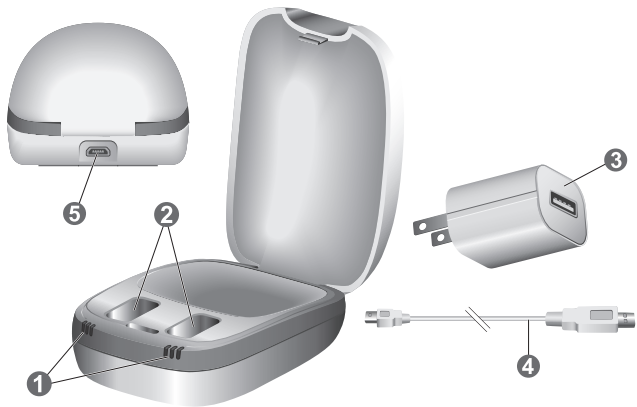

-
- | | |
|------------------|-----------------------------|
| ① Status LEDs | ④ USB power cable |
| ② Charging slots | ⑤ USB port for power supply |
| ③ Power plug | |
-

Preparing the charger

NOTICE

Use of incompatible parts poses a risk of malfunction.

- ▶ Use only the power supply and country specific adapter that is provided with the charger.


WARNING

Risk of electric shock!

- ▶ Connect the charger only with a 5V DC USB power supply which complies with IEC 60950-1 and/or IEC62368-1.

Connecting the USB power cable

- ▶ Connect the micro-USB side of the USB power cable to the charger **1**.
- ▶ Depending on your country, you may have to connect the other side of the USB power cable to a suitable country-specific power plug **2**.

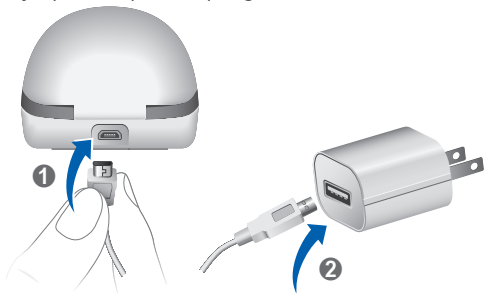Charging

NOTICE

Please make sure that the power plug is easily accessible to remove it from the power supply if necessary.

- ▶ Connect the charger to the power supply.

The charger has power and is ready to use when all LEDs for each charging slot turn on in succession.

- ▶ Put the hearing aids in the charging slots as shown in the picture.
- ▶ The hearing aids automatically turn off and start charging.
- ▶ You can charge both hearing aids at the same time.
- ▶ The charging status is indicated by the LEDs. See "Charging status" section of this manual.
- ▶ When you want to use your hearing aids, take them out of the charger. They will automatically turn on.
- ▶ You can leave the charger plugged in or remove it from the power supply.

Correct position of the hearing aids

- The left and right hearing aids can be inserted into either charging slot. The slots are not side specific.
- The hearing aids sit loosely in the charging slots.
- The ear pieces of the hearing aids should always be placed into the recessed portion of the charger.
- Make sure the bottom part of the hearing aids are placed well into the charging slots.

Charging status

LED	Status description
-----	--------------------

No LED:

Charger is not connected to power supply.

Single green LED turns on successively:

Charger is connected to a power supply and ready to charge. No hearing aid is inserted in the respective charging slot or the hearing aid is not inserted correctly.

LED**Status description**

Flashing orange LED:

Charging status of the hearing aid is displayed before charging starts.

Single orange LED:

Charging status 0 to 33 %

Single green LED:

Charging status 34 to 66 %

Two green LEDs:

Charging status 67 to 99 %

LED**Status description**

Three green LEDs:
Hearing aid is fully charged.

Single orange LED flashes successively:
Cooling phase
Charging is resumed after 5 minutes.

Flashing red LEDs: Error
Read more in section "Troubleshooting".

Charging tips

You may have some questions on how to best charge your hearing aids. Read the questions and answers below. If you have more questions, your Hearing Care Specialist can help you.

How often do I need to charge my hearing aids?

We recommend charging your hearing aids daily, even if the power cell is not empty. Charge them overnight so you can start your day with fully-charged hearing aids.

How long does charging take?

A full charging cycle takes approximately 3 to 4 hours.

After 30 minutes, it is possible to use your hearing aids for up to 7 hours.

Charging times may vary due to the use of the hearing aids and the age of the power cell.

Can I leave the hearing aids in the charger?

Even when fully charged, you can leave the hearing aids in the charger. They automatically stop charging when they are fully charged.

What happens if I forgot to charge?

Your hearing aids will automatically switch-off when the power cell is empty. Charge the hearing aids as soon as possible.

Hearing aids that have not been charged for over 6-months may be damaged due to an unrecoverable deep-discharge of the power cell. Deep-discharged power cells cannot be charged and must be replaced.

Can I use the charger for drying the hearing aids?

Your charger dries the hearing aids while charging. You can also use conventional products to dry your aids.

What if I intend not to use the hearing aids for several days or longer?

- Short-term storage (up to several days): Turn off your hearing aids via the push button or place them in the powered charger.

The charger must be connected to the power supply. If the charger is not connected to power supply and you place your hearing aids in the charger, the hearing aids are not turned off.

Note that when you turn off the hearing aids via a remote control or smartphone app, the hearing aids are not turned off completely. They are in standby-mode and still draw some power.

- Long-term storage (weeks, months, ...): First, fully charge your hearing aids. Then, turn them off via the push button before storing them. We recommend using drying products while storing the hearing aids.

At 6-month intervals you must charge the hearing aids to avoid an unrecoverable deep-discharge of the power cell. Deep-discharged power cells cannot be charged anymore and will have to be replaced. We recommend recharging more frequently than 6-month intervals.

Observe the storage conditions in hearing aids' user manual.

Ask your Hearing Care Specialist for recommended drying products.

What should I pay attention to when charging?

The hearing aids and charger must be clean and dry. Observe the operating conditions in section "Operating, transport, and storage conditions." Avoid charging at high ambient temperatures, as this can shorten the life span and capacity of the power cell.

If you use the charger in a very warm environment, the charger will stop charging in order to cool down. This will happen when the charger's temperature exceeds 45° C. The LEDs will flash orange during the cooling phase. Charging resumes automatically after 5 minutes. Do not touch the hearing aids while the LEDs are flashing orange, because the hearing aids may be hot.

 **What should I pay attention to when using a power strip with energy-saving features?**

- Turn on the power strip before you insert the hearing aids into the charger. Otherwise, the hearing aids may not be detected by the charger.
- Do not turn off the power strip while the hearing aids are in charger. Otherwise, the hearing aids will automatically turn on and discharge in the charger.

Before you turn off the power strip, remove the hearing aids from the charger and turn them off via the push button.

Further information

Maintenance and care

NOTICE

Risk of injuries caused by electric shock!

- ▶ If cleaning is required, disconnect the charger from the power supply.
- ▶ Clean the charger with a soft, dry tissue.
- ▶ Never use running water or immerse the charger in water.
- ▶ Do not sterilize or disinfect the charger.

Troubleshooting

Problem and possible solutions

No LED lights are on.

- Check if the power supply is connected correctly.
-

LEDs are flashing orange successively.

Charger has reached the temperature limit and is now in a cooling phase. Charging is resumed after 5 minutes.

- Do not touch the hearing aids while the LEDs are flashing orange. The hearing aids may be hot.
-

Problem and possible solutions

LEDs are flashing red alternately.

An error in the charger has occurred.

- Take the hearing aids out of the charger and disconnect the charger from the power supply.
- Reconnect the charger to the power supply and re-insert the hearing aids in the charging slots.

If the LEDs are still flashing red:

- Take the hearing aids out of the charger and disconnect the charger from the power supply.
 - Contact your Hearing Care Specialist for support.
-

Problem and possible solutions

Three LEDs are flashing red together.

An error in the hearing aids has occurred.

- Take the hearing aids out of the charger and disconnect the charger from the power supply.
 - Contact your Hearing Care Specialist for support.
-

Consult your Hearing Care Specialist if you encounter further problems.

Important safety information

Personal safety

WARNING

Risk of burns!

- ▶ Do not place anything other than your hearing aids in the charger.

WARNING

Risk of electrical shock!

- ▶ Do not use obviously damaged devices. Return them to your local Miracle-Ear Center.

WARNING

Choking hazard!

Your device contains small parts which can be swallowed.

- ▶ Keep hearing aids, batteries and accessories out of reach of children and cognitively impaired persons.
- ▶ If parts have been swallowed consult a physician or hospital immediately.

WARNING

Note that any unauthorized changes to the product may cause damage to the product or cause injury.

- ▶ Use only approved parts and accessories. Ask your Hearing Care Specialist for support.

WARNING

Risk of explosion!

- ▶ Do not use your device in explosive atmospheres (e. g. in mining areas).

WARNING

Charging systems may interfere with measuring devices and electronic equipment.

- ▶ On airplanes: The charger complies with RTCA/DO-160G, Section 21, Category H. Therefore, use of the charger on airplanes during the flight should be permissible.

The aircraft crew may ask you to not charge the hearing aids during the flight.

WARNING

Charging systems may interfere with measuring devices and electronic equipment.

- ▶ In hospitals: The charger complies with CISPR 35 and CISPR 32. Therefore, use of the charger in hospitals should be permissible.

The charger may interfere with highly sensitive devices. In those cases appropriate steps shall be taken (increase distance from sensitive devices or disconnect the charger from power supply).

WARNING

Interference with implantable devices

- ▶ The device can be used with all electronic implants that comply with the ANSI/AAMI/ISO 14117:2012 Electromagnetic compatibility standard for human implants.
- ▶ In case your implant does not comply with the ANSI/AAMI/ISO 14117:2012 standard, consult the manufacturer of your implantable device for information about the risk of disturbance.

Product safety

NOTICE

- ▶ Protect your devices from extreme heat. Do not expose them to direct sunlight.

NOTICE

- ▶ Protect your devices from high humidity.

NOTICE

- ▶ Do not dry your devices in the microwave oven.

NOTICE

- ▶ Do not use the device when the status indicators are red.

NOTICE

While connected to power, the charger's magnetic field may cause interference to electrical devices and may delete stored data.

- ▶ While connected to power, keep the charger away from credit cards, computers, monitors, television sets, storage media and other electronic equipment.

NOTICE

- ▶ Do not drop. Internal components may get damaged.

NOTICE

- ▶ Do not disassemble the device.

Important information

Explanation of symbols

Symbols used in this document

Points out a situation that could lead to serious, moderate, or minor injuries.

Indicates possible property damage.

Advice and tips on how to handle your device better.

Symbols on the device or packaging

Indicates the legal manufacturer of the device.

Do not dispose of the device with general domestic waste. Read more in section "Disposal information".

Read and follow the instructions in the user guide.

Operating, transport, and storage conditions

Operating conditions

Temperature	10 to 30 °C
-------------	-------------

Relative humidity	20 to 75 %
-------------------	------------

During extended periods of transport and storage, please observe the following conditions:

	Storage	Transport
Temperature	10 to 40 °C	-20 to 60 °C
Relative humidity	10 to 80 %	5 to 90 %

For other parts, such as hearing aids, other conditions may apply.

Technical data of the power supply

Input voltage:	100-240 V AC
Input current:	0.5 A
Input frequency:	50-60 Hz
Output voltage:	5 V DC
Output current:	1 A

Disposal information

Recycle devices, accessories and packaging according to local regulations.

This device complies with Part 18 of the FCC Rules and with CAN RSS-216 / CNR-216.

Operation is subject to the following three conditions:

- This device may not cause harmful interference.
- This device must accept any interference received, including interference that may cause undesired operation.
- The user shall keep a minimal distance of 20 cm (8 in) during the charging process.

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Copyright ©2020 Miracle-Ear, Inc.
Distributed By: Miracle-Ear, Inc.
Derechos de autor ©2020 Miracle-Ear, Inc.