



Miracle-Ear**SPARK**[™]

User Manual

MEMINI E 5P R-R AI PH (5/4),
MEMINI E D 5P R-R AI PH,
MECHARGE RIC AI 5P FP



Your hearing aid and charger details

- ① If no box is checked and you do not know the model of your hearing aid or charging accessory, please ask your hearing care professional.
- ① The hearing aids described in this user guide contain a built-in and non-removable rechargeable lithium-ion battery.
- ① Please also read safety information regarding handling rechargeable hearing aids (chapter 23).

Hearing aid models

- MEMINI E 5P R-R AI PH (5/4)
- MEMINI E D 5P R-R AI PH

Earpieces

- Dome
- SlimTip
- cShell

Charging accessory

MECHARGE RIC AI 5P FP

Contents

Your hearing aid and charger details	3	Further information	
Welcome	7	16. Environmental conditions	43
1. Quick guide		17. Care and maintenance	45
2. Parts of the hearing aid and charger	8	18. Exchanging the earwax protection	48
Using the charger	10	19. Service and warranty	54
3. Preparing the charger	14	20. Compliance information	56
4. Charging	15	21. Information and description of symbols	63
5. Left and right hearing aid markings	24	22. Troubleshooting	68
6. Putting on the hearing aid	25	23. Important safety information	74
7. Removing the hearing aid	26		
8. Multi-function button	27		
9. On/Off	28		
10. Touch Control	30		
11. Connectivity overview	31		
12. Initial pairing	32		
13. Phone calls	35		
14. Flight mode	39		
15. Rebooting your hearing aid	42		

Welcome

Thank you for choosing Miracle-Ear® hearing aids to accompany you through everyday life. Like anything new, it may take you a little time to become familiar with them.

This guide, along with support from your hearing care professional, will help you understand the advantages and greater quality of life your hearing aids offer. Training is not required for handling of this device.


To receive the most benefit from your hearing aids, it is recommended that you wear them every day, all day. This will help you get adjusted to them.

1. Quick guide



Charging your hearing aids

i Before using your hearing aid for the first time, it is recommended to charge it for 3 hours.

After placing your hearing aid in the charger, the indicator light will show the charging state of the battery until the hearing aid is fully charged. When fully charged, the indicator light will be constantly on, i.e. solid green.

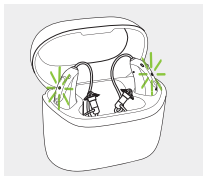

Left and right hearing aid markings

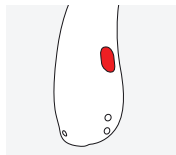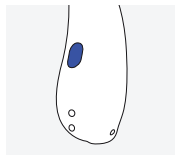

Blue marking for
left hearing aid.

Red marking for
right hearing aid.

Multi-function button with indicator light


The button has several functions, the primary function is as an on/off switch.

Together with your hearing care professional, you can decide if it also allows for volume control, program change and other functionalities. This is indicated in your individual instructions.

Phone calls: If paired with a Bluetooth® enabled phone, a short press will accept and a long press will reject an incoming call.

On/Off: Firmly press the lower part of the button for 3 seconds until the indicator light blinks:

On: indicator light is solid green
Off: indicator light is solid red

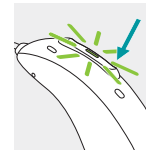

Enter flight mode: When device is off, press the lower part of the button for 7 seconds until the solid orange light appears. Then release the button.

Touch Control:


If paired with a Bluetooth® enabled device, multiple functions can be accessed using Touch Control, please refer to chapter 10. This is also indicated in your individual instructions. To use Touch Control, tap the top of your ear twice.

2. Parts of the hearing aid and charger

The following pictures show the hearing aid model described in this user guide. You can identify your personal models by:

- checking “Your hearing aid & charger details” on page 3.
- comparing your earpiece and hearing aid with the following models.


Possible earpieces

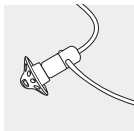

Dome




SlimTip

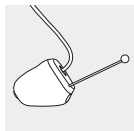


cShell

MEMINI E 5P R-R AI PH MEMINI E D 5P R-R AI PH

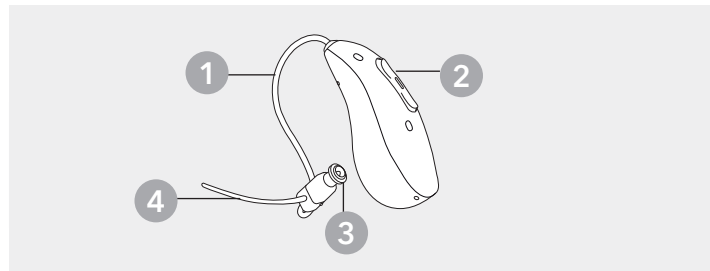


-
- | | |
|--|---------------------------------------|
| 1 Tube | 3 Speaker (without earpiece attached) |
| 2 Multi-function button with indicator light | 4 Anchor (optional) |
-

MECHARGE RIC AI 5P FP

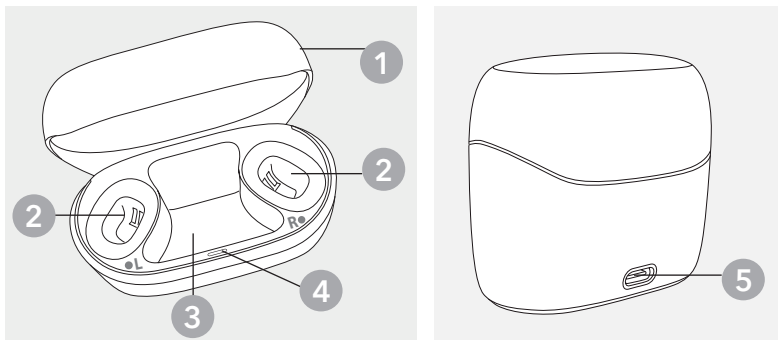


-
- | | |
|--|--|
| 1 Charger case lid | 4 Indicator light for internal battery status |
| 2 Hearing aid charging slots with left/right indicators | 5 USB-C connection to external power supply |
| 3 Cavity for earpieces | |
-

External power supply

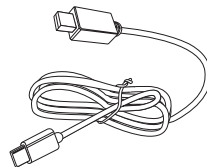

USB cable

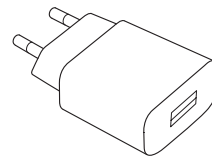

Power supply with wall plug adapter

Power adapter and charger specification

Power adapter output voltage: 5 VDC +/-5%, current 500 mA - 1 A

Power adapter input voltage: 100-240 VAC, 50/60Hz, current 0,25 A


Charger input voltage: 5 VDC +/-5%, current 350 mA

USB cable specifications: 5 V min. 1 A, USB-A to USB-C, maximum length 3 m

3. Preparing the charger

Connecting to the power supply

- a) Plug the larger end of the charging cable into the power adapter.
- b) Plug the smaller end into the USB port on the charger.

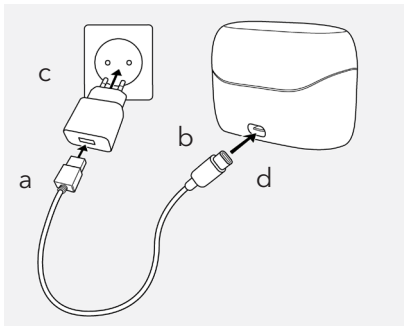

- c) Plug the power adapter into a power outlet.
- d) The indicator light around the USB port is green when the charger is connected to the power supply.


Make sure you have access to the power adapter during charging.

4. Charging

4.1 Using and charging the charger



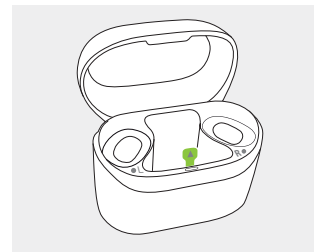

- ⓘ If the built-in battery is fully charged, the hearing aids can be charged up to 3 times.
- ⚠ Always make sure that the hearing aids and charger are dry and clean before charging.

Before using your charger, pull the insulating tab out to activate the internal battery. After pulling out the insulating tab, the indicator light will be red for 10 seconds.





The charger has an internal battery, which means it can't be switched off. Even when unplugged from a power outlet, it will continue charging the hearing aids until its internal battery runs out.

Once it runs out, the charger effectively switches off as it can no longer provide power to charge the hearing aids. Even when the charger's internal battery is completely discharged, the charger can still charge the hearing aids once connected to the power supply. The indicator light on the charger shows the status of the charger's internal battery.

Charger connected to a power supply		
Indicator light of the charger	Battery status	Description and actions
Solid green 	Fully charged	The internal battery of the charger is fully charged. No action needed.
Blinking green 	Charging >1 charging cycle	The internal battery of the charger has enough power to charge your hearing aids at least once while not connected to a power supply.
Blinking red 	Charging <1 charging cycle	The internal battery of the charger does not have enough power to charge your hearing aids at least once while not connected to a power supply. Keep the charger connected to a power supply until it is completely charged.
Solid red 	Battery error / charging error	An error was detected while trying to charge the internal battery of the charger. Check if your hearing aids still can be charged. Contact your hearing care professional.

Charger not connected to a power supply

Indicator light turns off after a short time to save battery

Indicator light of the charger	Battery status	Description and actions
Solid green 	>1 charging cycle left	The internal battery of the charger has enough power to charge your hearing aids at least once. No action needed.
Blinking green 	<1 charging cycle left	The internal battery of the charger is running low. Connect the charger to a power supply.
No indicator light	Discharged battery	The internal battery is completely discharged. Connect the charger to a power supply.


4.2 Charging the hearing aids

- ① Before using your hearing aid for the first time, it is recommended to charge it for 3 hours.
- ① Low power: You will hear two beeps when the battery is low. You will have approximately 60 minutes before you have to charge the hearing aid, after this time the hearing aid automatically shuts down (this can vary, depending on the hearing aid settings).
- ① The hearing aid contains a built-in and non-removable rechargeable lithium-ion battery.
- ① The hearing aid must be dry before charging.
- ① Please ensure to charge and use the hearing aid within operating temperature range: +5 to +40 ° Celsius (41 to 104 ° Fahrenheit).

Using the charger

1

Insert the hearing aids into the charging slots and the earpieces in the large cavity. Make sure the left and right hearing aid markings match the left (blue), right (red) indicators next to the charging slots. The hearing aids will automatically switch off once inserted into the charger.

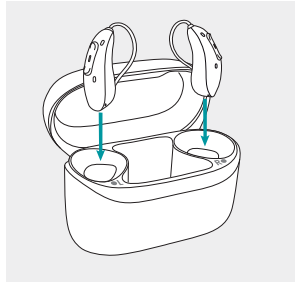





2

The indicator light on the hearing aid will show the charging state of the battery until the hearing aid is fully charged. When fully charged the indicator light will remain on, i.e. solid green.

The charging process will automatically stop when the battery is fully charged, so the hearing aid can be safely left in the charger. It can take up to 3.5 hours to charge the hearing aid. The charger lid can be closed while charging.






Charging times

Indicator light	Charging state	Approximate charging times
	0 - 10%	
	11 - 80%	30 min (20%) 60 min (40%) 120 min (70%)
	81 - 99%	
	100%	3.5 h

3


Remove the hearing aids from the charging slots by lifting them out of the charger.

The hearing aid can be set to switch on automatically when removed from the charger. In this case the indicator light will start blinking. The solid green light indicates the hearing aid is ready for use.

-  Do not hold the tubes to remove the hearing aids from the charger as this can damage the tubes.
-  The hearing aids stored in the charger will go into Sleep mode after 15 minutes if they are fully charged.
-  Always close the lid when charging your hearing aids in a humid environment.

5. Left and right hearing aid markings

There is a red or blue marking on the back of the hearing aid and on the speaker. This will tell you if it is a left or a right hearing aid.

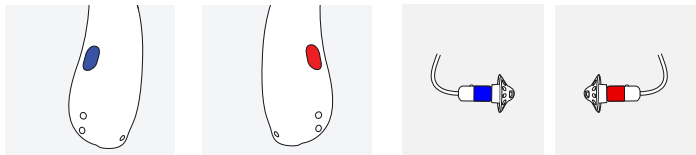

Blue marking for **left hearing aid**.

Red marking for **right hearing aid**.

6. Putting on the hearing aid


1

Place the hearing aid behind your ear.




2

Insert the earpiece into your ear canal.

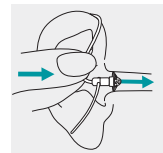


3

If there is an anchor attached to the earpiece, tuck it into the bowl of your ear to secure your hearing aid.

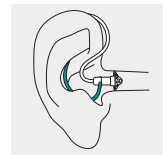

7. Removing the hearing aid

Pull on the bend of the tube and remove the hearing aid from behind the ear.

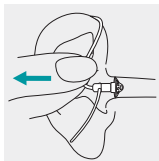


8. Multi-function button

The multi-function button has several functions.

The primary function is as an on/off switch. Together with your hearing care professional, you can decide if it also allows for volume control, program change and other functionalities.


If the hearing aid is paired with a Bluetooth® enabled phone, a short press on the upper or the lower part of the button will accept an incoming call and a long press will reject an incoming call – refer to chapter 14.

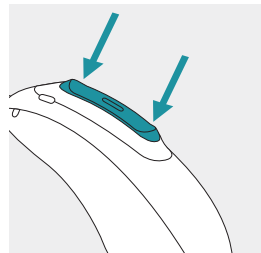

9. On/Off

Switch on hearing aid

The hearing aid is configured to switch on automatically when removed from the charger. If this function is not configured, firmly press and hold the lower part of the button for 3 seconds until the indicator light blinks. Wait until the solid green light indicates the hearing aid is ready.

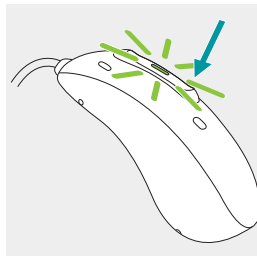

Switch off hearing aid

Press and hold the lower part of button for 3 seconds until the solid red light indicates the hearing is off.


	Blinking green	Hearing aid is switching on
	Solid red for 3 seconds	Hearing aid is switching off

i When you switch on the hearing aid you might hear a start-up melody.

10. Touch Control


If paired with a Bluetooth® enabled device, multiple functions can be accessed using Touch Control, e.g. accept/end phone call, pause/resume streaming, start/stop voice assistant. This is indicated in your individual "Hearing aid instructions".

To use Touch Control, tap the top of your ear twice.

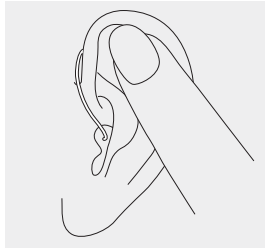

11. Connectivity overview

The illustration below shows the connectivity options available for your hearing aid.

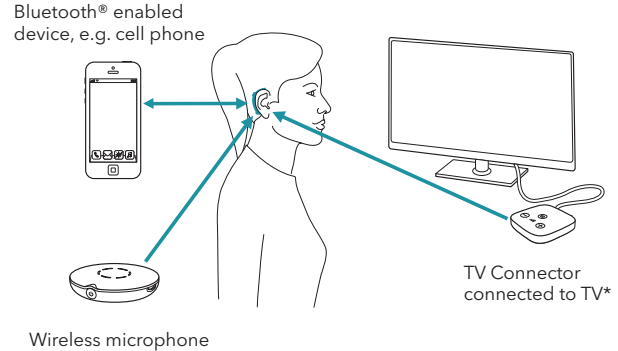

* The TV Connector can be connected to any audio source such as a TV, PC or hi-fi system.

12. Initial pairing

If you use the mobile application to pair your hearing aids with your phone, you can skip the instructions in chapter 12.1. The pairing happens automatically through the mobile application.

12.1 Initial pairing to a Bluetooth® enabled device


i It is only necessary to perform the pairing procedure once with each device featuring Bluetooth® wireless technology. After the initial pairing, your hearing aid will connect automatically to the device. Initial pairing can take up to 2 minutes.

1

On your device (e.g. a phone), ensure that Bluetooth® wireless technology is enabled and search for Bluetooth® enabled devices in the connectivity setting menu.

2

Switch on both hearing aids. You now have 3 minutes to pair your hearing aids with your device.

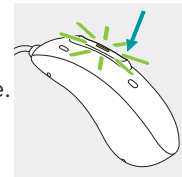

3

Your device shows a list of Bluetooth® enabled devices. Select the hearing aid from the list to simultaneously pair both hearing aids. A beep confirms successful pairing.

i For more information about pairing instructions for Bluetooth® wireless technology specific to some of the most popular phone manufacturers, contact your hearing care professional.

12.2 Connecting to the device


After your hearing aids have been paired to your device, they will automatically connect again when switched on.

- ① The connection will be maintained as long as the device remains ON and within range.
- ① Your hearing aid can be connected to up to two devices and paired to up to eight devices.

After the device goes out of range, you can trigger an automatic reconnection by pressing the multi-function button for less than 2 seconds.

13. Phone calls

Your hearing aids connect directly with Bluetooth® enabled phones. When paired and connected to your phone, you will hear notifications, incoming call notification and the caller's voice directly in your hearing aids. Phone calls are "hands free", this means your voice is picked up by the hearing aid's microphones and transmitted back to the phone.

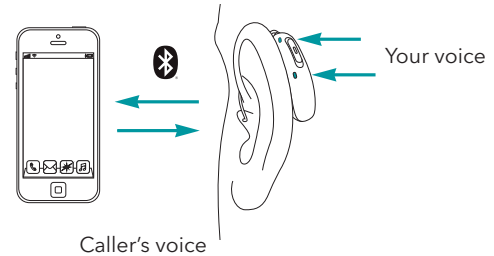


13.1 Making a call

Enter the phone number and press the dial button. You will hear the dialing tone through your hearing aids. Your voice is picked up by the hearing aid microphones and transmitted to the phone.

13.2 Accepting a call



When receiving a call, an incoming-call notification will be heard in the hearing aids (e.g. ring tone or voice notification).

The call can be accepted by tapping the top of your ear twice, a short press on the upper or lower part of the multi-function button on the hearing aid (less than 2 seconds), or directly on your phone.

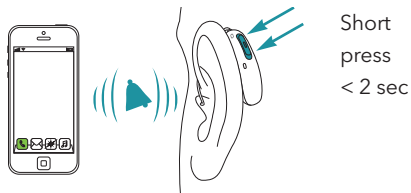


13.3 Ending a call

A call can be ended by a long press on the upper or lower part of the multi-function button on the hearing aid (more than 2 seconds) or directly on your phone. The phone call can also be ended by tapping the top of your ear twice.

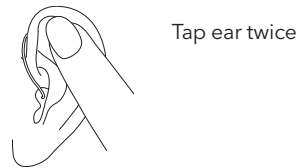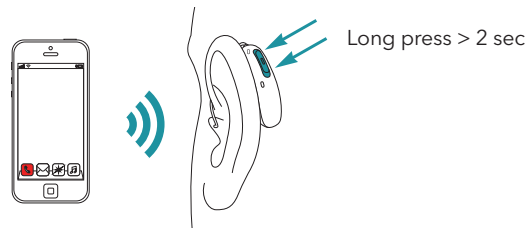

13.4 Rejecting a call

An incoming call can be rejected by a long press on the upper or lower part of the multi-function button on the hearing aid (more than 2 seconds) or directly on your phone.

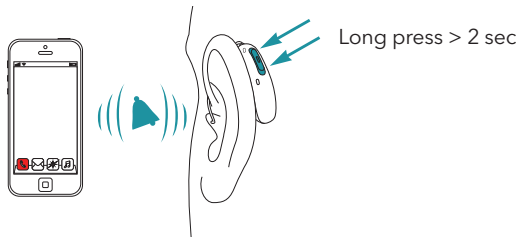

14. Flight mode


Your hearing aids operate between 2.4 GHz-2.48 GHz frequency range. When flying some operators require all devices to be switched into flight mode. Entering flight mode will not disable normal hearing aid functionality, only the Bluetooth® connectivity functions.

14.1 Enter flight mode

To disable the wireless function and enter flight mode in each hearing aid:


1. 

If the hearing aid is turned off, proceed to Step 2. If the hearing aid is on, turn it off by pressing and holding the lower part of the button for 3 seconds, until the indicator light turns solid red for 3 seconds, meaning that the hearing aid is switching off.

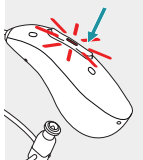


2. 

Press and hold the lower part of the button. The green indicator light blinks when the hearing aid starts up.

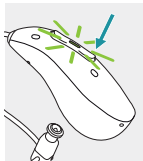


3. 

Keep pressing the button for 7 seconds, until the indicator light is solid orange, then release the button. The solid orange light indicates the hearing aid is in flight mode.

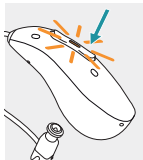

14.2 Exit flight mode

To enable the wireless function and exit flight mode in each hearing aid, switch the hearing aid off and then on again - refer to chapter 9.

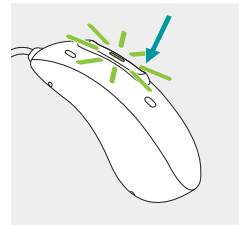

In flight mode, your hearing aid cannot connect directly to your phone.

15. Rebooting your hearing aid

Use this technique if your hearing aid is in an error state. This will not remove or delete any program or settings.

Press the lower part of the multi-function button for at least 15 seconds. It does not matter if the hearing aid is switched on or off before you start pressing the button.

After 15 seconds, the hearing aid will restart with the indicator light blinking green.

16. Environmental conditions

The product is designed such that it functions without problems or restrictions if used as intended, unless otherwise noted in this user guide.

Please ensure to use, charge, transport and store the hearing aids according to the following conditions:

	Charging and operating	Transport	Storage
Temperature	+5 to +40 °C (41 to 104 °F)	-20 to +60 °C (-4 to 140 °F)	-20 to +60 °C (-4 to 140 °F)
Humidity (non condensing)	0% to 90%	0% to 93%	0% to 93%
Atmospheric pressure	500 to 1060 hPa	500 to 1060 hPa	500 to 1060 hPa

Prolonged storage at temperatures lower than 10 °C and higher than 30 °C may negatively affect the battery performance of the product.

If the device has been subject to storage and transport conditions outside of the recommended use conditions, bring the device back into recommended use conditions before starting the device.

These hearing aids are classified as IP68. This means that they are water and dust resistant and designed to withstand daily life situations.

This charger is classified as IP22. This means that the charger is protected from vertically falling drops of water when the charger is tilted up to 15 degrees from the horizontal. IP22 is satisfied when the charger lid is closed.

Please ensure to charge the hearing aids regularly during long-term storage.

17. Care and maintenance

Expected Service Life:

The hearing aids and the compatible charger have the expected service life of five years. The devices are expected to remain safe for use for this duration.

Commercial service period:

Routine and diligent care of your hearing aids and charger will contribute to the outstanding performance for the duration of the expected service life of your devices.

Manufacturer will provide a minimum of a five year repair service period after the respective hearing aid, charger and essential components have been phased out of the product portfolio.


Please use the following specifications as a guideline. For further information regarding product safety, see chapter 23.


General information

Before using hair spray or applying cosmetics, you should remove your hearing aid from your ear, because these products may damage it.

Your hearing aids are resistant to water, sweat and dust under the following conditions:

- After exposure to water or dust, the hearing aid is cleaned and dried.
- The hearing aid is used and maintained as described in this user guide.

 Always make sure that the hearing aids and charger are dry and clean before charging.

 Always make sure that the charger is disconnected from the power mains before cleaning.

Daily

Hearing aid: Inspect the earpiece for earwax and moisture deposit and clean the surfaces with a lint-free cloth. Never use cleaning agents such as household detergents, soap, etc. for cleaning your hearing aid. It is not recommended to rinse with water. If you need to clean your hearing aid intensively, ask your hearing care professional for advice and information on filters or drying methods.

Charger: Make sure that charging slots are clean. Otherwise, clean the surfaces with a soft, lint-free cloth. Never use cleaning agents such as household detergents, soap, etc. for cleaning your charger. Do not rinse your charger with water.

Weekly

Hearing aid: Clean the earpiece with a soft, damp cloth or with a special cleaning cloth for hearing aids. For more in-depth maintenance instructions, please see your hearing care professional. Clean the charging contacts on the hearing aid with a soft, damp cloth.

Charger: Remove dust or dirt from the charging slots with a soft, lint-free cloth.


18. Exchanging the earwax protection

Your hearing aid is equipped with an earwax protection to protect the speaker from earwax damage. Please check the earwax protector regularly and exchange it if it looks dirty or if the hearing aid volume or sound quality is reduced. The earwax protection should be replaced every four weeks on a regular receiver. Your hearing care professional can advise you on individual replacement cycles.

18.1 Removing the earpiece from the speaker


1

Remove the earpiece from the speaker by holding the speaker in one hand and the earpiece in the other.

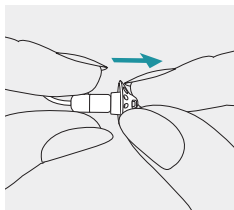


2

Gently pull off the earpiece to remove.

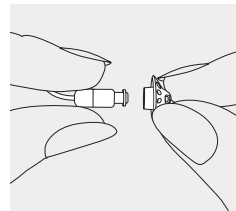

3


Clean the speaker with a lint-free cloth.

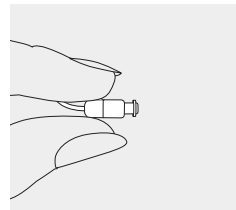

18.2 Exchanging the earwax protection with CeruStop


1

Insert the removal side of the exchange tool into the used earwax protector. The shaft of the holder should touch the rim of the earwax protector.

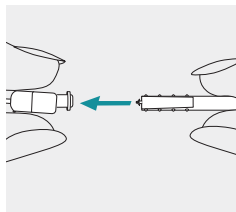


2

Carefully pull and remove the earwax protector straight out of the speaker. Please do not twist the earwax protector when removing.

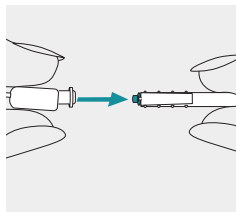


3

To insert the new earwax protector, gently push the insertion side of the exchange tool straight into the hole of the speaker until the outer ring is perfectly aligned.

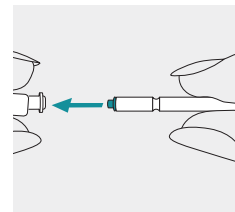

4


Pull the tool straight out. The new earwax protector will remain in place. Now attach your earpiece to the speaker.

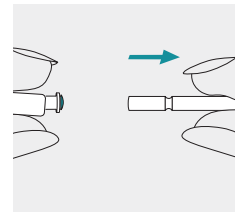

18.3 Attaching the earpiece to the speaker


1

Hold the speaker in one hand and the earpiece in the other.

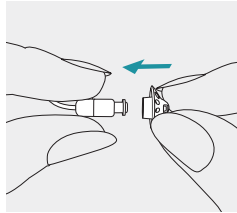


2

Slide the earpiece over the speaker sound outlet.

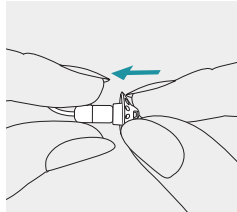

3

The speaker and the earpiece should fit perfectly together.

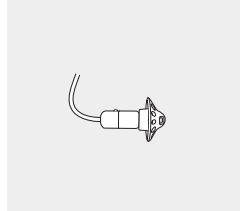

19. Service and warranty

Local warranty

Please ask the hearing care professional where you purchased your hearing aids and charger about the terms of the local warranty.

International warranty

The manufacturer offers a one year limited international warranty, valid from the date of purchase. This limited warranty covers manufacturing and material defects in the hearing aid and charger respectively, but not accessories such as tubes, earpieces, external receivers. The warranty only comes into force if a proof of purchase is shown.

The international warranty does not affect any legal rights that you might have under applicable national legislation governing sale of consumer goods.

Warranty limitation

This warranty does not cover damage from improper handling or care, exposure to chemicals or undue stress. Damage caused by third parties or non-authorized service centers renders the warranty null and void. This warranty does not include any services performed by a hearing care professional in their office.

Serial numbers

Hearing aid left side: _____

Hearing aid right side: _____

Charger: _____

Date of purchase: _____

Authorized hearing care
professional (stamp/signature):

20. Compliance information

The hearing aid described in this user guide is certified under:

MEMINI E 5P R-R AI PH (5/4)
MEMINI E D 5P R-R AI PH

USA

FCC ID: KWC-IRP

Notice 1:

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- 1) this device may not cause harmful interference, and
- 2) this device must accept any interference received, including interference that may cause undesired operation.

Notice 2:

Changes or modifications made to this device not expressly approved by the manufacturer may void the FCC authorization to operate this device.

Notice 3:

This device has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This device generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However,

there is no guarantee that interference will not occur in a particular installation. If this device does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the device and receiver.
- Connect the device into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Charger:

- Please keep at least 20 cm separation distance between the charger and the human body during operation.

Notice 4:

- The device is in compliance with the portable RF exposure requirements listed on RSS 102 and FCC 47 CFR Part 2.1093 for a distance of use equal or greater than 0.5 mm.

Radio information of your hearing aid

Antenna type	Resonant loop antenna
Operation frequency	2.4 GHz - 2.48 GHz
Modulation	GFSK, Pi/4-DQPSK, 8-DPSK, GMSK
Radiated power	< 2.5 mW
Bluetooth®	
Core version	Bluetooth® Core 5.3, Dual-Mode
Profiles supported	HFP (Hands-free profile), A2DP

Electromagnetic emissions

Manufacturer's Declaration: The devices are intended for use in the electromagnetic environments listed below. The user should assure that they are used in such environments.

Emissions Test	Compliance	Electromagnetic environment guidance
RF emissions CISPR 11	Group 1	The device generates or uses RF energy only for their internal functioning. Therefore, the RF emissions are very low and are not likely to cause any interference to nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for use in domestic establishment and in establishment directly connected to a low voltage power supply network which supplies buildings used for domestic purposes.

Electromagnetic immunity

Immunity Test	IEC60601-1-2 Test Level	Compliance Level
Electrostatic Discharge (ESD) IEC 61000-4-2	+/- 8 kV contact +/- 2 kV, +/- 4 kV, +/- 8 kV, +/- 15 kV air	+/- 8 kV contact +/- 2 kV, +/- 4 kV, +/- 8 kV, +/- 15 kV air
Radiated RF Electromagnetic Field Immunity Test IEC 61000-4-3	10 V/m 80 MHz - 2.7 GHz 80% AM at 1 kHz	10 V/m 80 MHz - 2.7 GHz 80% AM at 1 kHz

Immunity Test	IEC60601-1-2 Test Level	Compliance Level
	9 V/m; 710, 745, 780 MHz; Pulse modulation 18 Hz	9 V/m; 710, 745, 780 MHz; Pulse modulation 18 Hz
	9 V/m; 5240, 5500, 5785 MHz; Pulse modulation 217 Hz	9 V/m; 5240, 5500, 5785 MHz; Pulse modulation 217 Hz
Proximity Magnetic Field IEC 61000-4-3	27 V/m; 385 MHz; Pulse modulation 18 Hz	27 V/m; 385 MHz; Pulse modulation 18 Hz
	28 V/m; 450 MHz FM +/- 5 kHz deviation; 1 kHz sine	28 V/m; 450 MHz FM +/- 5 kHz deviation; 1 kHz sine
	28 V/m; 810, 870, 930 MHz; Pulse modulation 18 Hz	28 V/m; 810, 870, 930 MHz; Pulse modulation 18 Hz
	28 V/m; 1720, 1845, 1970, 2450 MHz; Pulse modulation 217 Hz	28 V/m; 1720, 1845, 1970, 2450 MHz; Pulse modulation 217 Hz
Rated power frequency magnetic fields IEC 61000-4-8	30 A/m 50 Hz or 60 Hz	30 A/m 50 Hz or 60 Hz

21. Information and description of symbols



This symbol indicates that the products described in these user instructions adhere to the requirements for an applied part of Type B of EN 60601-1. The surface of the hearing aid is specified as an applied part of Type B.



Indicates the date when the medical device was manufactured.



This symbol indicates that it is important for the user to read and take into account the relevant information in these user guides.



This symbol indicates that it is important for the user to pay attention to the relevant warning notices in these user guides.



This symbol indicates that it is important for the user to pay attention to the relevant warning notices related to batteries in these user guides.



Important information for handling and product safety.



This symbol confers that the electromagnetic interference from the device is under limits approved by the US Federal Communications Commission.



The **Bluetooth**[®] word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by the manufacturer is under license. Other trademarks and trade names are those of their respective owners.



Indicates the manufacturer's serial number so that a specific medical device can be identified.



Indicates the manufacturer's catalogue number so that the medical device can be identified.



This symbol indicates that electronic instructions for use are available.



IP Rating = Ingress Protection Rating. IP68 rating indicates that the hearing aid is water and dust resistant. It survived continuous immersion in 1 meter of water for 60 minutes and 8 hours in a dust chamber as per the IEC60529 standard.

IP22

IP Rating = Ingress Protection Rating. IP22 rating indicates that the device is protected against solid foreign objects of 12,5 mm diameter and greater and is additionally protected against vertically falling water drops when enclosure tilted up to 15°.



Indicates the temperature limits to which the medical device can be safely exposed.



Indicates the range of humidity to which the medical device can be safely exposed.



Indicates the range of atmospheric pressure to which the medical device can be safely exposed.



During transportation keep dry.



This symbol indicates a device that is not safe to use in magnetic resonance (MR) environment (e.g. during an MRI scan).



This symbol indicates that this is a Prescription Device. CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician or hearing care professional.

22. Troubleshooting

Cause	What to do
Problem: Hearing aid not functioning	
Speaker/earpiece blocked	Clean speaker opening/earpiece
Hearing aid switched off	Press lower part of multi-function button for 3 seconds (chapter 8)
Battery fully discharged	Charge hearing aid (chapter 4)
Hearing aid in error state	Reboot the hearing aid (chapter 15)
Problem: Hearing aid whistles	
Hearing aid not inserted correctly into ear	Insert hearing aid correctly (chapter 6)
Cerumen in ear canal	Contact your ENT/GP or hearing care professional.
Problem: Hearing aid too loud	
Volume too loud	Decrease volume if volume control available (chapter 8)

Cause	What to do
Problem: Hearing aid not loud enough or distorted	
Volume too low	Increase volume if volume control available (chapter 8)
Low battery	Charge hearing aid (chapter 4)
Speaker/earpiece blocked	Clean speaker opening/earpiece
Hearing has changed	Contact your hearing care professional
Problem: Hearing aid plays two beeps	
Indication for low battery	Charge hearing aid (chapter 4)
Problem: Hearing aid will not switch on	
Battery fully discharged	Charge hearing aid (chapter 4)

Cause	What to do
Problem: Indicator light on hearing aid does not light up when hearing aid placed in charging slots	
Hearing aid not inserted correctly into charger	Insert hearing aid correctly into charger (chapter 4)
Hearing aid battery fully discharged	Wait three hours after placing hearing aid in charger, regardless of LED behavior
Problem: Indicator light on hearing aid is solid red while hearing aid is in the charger	
Hearing aid is out of its operating temperature range	Warm hearing aid up or cool it down. Operating temperature range is +5° to +40° Celsius (+41° to +104° Fahrenheit)
Faulty battery	Contact your hearing care professional

Cause	What to do
Problem: Indicator light on hearing aid switches off when hearing aid is removed from charger	
"Auto On" feature is disabled	Switch on the hearing aid (chapter 9)
Problem: Indicator light on hearing aid remains solid green after removal from charger	
Indicator light on hearing aid was red when hearing aid was placed into the charger	Reboot the hearing aid (chapter 15)
Problem: Battery does not last the whole day	
Shortened battery life	Contact your hearing care professional. A battery replacement may be required.

Cause	What to do
Problem: Phone call function does not work	
Hearing aid is in flight mode	Switch hearing aid off and then on again (chapter 14)
Hearing aid not paired to the phone	Pair hearing aid to the phone (chapter 12)

Problem: Charging does not start (but indicator light around the charger USB port lights up green when the charger is connected to the power outlet.)	
Charger not connected to power source.	Connect charger to external power supply.
Hearing aid not inserted correctly into charger	Insert the hearing aid correctly into charger (chapter 4.1)

Cause	What to do
Problem: Charging of the hearing aid does not start	
Hearing aids not inserted correctly into charger.	Insert the hearing aids correctly into the charger (chapter 4).
Internal battery of the charger discharged.	Connect the charger to the power supply (chapter 3).

❗ If the problem persists, contact your hearing care professional for assistance.

23. Important safety information

Please read the relevant safety information and limitations-of-use information on the following pages before using your hearing aid.

Intended use

Hearing aid: The hearing aid is intended to amplify and transmit sound to the ear and thereby compensate for impaired hearing.

The Tinnitus software feature is intended for use by people with tinnitus who also desire amplification. It provides supplementary noise stimulation which can help defocus the user's attention from their tinnitus.

Charger: The chargers are intended to charge the battery of the rechargeable hearing aid.

Intended patient population

Hearing aid: This device is intended for patients with unilateral and bilateral, mild to profound hearing loss from 8 years of age. The Tinnitus software feature is intended for patients with chronic tinnitus from 18 years of age.

Charger: The intended patient population is the user of the compatible hearing aids.

Intended user:

Hearing aids: Intended for people with hearing loss using a hearing aid and their caregivers. A hearing care professional is responsible for adjusting the hearing aid.

Charger: Intended for people with hearing loss using the compatible rechargeable hearing aid, their caregivers and hearing care professional.

Indications:

Clinical indications for the use of hearing aids and Tinnitus software feature are:

Presence of organic hearing loss, presenting each of the following characteristics:

- Uni- or bilateral
- Conductive, sensorineural or mixed
- Mild to profound

Presence of a chronic tinnitus (> 3 months, after occurrence) in addition to the presence of a hearing loss.

Charger: Indications are not derived from charger but from the compatible hearing aids.

Contraindications:

Clinical contra-indications for the use of hearing aids and Tinnitus software feature are:

- Acute tinnitus (< three months after it occurs)
- Deformity of the ear (i.e. closed ear canal; absence of the auricle)
- Neural hearing loss (i.e. retro-cochlear pathologies such as absent / non-viable auditory nerve)

Charger: Contraindications are not derived from the charging accessories, but from the compatible hearing aids.

The primary criteria for the referral of a patient for a medical or other specialist opinion and / or treatment are as follows:

- Visible congenital or traumatic deformity of the ear;
- History of active drainage from the ear in the previous 90 days;
- History of sudden or rapidly progressive hearing loss in one or both ears within the previous 90 days;
- Acute or chronic dizziness;
- Audiometric air-bone gap equal to or greater than 15 dB at 500 Hz, 1000 Hz and 2000 Hz;
- Visible evidence of significant cerumen accumulation or a foreign body in the ear canal;
- Pain or discomfort in the ear;
- Abnormal appearance of the eardrum and ear canal such as:
 - Inflammation of the external auditory canal
 - Perforated eardrum
 - Other abnormalities which the hearing care professional believes are of medical concern

Special care should be exercised in selecting and fitting a hearing aid whose maximum sound pressure level exceeds 132 decibels (dB) because there may be risk of impairing the remaining hearing of the hearing aid user. (This provision is required only for those hearing aids with a maximum sound pressure capability greater than 132 dB).

The hearing care professional may decide that referral is not appropriate or in the best interests of the patient when the following applies:

- When there is sufficient evidence that the condition has been fully investigated by a medical specialist and any possible treatment has been provided;
- The condition has not worsened or changed significantly since the previous investigation and / or treatment.
- If the patient has given their informed and competent decision not to accept advice to seek a medical opinion, it is permissible to proceed to recommend appropriate hearing aid systems subject to the following considerations:

- The recommendation will not have any adverse effects on the patient's health or general well being;
- The records confirm that all necessary considerations about the patient's best interests have been made

If legally required, the patient has signed a disclaimer to confirm that the referral advice has not been accepted and that it is an informed decision.

Clinical benefit:

Hearing aids: Improvement of speech understanding

Tinnitus feature: Tinnitus software feature provides supplementary noise stimulation which can help defocus the user's attention from their tinnitus.

Charger: The charger itself does not provide a direct clinical benefit. An indirect clinical benefit is provided by enabling the parent device (rechargeable hearing aids) to achieve their intended purpose.

Side effects:

Physiological side-effects of hearing aids like tinnitus, dizziness, wax build up, too much pressure, sweating or moisture, blisters, itching and/or rashes, plugged or fullness and their consequences like headache and/ or ear pain, may be resolved or reduced by your hearing care professional.

Charger: Side effects are not derived from the charger, but from the compatible hearing aids.

Conventional hearing aids have the potential to expose patients to higher levels of sound exposure, which might result in threshold shifts in the frequency range affected by acoustic trauma.

The hearing aid is suitable for the home healthcare environment and due to their portability it may happen that it is used in professional healthcare facility environment like physician offices, dental offices etc.

A hearing aid will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic

conditions. Infrequent use of a hearing aid does not permit a user to attain full benefit from it. The use of a hearing aid is only part of hearing habilitation and may need to be supplemented by auditory training and instruction in lipreading.

Feedback, poor sound quality, too loud or too soft sounds, inadequate fitting or problems when chewing or swallowing, can be resolved or improved during the fine-tuning process at the fitting procedure by your hearing care professional.

To report an unexpected operation or event, please contact the manufacturer or a representative.

23.1 Warnings

⚠ The majority of hearing aid users have a hearing loss for which a deterioration in hearing is not to be expected with regular use of a hearing aid in everyday situations. Only a small group of hearing aid wearers with hearing loss, may be at risk of deterioration in hearing after a long period of use.

⚠ The hearing aids (specially programmed for each hearing loss) must only be used by the intended person. They should not be used by any other person as they could damage hearing.

⚠ Special medical or dental examination including radiation described below, may adversely affect the correct functioning of your hearing aids. Remove and keep them outside the examination room/area before undergoing:

- Medical or dental examination with X-ray (also CT scan)
 - Medical examinations with MRI/NMRI scans, generating magnetic fields
- Hearing aids don't need to be removed when passing security gates (airports etc.). If X-ray is used at all, it will be


in very low doses, and will not affect the hearing aids.


⚠ Patients implanted with magnetic externally programmable CSF shunt valves may have a risk of experiencing an unintended change in their valve setting when exposed to strong magnetic fields. The receiver (loudspeaker) of hearing aids, the Lyric SoundLync, the Lyric programming wand and the MiniControl contain static magnets. Keep a distance of 2 inches (approx. 5 cm) or more between the magnets and


the site of the implanted shunt valve.

⚠ Hearing programs in the directional microphone mode reduce background noises. Please be aware that warning signals or noises coming from behind, e. g. cars, are partially or entirely suppressed.


⚠ Do not place the charging accessories close to an induction cooking surface. Conductive structures inside the charging accessories could absorb inductive energy resulting in thermal destruction.


 Always make sure that the hearing aids and charger are dry and clean. If the charger or power supply is damaged, the device must not be used anymore as it may cause electric shock.

 Do not touch the contacts of the charger when powered.


 Hearing aids and their parts should be kept out of reach of children and anyone who might swallow, choke on or otherwise cause injury to themselves with them. If swallowed consult a physician or hospital immediately as the hearing aid or its parts can be


poisonous. In the event of choking immediately administer treatment or contact emergency services.


 The usage of this device by children and individuals with cognitive impairment should be supervised at all times to ensure their safety


 **WARNING:** This device uses a lithium button/coin battery. These batteries are hazardous and can cause severe or fatal injuries in 2 hours or less if they are swallowed or placed inside any part of the body, whether the

battery is used or new! Keep out of the reach of children, individuals with cognitive impairment, or pets. If you suspect that a battery was swallowed or placed inside any part of the body, immediately consult your physician without any delay!

 Protect your hearing aid and charging accessory from heat and sunlight (never leave near a window or in the car). Never use a microwave or other heating devices to dry your hearing aid (due to risk of fire or explosion). Ask your hearing care professional about suitable drying methods.

 Only authorized personnel are allowed to open the charging accessory due to possible electrical hazard.

 Cables and cords wrapped around the neck may cause strangulation. Do not leave the device and its components unattended with children, individuals with cognitive impairment or pets.

 Do not remove receiver cables from the hearing instruments. If removal or exchange is required it is recommended to contact the hearing care professional.

⚠ Changes or modifications to the hearing aid or charging accessory that were not explicitly approved by the manufacturer are not permitted. Such changes may damage your ear or the devices.

⚠ Do not use the device in explosive areas (mines or industrial areas with danger of explosions, oxygen-rich environments or areas where flammable anesthetics are handled). The device is not ATEX certified.

⚠ Hearing aids should not be fitted with domes / wax

protection systems when used by clients with perforated eardrums, inflamed ear canals or otherwise exposed middle ear cavities. In these cases, we recommend the use of a classic earmold. In the unlikely case that any part of this product should remain in the ear canal, it is strongly recommended to see a physician for safe removal.

⚠ Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is

necessary, this equipment and the other equipment should be observed to verify that they are operating normally.


⚠ The hearing aids have to be dry before charging. Otherwise the charging reliability cannot be guaranteed.

⚠ Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the hearing aid or accessory, including cables specified by the manufacturer.

Otherwise, degradation of the performance of this equipment could result.


⚠ Your device operates between 2.4 GHz and 2.48 GHz frequency range. When flying please check if flight operator requires devices to be switched into flight mode, see corresponding chapter regarding flight mode in this user guide.


⚠ Do not drop your hearing aid or charging accessory! Dropping onto a hard surface can damage your hearing aid or charging accessory.


 The following is only applicable for persons with active implantable medical devices (i.e. pacemakers, defibrillators, etc.)

- Keep the wireless device at least 15 cm (6 inches) away from the active implant. If you experience any interference do not use the wireless hearing aids and contact the manufacturer of the active implant. Please, note that the interference can also be caused by powerlines, electrostatic discharge, airport metal detectors etc.
- Keep magnets (e.g. battery handling tool, EasyPhone magnet, etc.) at least 15 cm


(6 inches) away from the active implant.

 Do not use your hearing aid or charger in areas where electronic equipment is prohibited.


 In very rare cases, the dome can remain in your ear canal when removing the hearing tube from the ear. In the unlikely case that the dome does get stuck in your ear canal, it is strongly recommended to see a physician for safe removal.


 These hearing aids are water resistant. They are designed to withstand normal


activities and occasional accidental exposure to extreme conditions. These hearing aids are not specifically designed for extended periods of water submersion on a continual basis, that is worn in activities such as swimming or bathing. Always remove your hearing aid before these activities, as the hearing aid contains sensitive electronic parts.


 Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic

emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

 Charger and power supply must be protected from shocks. If the charger or power supply is damaged after a shock, the devices must not be used anymore.


 Ship the rechargeable devices according to local regulations as the devices contain lithium-ion batteries. For shipping, they are classified as dangerous goods. If in doubt ask the courier responsible for shipping how to ship the devices properly.


 To supply power to the charging accessories, use only IEC 60950-1*, IEC 62368-1 or IEC60601-1 certified charging equipment (e.g. power supply) within the specification in Chapter 2 Parts of the hearing aid and charger.


 The charger contains Lithium-Ion rechargeable battery that might swell and deform the housing of the charger. If you notice any swelling or deformation in the charger, stop using it immediately. Swollen batteries can be a serious safety hazard, posing risks

such as leakage, overheating, or even explosion. Do not attempt to remove, charge, puncture or modify the swollen battery. Do not ignore signs of battery damage. Using the charger with a damaged battery can affect the charger performance and permanently damage the charger. Avoid handling the charger with bare hands if it shows signs of damage or leakage. Avoid direct contact with the leaked electrolyte. If contact occurs, rinse with water and seek medical attention if needed. If you have any questions or concerns, seek professional guidance at your hearing

care professional before using the charger. Dispose of the charger at waste disposal sites intended for electronic waste, or give your charger to your hearing care professional for disposal.

 Do not expose your device to temperatures outside the operation conditions indicated in the User Guide (see chapter 16). They contain lithium-ion batteries, which might swell, leak or explode upon exposure to temperatures outside those conditions.

 Only use approved chargers described in this user guide for charging your hearing aids, otherwise the devices may be damaged.

 Do not place these rechargeable devices in your checked-in luggage when traveling by plane as they contain lithium-ion batteries. The devices should be placed in your carry-on/ hand luggage.

23.2 Cautions

- ⚠ If you feel pain in or behind your ear, if it is inflamed or if skin irritation and accelerated accumulations of earwax occur, please check with your hearing care professional or physician.
- ⚠ Titanium shells of custom hearing aids and earpieces can contain small quantities of acrylates from adhesives. People who are sensitive to such substances may potentially experience an allergic skin reaction. Please stop using the device immediately if such a reaction occurs and check with your hearing care professional or physician.
- ⚠ Only use the device according to chapter 16 “Environmental conditions”. Using the device otherwise may heat the hearing aid up to temperatures, which in the worst-case can lead to skin burns.
- ⚠ Avoid strong physical impacts to the ear when wearing a hearing aid with customized ear piece. The stability of a customized ear piece is designed for normal use. A strong physical impact to the ear (e.g.

during sports) may cause the customized ear piece to break. This may lead to perforation of the ear canal or ear drum.

- ⚠ Make sure that you always completely dry your hearing aid after use. Store the hearing aid in a safe, dry and clean place.
- ⚠ After mechanical stress or shock to the hearing aid, please ensure that the hearing aid shell is intact before placing it in the ear.

23.3 Notes

- ① If you are not using your hearing aid for a long period of time, store it in a box with a drying capsule or in a well ventilated location. This allows moisture to evaporate from your hearing aid and prevents a potential impact on its performance.
- ① The dome should be changed every three months or when it becomes stiff or brittle. This is to prevent the dome from detaching from the tube spout during insertion into or removal from the ear.

❶ Never wash the microphone inputs. Doing so could cause it to lose its special acoustic features.

❶ Always keep the USB socket of the device and the USB plug of the cable clean and dry. Foreign particles or moisture in the USB connection can lead to an electrical fault, which can heat up the USB connection and, in the worst case, cause it to melt. Use the provided cable to supply power to the device. If you notice the indicator light around the USB port of the device went

off during charging, check the USB connection for proper conditions.

❶ The built-in battery of the rechargeable charger must be recharged every 3 months. This will prevent a deep discharge of the built-in battery. Deep discharged batteries may be damaged if recharged and could catch fire.

Important information: Cell phones

Some hearing aid users have reported a buzzing sound in their hearing aids when they are using cell phones. According to the ANSI 63.19 (American National Standard Methods of Measurement of Compatibility between Wireless Communications Devices and Hearing Instruments) standard, the compatibility of a particular hearing aid and cell phone can be predicted by adding the rating for the hearing aid immunity to the rating for the cell phone emissions. The sum of the hearing aid rating (e.g. M2/T2=2) and the telephone rating (e.g. M3/T3=3) is 5, and any combination that equals 5 will provide "normal use"; a sum of 6 or greater would indicate "excellent performance". The equipment performance measurements, categories and system classifications are based upon the best information available but cannot guarantee that all users will be satisfied. The immunity of this hearing aid is at least M2/T2.

Note: the performance of individual hearing aids may vary with individual cell phones. Therefore, please try the hearing aid with your cell phone or, if you are purchasing a new phone, be sure to try it with your hearing aid prior to purchase. For additional guidance, please ask your hearing care professional for the booklet entitled “hearing aid compatibility with digital wireless cell phones”.

Important information in case of battery ingestion

If a battery is swallowed, call the 24-hour National Battery Ingestion Hotline at 800-498-8666 or consult your physician immediately.

Important notice for prospective noise generator users

The Tinnitus Balance noise generator is a broadband sound Generator which may have been enabled in your hearing aid. It provides a means of sound enrichment that can be used as part of a personalized tinnitus management program to provide temporary relief from tinnitus. It should always be used as prescribed by your audiologist. The underlying principle of sound enrichment is to provide supplementary noise stimulation which can help defocus your attention from your tinnitus and avoid negative reactions. Sound enrichment, coupled with instructional counseling, is an established approach to managing tinnitus.

Good health practice requires that a person reporting tinnitus have a medical evaluation by a licensed ear physician before using a sound generator. The purpose of such an evaluation is to ensure medically treatable conditions that may be causing tinnitus are identified and treated prior to using a sound generator.

Should you develop any side effects whilst using the noise generator such as headaches, nausea, dizziness or heart palpitations or experience a decrease in auditory function, you should discontinue use and seek a medical evaluation.

According to OSHA (Occupational Safety & Health Administration, U.S. Department of Labor) regulations, the volume of the noise generator can be set to a level which could lead to permanent hearing damage when used for a prolonged period of time. Should the noise generator be set to such a level in your hearing aid, your audiologist will advise you of the maximum amount of time per day you should use the noise generator. The noise generator should never be used at uncomfortable levels.

Your Miracle-Ear® Representative:



Manufactured for Miracle-Ear, Inc.
150 South Fifth Street, Suite 2300
Minneapolis, MN 55402



Type: PDL	Status: INWORKS
Document No.: PDL-26205	Effective Date: 17-Mar-2025
Rev.: 1	
Title: 029-1442-XX User Guide MEMINI E 5P R-R AI PH	
Process: Innovation, Design & Development	
Owner: 361SSOCKO Sylwia Soćko	

Attributes

<u>Attribute Type</u>	<u>Value</u>	<u>Description</u>
Project	Winehouse Charger Go Amplifon and Miracle Ear DEVELOPMENT	

Approvals

<u>Level</u>	<u>Actor</u>	<u>Job Title</u>	<u>Sign-off Date</u>	<u>Sign-off By</u>
1	Serafina D Alessi	Head of Design QA		
2	Fotini Pachidou	Senior Launch Product Manager		
2	Gabriel Perez	Senior Project Manager		
2	Patrick Bertschy	Product Manager		
2	Daniele Munari	Regulatory Clinical Affairs Manager		
2	David Sooprayen	Head of Regulatory Affairs		
2	Jeremias Perren	Senior Manager Hardware Design		

Revision Notes

<u>Access Activity</u>	<u>Note</u>	<u>Accessed By</u>	<u>Accessed Date</u>
Remark	ME version of Phonak charger UG: PDL-22021 and Phonak HD UG: PDL-19427. Audiology role not required, because there's no modification of content in comparison to the base project. CHANGE-PLN-1100-01931.	361SSOCKO	19-May-2025
Remark	A quick update on the back cover - code added (HE300647)	361SSOCKO	21-May-2025