MEENERGY™ B P 5/4/3/2

with GENIUS™ 4.0 Technology User Manual

www.miracle-ear.com

Made for

 **iPhone | iPad | iPod**

Content

Welcome 4

Your hearing aids 5

Hearing aid type	5
Getting to know your hearing aids	6
Components and names	6
Controls	8
Hearing programs	10
Features	10

Daily use 11

Charging	11
Turning on and off	12
Switching to standby mode	13
Inserting and removing the hearing aids	14
Adjusting the volume	17
Changing the hearing program	18
Further adjustments (optional)	18

Special listening situations 19

On the phone	19
Audio streaming with iPhone	19
Airplane mode for hearing aids	20
Audio induction loops	20

Maintenance and care 22

Hearing aids	22
Ear pieces and tubes	24
Professional maintenance	27

Important safety information 28

Further information 31

Safety information	31
Accessories	31
Operating, transport, and storage conditions	31
Disposal information	33
Symbols used in this document	33
Troubleshooting	34
Country-specific information	35
Service and warranty	37

Welcome

Thank you for choosing Miracle-Ear® hearing aids to accompany you through everyday life. Like anything new, it may take you a little time to become familiar with them.

This guide, along with support from your Hearing Care Specialist, will help you understand the advantages and greater quality of life your hearing aids offer.

To receive the most benefit from your hearing aids, it is recommended that you wear them every day, all day. This will help you get adjusted to them.

CAUTION

It is important to read this user manual and the safety manual thoroughly and completely. Follow the safety information to avoid damage or injury.

Before using your hearing aids for the first time, fully charge them. Follow the instructions in the charger's user manual.

Your hearing aids

This user manual describes optional features that your hearing aids may or may not have. Ask your Hearing Care Specialist to indicate the features that apply to your hearing aids.

Hearing aid type

Your hearing aids are BTE (Behind-The-Ear) models. A tube delivers the sound from the hearing aid into the ear. The hearing aids are not intended for children under the age of 3 years or persons with a developmental age of under 3 years.

A power cell (lithium-ion rechargeable battery) is permanently built into your hearing aid. This enables easy charging with our standard Charger or our optional Dry & Clean Charger.

The wireless functionality enables advanced audiological features and synchronization between your two hearing aids.

Your hearing aids feature Bluetooth® low energy* technology allowing for easy data exchange with your smartphone, and for seamless audio streaming with your iPhone.**

*The Bluetooth word mark and logos are owned by the Bluetooth SIG, Inc., and any use of such marks by the legal manufacturer of this product is under licenses. Other trademarks and trade names are those of their respective owners.

**iPad, iPhone, and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries.

Getting to know your hearing aids

It is recommended to familiarize yourself with your new hearing aids. With the hearing aids in your hand try using the controls and note their location on the hearing aid. This will make it easier for you to feel and press the controls while wearing the hearing aids.

i If you have problems pressing the controls of your hearing aids while wearing them, you can ask your Hearing Care Specialist about the Miracle-Ear® App or if a remote control is available to help control your hearing aids.

Components and names

Your hearing aids are either attached to an earhook and a custom-made ear piece or to a standard tube (ThinTube) and a standard ear piece (OpenTip).

- | | |
|--|--------------------|
| ① Ear piece | ⑤ Earhook |
| ② Tube (ThinTube) | ⑥ Charging contact |
| ③ Microphone openings | ⑦ Side indicator |
| ④ Rocker switch (control, on/off switch) | |

You can use standard ear pieces together with a standard tube or custom-made ear pieces together with an earhook and tube.

You can easily exchange the standard ear pieces and clean the standard tube. Read more in section "Maintenance and care".

Standard ear pieces / tubes	Size
☐  Instant Open	
☐  Instant Tulip	
☐ ThinTube	

Custom-made ear pieces
☐ Examples:

Controls

With the rocker switch you can switch hearing programs. Your Hearing Care Specialist has programmed your desired functions to the rocker switch.

Rocker switch function	L	R
Press briefly:		
Program up/down	☐	☐
Volume up/down	☐	☐
Tinnitus therapy signal level up/down	☐	☐
TV stream on/off	☐	☐
Press for about 2 seconds:		
Program up/down	☐	☐
Volume up/down	☐	☐
Tinnitus therapy signal level up/down	☐	☐
TV stream on/off	☐	☐
Press for more than 3 seconds:		
Turn on/off	☐	☐

L = Left, R = Right

i You can also use a remote control to change hearing programs and adjust the volume of your hearing aids. With the Miracle-Ear® App you have even more control options.

Hearing programs

1
2
3
4
5
6

Read more in section "Changing the hearing program".

Features

- ☐ The tinnitus therapy feature generates a sound to distract you from your tinnitus.
- ☐ A telecoil is built into the aid, so it can connect with audio induction loops.

Read more in section "Audio induction loops".

Daily use

To assure comfortable and easy use of your hearing aids they are equipped with an onboard control element. In addition we offer the Miracel-Ear® App for Android as well as iPhones which makes handling your devices even easier. Contact your Hearing Care Specialist for downloading and installing the Miracle-Ear® App.

Charging

You have the choice between two charger options: The standard Charger for charging only or the Dry & Clean Charger.

Place your hearing aids in the charger.

- Follow the instructions in the charger's user manual. You can also find useful tips for charging in the charger's user manual.

Low power indication

You will hear an alert signal to indicate the power cell is running low. The signal will be repeated every 20 minutes. Depending on the usage of your hearing aids, you have approximately 1.5 hours to charge your hearing aids before they stop working.

Note that the power cell runtime is reduced if too many features are used simultaneously.

Turning on and off

You have the following options to turn your hearing aids on or off.

Via charger:

- ▶ Turning on: Take the hearing aids out of the charger.
The startup melody is played in your hearing aids.
The default volume and hearing program are set.
- ▶ Turning off: Place the hearing aids in the charger.

Note that the charger must be connected to the power supply. For details refer to the charger's user manual.

Via rocker switch:

- ▶ Turning on: Press and hold the lower part of the rocker switch until the startup melody starts to play. Release the rocker switch while the melody is playing.

The default volume and hearing program are set.

- ▶ Turning off: Press and hold upper or lower part of the rocker switch for several seconds. A power down melody is played.

See also the tips for charging in the charger's user manual.

Switching to standby mode

Via remote control or the Miracle-Ear® App, you can switch your hearing aids to standby mode. This mutes the hearing aids. When you leave standby mode, the previously used volume and hearing program are set.

Note:

- In standby mode, the hearing aids are not turned off completely. They draw some power.
Therefore we recommend to use standby mode only for a short period of time.

- If you want to leave standby mode, but the remote control or the Miracle-Ear® App is not at hand:
Turn your hearing aids off and on again (via rocker switch or by shortly placing them in the charger until one or more green LEDs light up). In this case, the default volume and hearing program are set.

Inserting and removing the hearing aids

Your hearing aids have been fine-tuned for your right and left ear. Colored markers indicate the side:

- Red marker = right ear
- Blue marker = left ear


Inserting a hearing aid:

- ▶ Hold the tube close to the ear piece.
- ▶ Carefully push the ear piece into the ear canal **1**.
- ▶ Twist it slightly until it sits well.

Open and close your mouth to avoid accumulation of air in the ear canal.

- ▶ Lift the hearing aid and slide it over the top of your ear **2**.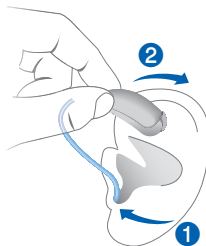

CAUTION

Risk of injury!

- ▶ Insert the ear piece carefully and not too deeply into the ear.

- It may be helpful to insert the right hearing aid with the right hand and the left hearing aid with the left hand.
- If you have problems inserting the ear piece, use the other hand to gently pull your earlobe downwards. This opens the ear canal and eases insertion of the ear piece.

The optional retention strand helps to securely retain the ear piece in your ear. To attach the optional retention strand:

- ▶ Align the end of the retention strand with the flat end of the receiver.

- ▶ Then press the retention strand towards the receiver until it clicks into place.


To position the retention strand:

- ▶ Bend the retention strand and position it carefully into the bottom of the bowl of your ear (refer to the picture).

Removing a hearing aid:

- ▶ Lift the hearing aid and slide it over the top of your ear ①.
- ▶ Hold the tube close to the ear piece and pull the ear piece out carefully ②.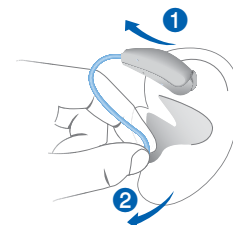

CAUTION

Risk of injury!

- ▶ In very rare cases the ear piece could remain in your ear when removing the hearing aid. If this happens, have the ear piece removed by a medical professional.

Clean and dry your hearing aids after usage. Read more in section "Maintenance and care".

Adjusting the volume

Your hearing aids automatically adjust the volume to the listening situation.

- ▶ If you prefer manual volume adjustment, press the upper part of the rocker switch to increase the volume or press the lower part to decrease the volume.

Refer to section "Controls" for the setting of your rocker switch.

An optional signal can indicate the volume change. Once the maximum or minimum volume has been reached, you may hear an optional signal tone.

Changing the hearing program

Depending on the listening situation, your hearing aids automatically adjust their sound.

Your hearing aids may also have several hearing programs which allow you to change the sound, if needed. An optional signal tone can indicate the program change.

- To change the hearing program, press the rocker switch briefly.

Refer to section "Controls" for the setting of the rocker switch. Refer to section "Hearing programs" for a list of your hearing programs.

The default volume is applied.

Further adjustments (optional)

You can also use the hearing aid controls to change other features, for example, the level of the tinnitus therapy signal.

Refer to section "Controls" for the setting of your controls.

Special listening situations

On the phone

When you are on the phone, hold the telephone receiver slightly above your ear. The hearing aid and telephone receiver have to be aligned. Turn the receiver slightly so that the ear is not completely covered.

Telephone program

You may prefer a certain volume when using the phone. Ask your Hearing Care Specialist to configure a telephone program.

- Switch to the telephone program whenever you are on the phone.

If a telephone program is configured for your hearing aids, it is listed in section "Hearing programs".

Audio streaming with iPhone

Your hearing aids are Made for iPhone hearing aids. This means you can get phone calls and listen to music from your iPhone directly into your hearing aids.

For more information about compatible iOS devices, pairing, audio streaming and other useful functions, contact your Hearing Care Specialist.

Airplane mode for hearing aids

Airplane mode is when Bluetooth® of your hearing aids is turned off. In areas where the use of Bluetooth technology is not allowed (e.g. on some airplanes), you can deactivate Bluetooth mode. This turns Bluetooth in your hearing aids off temporarily. The hearing aids still work without Bluetooth, however direct audio streaming will not be possible and other functions will not be available.

- To deactivate or activate the Bluetooth mode, use the Miracle-Ear® App.

An alert tone indicates when Bluetooth mode is deactivated or activated.

Audio induction loops

Some telephones and public places like theatres offer the audio signal (music and speech) via an audio induction loop. With this system, your hearing aids can directly receive the desired signal – without distracting noise in the environment.

You can usually recognize audio induction loop systems by certain signs.

Whenever you are in a place with an audio induction loop:

- Switch to the telecoil program.

If a telecoil program is configured for your hearing aids, it is listed in section “Hearing programs”.

Telecoil program and remote control

NOTICE

For hearing aids with wireless connectivity:
When the telecoil program is active, the remote control may cause a pulsing noise.

- Use the remote control with a distance greater than 10 cm (4 inches).

Maintenance and care

To prevent damage it is important that you take care of your hearing aids and follow a few basic rules, which will soon become a part of your daily routine.

Hearing aids

Cleaning

Your hearing aids have a protective coating. However, if not cleaned regularly you may damage the hearing aids or cause personal injury.

- ▶ Clean your hearing aids daily with a soft, dry tissue.
- ▶ Never use running water or immerse the devices in water.
- ▶ Never apply pressure while cleaning.
- ▶ Do not use alcoholic solvent on the charging contacts.
- ▶ To avoid corrosion, use a soft and dry tissue to wipe off sweat or liquid from the charging contacts or hearing aids before inserting them in the charger.
- ▶ Ask your Hearing Care Specialist for recommended cleaning products, special care sets, or more information on how to keep your hearing aids in good condition.

Drying

Ask your Hearing Care Specialist for recommended drying products and for individual instructions on when to dry your hearing aids.

Storage

- Short-term storage (up to several days): Turn off your hearing aids via the rocker switch or place them in the powered charger.

The charger must be connected to the power supply. If the charger is not connected to power supply and you place your hearing aids in the charger, the hearing aids are not turned off.

Note that when you turn off the hearing aids via a remote control or the Miracle-Ear® App, the hearing aids are not turned off completely. They are in standby-mode and still draw some power.

- Long-term storage (weeks, months, ...): First, fully charge your hearing aids. Turn them off via the rocker switch before storing them. We recommend using drying products while storing the hearing aids.

At 6 month intervals you will have to charge the hearing aids to avoid an unrecoverable deep-discharge of the power cell. Deep-discharged power cells cannot be charged anymore and they have to be replaced. We recommend recharging more frequently than 6 month intervals.

Observe the storage conditions in section "Operating, transport and storage conditions".

Ear pieces and tubes

Cerumen (ear wax) may accumulate on the ear pieces. This may affect sound quality.

- For hearing aids with a standard tube (ThinTube):

Clean the ear pieces daily, clean the tubes as required.

Exchange the ear pieces and tubes approximately every three to six months, or sooner if you notice cracks or other changes.

- For hearing aids with earhook:

Clean the ear pieces daily.


Ask your Hearing Care Specialist to clean the tubes or to exchange the ear pieces and tubes as needed.

Cleaning ear pieces

- Clean the ear piece upon removal with a soft and dry tissue.

This prevents cerumen from becoming dry and hardening.

- Never use running water or immerse the ear piece in water.

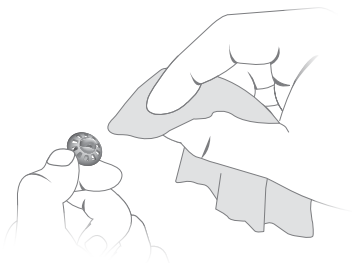The optional Dry&Clean Charger uses UV light for cleaning and dries tubes and ear pieces while charging the hearing aids. Ask your Hearing Care Specialist for more information on the best solution for you.

Exchanging standard ear pieces (OpenTip)

- Pull off the used ear piece and plug on the new ear piece.

Exchanging standard tubes

- Unscrew the used tube from the hearing aid.

- Screw on the new tube.

Professional maintenance

Your Hearing Care Specialist can perform a thorough professional cleaning and maintenance.

Custom-made ear pieces and wax filters should be exchanged as needed by your Hearing Care Specialist.

Ask your Hearing Care Specialist for recommended maintenance intervals and support.

Important safety information

This section contains important safety information regarding the power cell. For more safety information, refer to the safety manual supplied with the hearing aids.

WARNING

A lithium-ion rechargeable battery is permanently built into your device. Incorrect use of the device can cause the lithium-ion rechargeable battery to burst.

Risk of injury, fire or explosion!

- ▶ Follow the safety instructions for lithium-ion rechargeable battery in this section.

WARNING

Risk of electric shock!

- ▶ Connect the charger only with a USB power supply which complies with IEC 60950-1 and/or IEC62368-1.

WARNING

- ▶ Observe the operating conditions.
- ▶ Pay special attention to protect your device from extreme heat (e.g. fire, microwave oven, induction oven or other high induction fields) and sunlight.
- ▶ Charge your hearing aids only with an approved charger. Ask your Hearing Care Specialist for support.
- ▶ Do not disassemble the device.
- ▶ Do not use the device in explosive atmospheres.
- ▶ Do not use devices that are deformed or obviously damaged.
- ▶ Do not use the device when its performance after charging reduces considerably. Return the device to your Hearing Care Specialist.
- ▶ Do not dispose of the device in fire or water.

WARNING

- Replacement and repair have to be performed by a professional. Therefore:
- Do not repair or replace the lithium-ion rechargeable battery yourself.
- In the unlikely event of a lithium-ion rechargeable battery bursting or exploding, keep all combustible materials away.

In any of the above cases, return your devices to the Hearing Care Specialist.

Further information

Safety information

For more safety information, refer to the safety manual supplied with the device.

Accessories

You can use the Miracle-Ear® App for convenient remote control of your hearing aids. The app also allows you to deactivate signal tones, to create individual sound profiles and many more functions.

You can also use a remote control.

Consult your Hearing Care Specialist for further information.

Operating, transport, and storage conditions

The hearing aids operate in the following environmental conditions (also valid in between uses):

Operating conditions	Discharging	Charging
Temperature	0 to 50 °C (32 to 122 °F)	10 to 35 °C (50 to 95 °F)
Relative humidity	5 to 93 %	20 to 75 %
Atmospheric pressure	700 to 1060 hPa	700 to 1060 hPa

During extended periods of transport and storage, please observe the following conditions:

	Storage	Transport
Temperature (recommended)	10 to 25 °C (50 to 77 °F)	-20 to 60 °C (-4 to 140 °F)
Temperature (maximum range)	10 to 40 °C (50 to 104 °F)	-20 to 60 °C (-4 to 140 °F)
Relative humidity (recommended)	20 to 80 %	5 to 90 %
Relative humidity (maximum range)	10 to 80 %	5 to 90 %
Atmospheric pressure	700 to 1060 hPa	700 to 1060 hPa

Note that at a minimum you have to charge the hearing aids every 6 months. We recommend you charge your hearing aids at least every 3 months. For details refer to section "Maintenance and care".

For other parts, such as the charger, other conditions may apply.

WARNING

Risk of injury, fire or explosion during shipment.

- Ship lithium batteries and hearing aids according to local regulations.

Disposal information

- To avoid environmental pollution, do not throw power cells or hearing aids into household trash.
- Recycle or dispose of power cells or hearing aids according to local regulations.

Symbols used in this document


Points out a situation that could lead to serious, moderate, or minor injuries.

Indicates possible property damage.

Advice and tips on how to handle your device better.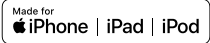

"Made for iPhone", "Made for iPad", and "Made for iPod" mean that an electronic accessory has been designed to connect specifically to iPhone, iPad, or iPod, respectively, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPhone, iPad, or iPod may affect wireless performance.

Troubleshooting

Problem and possible solutions

Sound is weak.

- Increase the volume.
 - Charge the hearing aid.
 - Clean or exchange the ear piece.
-

Hearing aid emits whistling sounds.

- Reinsert the ear piece until it fits securely.
 - Decrease the volume.
 - Clean or exchange the ear piece.
-

Sound is distorted.

- Decrease the volume.
 - Charge the hearing aid.
 - Clean or exchange the ear piece.
-

Hearing aid emits signal tones.

- Charge the hearing aid.
-

Hearing aid does not work.

- Turn the hearing aid on.
 - Charge the hearing aid. Observe also the status LEDs on the charger.
-

Consult your Hearing Care Specialist if you encounter further problems.

Country-specific information

USA and Canada

Certification information for hearing aids with wireless functionality:

MEENERGY™ B P 5/4/3/2

- HVIN: RFM002
- Contains FCC ID: SGI-RFM002
- Contains IC: 267AB-RFM002

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Copyright ©2021 Miracle-Ear, Inc.
Distributed By Miracle-Ear, Inc.
Derechos de autor ©2021 Miracle-Ear, Inc.