MEMINI™ RF 10

with GENIUS™ 4.0 Technology User Manual

Content

Welcome 4

Your hearing aids 5

Instrument type	5
Getting to know your hearing aids	5
Components and names	6
Hearing programs	7
Features	7

Batteries 8

Battery size and handling tips	8
Replacing batteries	9

Daily use 10

Turning on and off	10
Inserting and removing the hearing aids	11
Adjusting the volume	14
Changing the hearing program	14

Special listening situations 15

On the phone	15
--------------	----

Maintenance and care 16

Hearing aids	16
Ear pieces	17
Professional maintenance	20

Further information 21

Safety information	21
Accessories	21
Symbols used in this document	21
Troubleshooting	22
Country-specific information	23
Service and warranty	27

Welcome

Thank you for choosing Miracle-Ear® hearing aids to accompany you through everyday life. Like anything new, it may take you a little time to become familiar with them.

This guide, along with support from your Hearing Care Specialist, will help you understand the advantages and greater quality of life your hearing aids offer.

To receive the most benefit from your hearing aids, it is recommended that you wear them every day, all day. This will help you get adjusted to them.

CAUTION

It is important to read this user guide and the safety manual thoroughly and completely. Follow the safety information to avoid damage or injury.

Your hearing aids

This user guide describes optional features that your hearing aids may or may not have. Ask your Hearing Care Specialist to indicate the features that are applicable to your hearing aids.

Instrument type

Your hearing aids are worn directly in the ear canal. They are not intended for children under the age of 3 years or persons with a developmental age of under 3 years.

The wireless functionality enables advanced audiological features and synchronization between your two hearing aids.


Getting to know your hearing aids

Your hearing aids have no controls.

You can ask your Hearing Care Specialist whether a smartphone app to control your hearing aids is available.

Components and names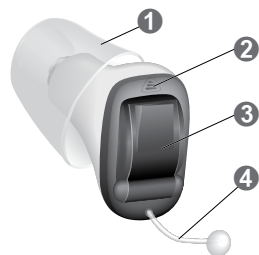

- ❶ Ear piece (Click Sleeve)
- ❷ Microphone
- ❸ Battery compartment (on/off switch)
- ❹ Removal cord

You can use the following standard ear pieces:

Standard ear pieces		Size
☐	Click Sleeve (vented or closed)	
☐	Click Dome™ single (open or closed)	
☐	Click Dome semi-open	
☐	Click Dome double	

You can easily exchange the standard ear pieces. Read more in section “Maintenance and care”.

Hearing programs

1
2
3
4
5
6

Read more in section “Changing the hearing program”.

Features

- ☐ Power-on delay enables whistle-free insertion of hearing aids.
Read more in section “Turning on and off”.
- ☐ The tinnitus control feature generates a sound to distract you from your tinnitus.

Batteries

When the battery is low the sound becomes weaker or you will hear an alert signal. The battery type will determine how long you have to replace the battery.

Battery size and handling tips

Ask your Hearing Care Specialist for recommended batteries.


Battery size:	10
---------------	----

- Always use the correct battery size for your hearing aids.
- Remove the batteries if you do not intend to use the hearing aids for several days.
- Always carry spare batteries.
- Remove empty batteries immediately and follow your local recycling rules for battery disposal.

Replacing batteries


Removing the battery:

- ▶ Open the battery compartment.
- ▶ Remove the battery by hand.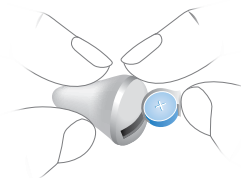

Inserting the battery:

- ▶ If the battery has a protective film, remove it only when you are ready to use the battery.
- ▶ Insert the battery with the "+" symbol facing upwards.

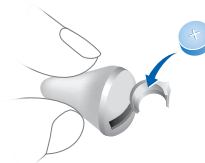- ▶ Carefully close the battery compartment. If you feel resistance, the battery is not inserted correctly. Do not attempt to close the battery compartment by force. It can be damaged.

Daily use

Turning on and off

You have the following options to turn your hearing aids on or off.

Via battery compartment:

- ▶ Turning on: Close the battery compartment.
The default volume and hearing program are set.
- ▶ Turning off: Open the battery compartment fully.

When wearing the hearing aids, an alert tone can indicate when an instrument is being turned on or off.

When the power-on delay is activated, the hearing aids turn on after a delay of several seconds. During this time you can insert the hearing aids into your ears without experiencing unpleasant feedback whistling.

The "power-on delay" can be activated by your Hearing Care Specialist.

Inserting and removing the hearing aids


CAUTION

Risk of injury!

- ▶ Always wear the hearing aid with an ear piece.
- ▶ Make sure that the ear piece is completely attached.


Inserting a hearing aid:

- ▶ Identify right and left hearing aid so that you insert it in the correct ear. The labeling or the color of the hearing aid indicates the side:
Red color, imprinted "R" = right ear
Blue color, imprinted "L" = left ear
- ▶ Take care that the flap of the Click Sleeve matches the line printed on the hearing aid.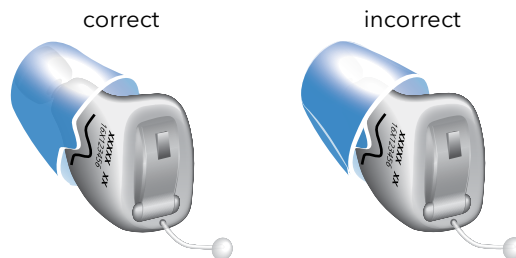


- ▶ Hold the hearing aid correctly before inserting it: Pay

special attention to the removal cord indicating the bottom. The arrow which is printed on the hearing aid has to point upwards.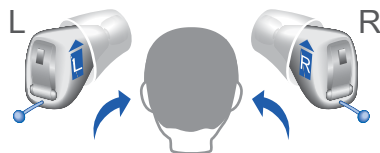

- ▶ Carefully push the hearing aid into the ear canal.
- ▶ Twist it slightly until it sits well.

Open and close your mouth to avoid accumulation of air in the ear canal.

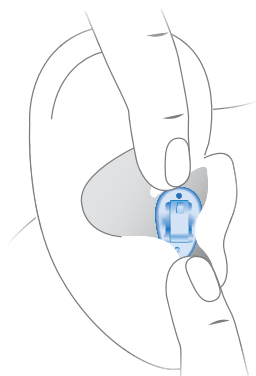CAUTION

Risk of injury!

- ▶ Insert the hearing aid carefully and not too deeply into the ear.

- It may be helpful to insert the right hearing aid with the right hand and the left hearing aid with the left hand.
- If you have problems inserting the hearing aid, use the other hand to gently pull your earlobe downwards. This opens the ear canal and eases insertion of the hearing aid.

Removing a hearing aid:

NOTICE

Never pull on the battery compartment door to remove the hearing aid. This could damage your hearing aid.

- ▶ Push lightly on the back of your ear to loosen the hearing aid.
- ▶ Remove the hearing aid by pulling the removal cord forward.

CAUTION

Risk of injury!

- ▶ In very rare cases the ear piece could remain in your ear when removing the hearing aid. If this happens, have the ear piece removed by a medical professional.

Clean and dry your hearing aids after usage. Read more in section "Maintenance and care".

Adjusting the volume

Your hearing aids automatically adjust the volume to the listening situation.

- If you prefer manual volume adjustment, use a remote control option.

An optional signal can indicate the volume change.

Changing the hearing program

Depending on the listening situation, your hearing aids automatically adjust their sound.

Your hearing aids may also have several hearing programs which allow you to change the sound, if needed. An optional signal tone can indicate the program change.

- To change the hearing program manually, use a remote control option.

Refer to section "Hearing programs" for a list of your hearing programs.

Special listening situations

On the phone

When you are on the phone, turn the receiver slightly so that it does not completely cover your ear.

Telephone program

You may prefer a certain volume when using the phone. Ask your Hearing Care Specialist to configure a telephone program.

- Switch to the telephone program whenever you are on the phone.

If a telephone program is configured for your hearing aids, it is listed in section "Hearing programs".

Maintenance and care

To prevent damage it is important that you take care of your hearing aids and follow a few basic rules, which will soon become a part of your daily routine.

Hearing aids

Drying and storage

- ▶ Dry your hearing aids overnight.
- ▶ For longer periods of non-use, store your hearing aids in a dry environment with an open battery compartment and the batteries removed.
- ▶ Ask your Hearing Care Specialist for more information.

Cleaning

For hygiene reasons and to maintain functionality, clean your hearing aids daily.

- ▶ Clean your hearing aids daily with a soft, dry tissue.
- ▶ Never use running water or immerse the devices in water.
- ▶ Never apply pressure while cleaning.
- ▶ Ask your Hearing Care Specialist for recommended cleaning products, special care sets, or more information on how to keep your hearing aids in good condition.


Ear pieces

Cleaning


Cerumen (ear wax) may accumulate on the ear pieces. This may affect sound quality. Clean the ear pieces daily.

- ▶ Clean the ear piece right after removal with a soft and dry tissue.

This prevents cerumen from becoming dry and hard.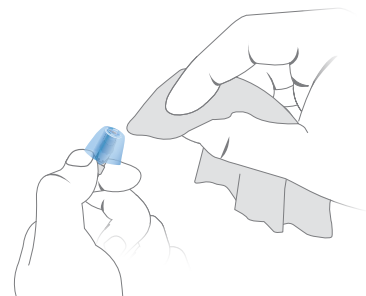

- ▶ Squeeze the tip of the ear piece.

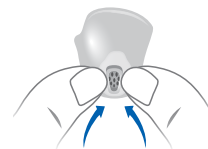
Exchanging

Exchange standard ear pieces approximately every three months. Replace them sooner if you notice cracks or other changes. The procedure for exchanging standard ear pieces depends on the type of ear piece. In section "Components and names", your Hearing Care Specialist has marked your type of ear piece.

Exchanging Click Sleeves


- Grip the Click Sleeve with your fingers and pull it off. If you cannot grip it, turn it inside out and then pull it off.

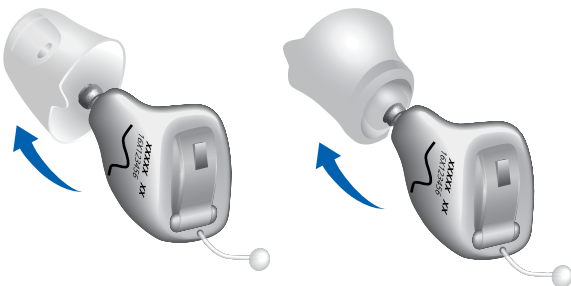- Take particular care that the new Click Sleeve clicks noticeably in place.


- When installing the new Click Sleeve, take care that the flap of the Click Sleeve matches the line printed on the device.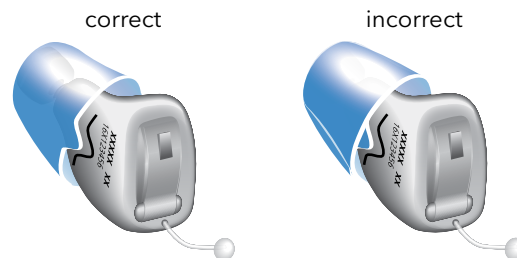

Exchanging Click Domes

If you have problems removing the old Click Dome, follow the instructions on the packaging of the Click Domes.

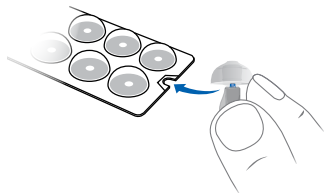- Take particular care that the new Click Dome attaches with a noticeable “click”.

Professional maintenance

Your Hearing Care Specialist can perform a thorough professional cleaning and maintenance.

Ask your Hearing Care Specialist for recommended maintenance intervals and support.

Further information

Safety information

For more safety information, refer to the safety manual supplied with the device.

Accessories

Available accessories depends on the hearing aid type. Examples are: a remote control, an audio streaming device, or a smartphone app for remote control.

Some hearing aid types support CROS and BiCROS solutions for people with single-sided deafness.

Consult your Hearing Care Specialist for further information.

Symbols used in this document

Points out a situation that could lead to serious, moderate, or minor injuries.

Indicates possible property damage.

Advice and tips on how to handle your device better.

Troubleshooting

Problem and possible solutions

Sound is weak.

- Gently close the battery compartment completely.
 - Clean the hearing aid.
 - Replace the empty battery.
 - Clean or exchange the ear piece.
-

Hearing aid emits whistling sounds.

- Reinsert the hearing aid until it fits securely.
 - Clean or exchange the ear piece.
-

Sound is distorted.

- Replace the empty battery.
 - Clean or exchange the ear piece.
-

Hearing aid emits signal tones.

- Replace the empty battery.
-

Hearing aid does not work.

- Turn the hearing aid on.
 - Gently close the battery compartment completely.
 - Replace the empty battery.
 - Make sure the battery is inserted correctly.
 - Power-on delay is active. Wait several seconds and check again.
-

Consult your Hearing Care Specialist if you encounter further problems.

Country-specific information

Certification information for hearing aids with wireless functionality:

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Copyright ©2020 Miracle-Ear, Inc.
Distributed By Miracle-Ear, Inc.
Derechos de autor ©2020 Miracle-Ear, Inc.